## SCHULTERBANDAGE OMO-ACTIVE / SHOULDER BRACE OMO-ACTIVE / ÉPAULIÈRE OMO-ACTIVE

DE AT CH

### SCHULTERBANDAGE OMO-ACTIVE

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GB IE NI

### SHOULDER BRACE OMO-ACTIVE

Instructions for use (3-5)

FR BE

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Mode d'emploi (5-6)

NL BE

### SCHOUDERBANDAGE OMO-ACTIVE

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PL

### BANDAŻ NA BARK OMO-ACTIVE

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CZ

### BANDAŽ NA RAMENO OMO-ACTIVE

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SK

### RAMENNÁ BANDÁŽ OMO-ACTIVE

Návod na použitie (11-13)

ES

### VENDAJE PARA HOMBRO OMO-ACTIVE

Instrucciones de uso (13-15)

DK

### SKULDERBANDAGE OMO-ACTIVE

Brugsanvisning (15-16)

IT

### FASCIA PER SPALLA OMO-ACTIVE

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HU

### VÁLLBANDÁZS OMO-ACTIVE

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SI

### BANDAŽA ZA RAMO OMO-ACTIVE

Navodila za uporabo (20-21)

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Artikelnummer

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Hersteller

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Congratulations on the purchase of your new product. You have chosen a high-quality article. The instructions for use are part of this product. They contain important information on safety and use. Please familiarise yourself with the instructions and safety information before using this product. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the brace and be sure to include them when handing on the product to another person.

## Medical device

### Only intended for private use!

### Note:

The medical device is intended for adults who are able to read, understand and independently follow these instructions for use. A reason for use, as named under Indications, must be present at the user.

The item is not intended for groups of persons who are excluded from its application by the sizes and contraindications listed below.

## Scope of delivery

- 1 x shoulder brace
- 1 x pad
- 1 x elastic strap
- 1 x instructions for use

## Technical Data

Size	Upper arm circumference
S	23 – 27 cm
M	27 – 32 cm
L	32 – 37 cm

To be worn on the left and right

## Intended purpose

The brace stabilises, supports and relieves.

The height-adjustable and individually positionable pad supports the stabilisation of the shoulder joint.

## Indications

To be used as support for:

- Shoulder joint injuries
- Joint pain due to wear (omarthrosis)
- Irritation (chronic, post-traumatic or postoperative)
- Mild joint instabilities

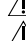
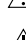
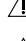
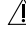
## Contraindications

 **Do not use – or use only after consultation with your doctor – in case of:**

- External injuries
- Complex chronic ligament instabilities
- Circulatory disorders
- Lymphatic drainage disorders
- Sensitivity and movement disorders of the arms (e.g. in diabetes mellitus)
- Eczema or other skin diseases

## Safety instructions

### Risk of injury

-  If problems occur while wearing the bandage, remove it immediately and consult a doctor.
-  Check the bandage before each use for tears and damage.
-  The product should only be used in perfect condition.
-  Make sure the bandage is comfortable but not too tight.
-  Never put the bandage on so tightly that the blood circulation is restricted.
-  Do not wear the bandage longer than 2 to 3 hours at a stretch and a maximum of 6 to 8 hours a day.
-  Do not wear the bandage while sleeping or during longer rest phases such as long periods of sitting.
-  Wash the areas of skin covered by the bandage thoroughly and regularly.
-  To avoid allergic skin reaction, the skin must be free of grease, gels, creams, oils or similar materials.
-  The article may only be used by other people when it has been cleaned.

### Possible side effects

-  If you experience symptoms (such as, e.g. tingling, pain, swelling) while wearing the bandage, remove the bandage immediately!
-  There is a risk of skin irritation and skin irritations! In individual cases, skin irritation and reactions and allergic reactions may occur. Make sure that the stated maximum wearing periods are not exceeded and the areas of skin covered by the bandage are thoroughly and regularly washed. Should any problems or complaints occur, consult a doctor immediately!

## Putting the bandage on

Try to avoid moving the affected arm while the brace is being applied.

- Open both hook-and-loop fasteners on the elastic strap and make sure that the middle of the elastic strap is in the middle of the brace.
- Close the rear hook-and-loop fasteners at the height of the centre of the brace (**Fig. 1**).
- Pull the front strap end towards the front eyelet and attach the front hook-and-loop fastener just behind the eyelet (**Fig. 2**).
- Hold the brace with the opening towards you and turn the strap outwards (**Fig. 3**).
- Guide the affected arm through the brace until the shoulder is well wrapped (**Fig. 4a**). Position the brace so that the upper ends of the V-shaped pad lie on the anterior and posterior bony protrusion of the shoulder, respectively. If necessary, correct the fit of the brace or pad (**Fig. 4b**).
- Insert the hand of the opposite arm into the strap loop from the outside and pull the arm back until the loop is under the armpit (**Fig. 5a**). Then grasp the part of the strap now lying in front of the arm and guide it over your head (**Fig. 5b**).
- Loosen the front hook-and-loop fastener and pull the strap until the desired tension is reached. Close the hook-and-loop fastener. (**Fig. 6**).

## Taking off the brace

- Open the front hook-and-loop fastener and loosen the strap.
- To avoid damage to your clothing or the brace, close the hook-and-loop fastener just behind the front eyelet.
- Grasp the lower edge of the brace and pull it down.
- Then pull the strap over your head.

## Cleaning and care

- Before using the brace for the first time, you should wash it thoroughly. Remove the pad and the elastic strap before washing.
- It is recommended that you wash it again after every two or three uses.
- After washing your brace you should rinse it thoroughly.
- Wash separately.
- Wash the pad separately in a laundry bag.

## Instructions for disposal

Please dispose of the product in an environmentally-friendly way through an approved disposal centre or your local waste facility. Observe currently applicable regulations.

## Obligation to notify

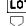
All serious incidents relating to the product must be reported immediately to the manufacturer and the competent authority.

## 3-year warranty

The product has been created with great care and under constant monitoring. You will receive a 3-year warranty for this product from the date of purchase. Please keep the receipt. The warranty is valid only for material and manufacturing faults and lapses in the event of misuse or improper handling. Your statutory rights, particularly the warranty rights, are not restricted by this warranty. If you have any complaints, please contact the service hotline

below or contact us by email. Our service staff will discuss how to proceed further with you as soon as possible. In any case, we will personally advise you. The warranty period is not extended by any repairs under the terms of the warranty, statutory warranty or goodwill. This also applies to replaced and repaired parts. After expiry of the warranty, incurred repairs are chargeable.

Service Germany  
Tel.: +49 (0)40 82 22 44-80  
E-Mail: info@horizonte.de

-  Consult instructions for use
-  Attention: Always adhere to the stated warning information
-  Catalogue number
-  Batch designation
-  Use by
-  Date of manufacture (year): 2022
-  Manufacturer
-  CE Marking of conformity
-  Medical device

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