www.lidl-service.com

ALUMINIUM GRIPPER

(GB) (IE) (NI)

ALUMINIUM GRIPPER

Instructions for use

(FR) (BE)

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(DE) (AT) (CH)

ALUMINIUM-GREIFHILFE

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(DK)

GRIBETANG I ALUMINIUM

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(NL) (BE)

ALUMINIUM GRIJPHULP

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

IAN 312092

(GB) (IE) (NI) (DK) (FR)
(BE) (NL) (DE) (AT) (CH)

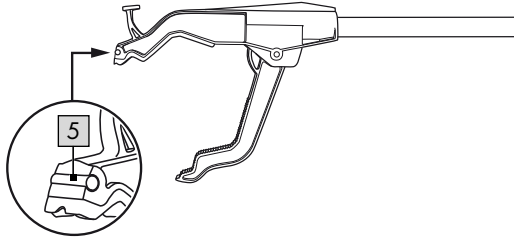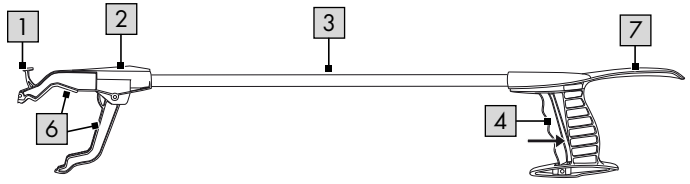A

Congratulations!

With your purchase you have decided on a high-quality product. Get to know the product before you start to use it.

 **Carefully read the following instructions for use.**

Use the product only as described and only for the given areas of application. Keep these instructions safe. When passing the product on to a third party, always make sure that the documentation is included.

This item conforms to Directive 93/42/EEC on medical devices.

Please use our service if you have questions about the item or have trouble reading these instructions.

Package contents

1 x aluminium gripper
1 x instructions for use

Technical data

Maximum load: 1 kg
Length: approx. 820mm
Weight: approx. 0.192kg

Symbols used

 Follow the instructions for use

 Date of manufacture (year-month):
2019 - 02

 Batch number

 Manufacturer

 For indoor use

 Maximum load

Intended use

The item is intended as an aid for gripping, picking up or lifting small items up to 1 kg for people who have ailments or disabilities that have led to a limitation of the action range of their upper extremities. They must still have some of their gripping ability. The item is intended for adults for private use and is not suitable for commercial purposes.

The expected service life of the item if used as intended and in compliance with the safety and care instructions is up to 3 years. Beyond this period, the item may continue to be used as long as it is in good and safe condition.

Safety information

WARNING!

Risk of suffocation for children!

Do not allow children to play with the item or the packaging materials unsupervised.

- Only use the item for its intended purpose.
- The item may only be used by persons who are aware of the safety information.
- Check the item for damage or wear before each use.
- Only use the item if it is in perfect condition! Do not use the item if it is visibly damaged or you suspect it is damaged!
- Only use the item if the temperature of the surfaces is safe (between 10 and 40 degrees Celsius).
The surface of the item can reach temperatures that can cause injury through hot water, sun exposure, sauna conditions, extreme cold, etc.
- Do not modify the item and do not repair it yourself! Only purchase replacement parts from the manufacturer. Improper repairs can cause unforeseen harm!

- Do not use the gripper on people or animals, and do not use it as a baton or for a similar purpose!
- Keep the gripping arm away from your eyes!
- Do not try to grip hot or fragile items!

Using the item (Fig. A)

Gripping head (2)

Note: The gripping head (2) can be rotated 360° around the aluminium shaft (3).

1. Position the gripping arms (6) around the object you want to pick up.
2. Pull the handle (4) in the direction of the arrow (Fig. A) and hold it in this position. The gripping arms will be pressed together and will grasp the object.
3. Move the item to the desired position and release the handle again. The gripping arms will open and the item will be released.

Magnet (5)

For picking up keys or other metal objects.

Hook (1)

For example, to help open drawers.

Shoehorn (7)

As an entry aid and for sliding on shoes more easily.

Care, maintenance

CAUTION!

Do not use corrosive or harsh cleaning agents or sponges.

These could damage the surface.

1. Clean the item as needed with a damp cloth and use a mild detergent if required.
2. Afterwards, dry the item completely before using it again.

Disposal

Dispose of the article and the packaging materials in accordance with current local regulations. Packaging materials such as foil bags are not suitable to be given to children. Keep the packaging materials out of the reach of children.

Dispose of the products and the packaging in an environmentally friendly manner.

The recycling code is used to identify various materials for recycling.

The code consists of the recycling symbol – which is meant to reflect the recycling cycle – and a number which identifies the material.

Notes on the guarantee and service handling

The product was produced with great care and under constant supervision. You receive a three-year warranty for this product from the date of purchase. Please retain your receipt.

The warranty applies only to material and workmanship and does not apply to misuse or improper handling. Your statutory rights, especially the warranty rights, are not affected by this warranty.

With regard to complaints, please contact the following service hotline or contact us by e-mail. Our service employees will advise as to the subsequent procedure as quickly as possible. We will be personally available to discuss the situation with you. Any repairs under the warranty, statutory guarantees or through goodwill do not extend the warranty period. This also applies to replaced and repaired parts.

Repairs after the warranty are subject to a charge.

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