KNIEGELENKBANDAGE / KNEE BRACE / GENOUILLÈRE

(DE) (AT) (CH)

KNIEGELENKBANDAGE

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(GB) (IE) (NI) (MT)

KNEE BRACE

Instructions for use (3-4)

(FR) (BE) (CH)

GENOUILLÈRE

Mode d'emploi (4-5)

(NL) (BE)

KNIEBANDAGE

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(PL)

OPASKA NA KOLANO

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(CZ)

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(SK)

BANDÁŽ NA KOLENNÝ KÍLB

Návod na použitie (10-11)

(ES)

RODILLERA

Instrucciones de uso (12-13)

(DK)

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Brugsanvisning (13-14)

(IT) (MT) (CH)

GINOCCHIERA ELASTICA

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(HU)

TÉRDBANDÁZS

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IAN 452675_2310

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GB **IE** **NI** **MT**

Congratulations on the purchase of your new product. You have chosen a high-quality article. The instructions for use are part of this product. They contain important information on safety and use. Please familiarise yourself with the instructions and safety information before using this product. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the product and be sure to include them when handing on the product to another person.

Medical device

Only intended for private use!

Note:

The medical device is intended for adults who are able to read, understand and independently follow these instructions for use. A reason for use, as named under Indications, must be present at the user.

The item is not intended for groups of persons who are excluded from its application by the contraindications listed below.

Scope of delivery

- 1 x knee brace
- 2 x spiral springs
- 1 x instructions for use

Technical Data

Size	Circumference above the knee	Circumference below the knee
S	44 – 47 cm	34 – 37 cm
M	47 – 50 cm	37 – 40 cm
L	50 – 53 cm	40 – 43 cm
XL	53 – 56 cm	43 – 46 cm

To be worn on the left and right

Intended purpose

The brace stabilises, supports and relieves.

Indications

To be used as support for:

- Injuries to the knee (e.g. strains, sprains or bruises)
- Joint pain due to degradation (gonarthrosis)
- Meniscus damage
- Mild joint instability
- Damage from overuse (“patella apex syndrome”, “Osgood-Schlatter disease”)
- Patella conditions (patella dysplasia following dislocation)

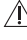
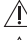
Contraindications

 **Do not use – or use only after consultation with your doctor – in case of:**

- External injuries
- Complex chronic ligament instabilities
- Circulatory disorders.
- Freshly torn ligaments (cruciate or collateral ligament)
- Eczema or other skin diseases

Safety instructions

Risk of injury

-  If problems occur while wearing the bandage, remove it immediately and consult a doctor.
-  Check the bandage before each use for tears and damage.
-  The product should only be used in perfect condition.
-  Make sure the bandage is comfortable but not too tight.
-  Never put the bandage on so tightly that the blood circulation is restricted.
-  Do not wear the bandage longer than 2 to 3 hours at a stretch and a maximum of 6 to 8 hours a day.

-  Do not wear the bandage while sleeping or during longer rest phases such as long periods of sitting.
-  Wash the areas of skin covered by the bandage thoroughly and regularly.
-  To avoid allergic skin reaction, the skin must be free of grease, gels, creams, oils or similar materials.
-  The article may only be used by other people when it has been cleaned.

Possible side effects

-  If you experience symptoms (such as, e.g. tingling, pain, swelling) while wearing the bandage, remove the bandage immediately!
-  There is a risk of skin irritation and skin irritations! In individual cases, skin irritation and reactions and allergic reactions may occur. Make sure that the stated maximum wearing periods are not exceeded and the areas of skin covered by the bandage are thoroughly and regularly washed. Should any problems or complaints occur, consult a doctor immediately!

Positioning of the product

- Pull the bandage from below to above the knee joint so that the ring-shaped gel pad is positioned in front (**Fig. 1**).
- Position the bandage so that the knee is enclosed by the ring-shaped pad (**Fig. 2**).
- Make sure that the bandage is worn wrinkle-free.

Cleaning and care

- Before using it for the first time, the bandage should be thoroughly washed. To do this, remove the spiral springs (**Fig. 3**) on both sides before washing.
- After using it two or three times, it should be washed again.
- Reinsert the spiral springs after washing.
- Wash separately.

Instructions for disposal

Please dispose of the product in an environmentally-friendly way through an approved disposal centre or your local waste facility. Observe currently applicable regulations.

Obligation to notify

All serious incidents relating to the product must be reported immediately to the manufacturer and the competent authority.

3-year warranty

The product has been created with great care and under constant monitoring. You will receive a 3-year warranty for this product from the date of purchase. Please keep the receipt. The warranty is valid only for material and manufacturing faults and lapses in the event of misuse or improper handling. Your statutory rights, particularly the warranty rights, are not restricted by this warranty. If you have any complaints, please contact the service hotline below or contact us by email. Our service staff will discuss how to proceed further with you as soon as possible. In any case, we will personally advise you. The warranty period is not extended by any repairs under the terms of the warranty, statutory warranty or goodwill. This also applies to replaced and repaired parts. After expiry of the warranty, incurred repairs are chargeable.

Service Germany
Tel.: +49 (0)40 82 22 44-80
E-Mail: support@horizonte.de

-  Consult instructions for use
-  Attention: Always adhere to the stated warning information
-  Catalogue number
-  Batch designation
-  Use by
-  Date of manufacture (year): 2024
-  Manufacturer
-  CE Marking of conformity
-  Medical device

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