POSTURE TRAINER

(GB) (IE) (NI)

POSTURE TRAINER

Instructions for use (2-3)

(NL)

HOUDINGSTRAINER

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(DK)

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Brugsanvisning (4-5)

(DE) (AT) (CH)

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IAN 467443_2404

Congratulations on the purchase of your new product. You have chosen a high-quality article. The instructions for use are part of this product. They contain important information on safety and use. Please familiarise yourself with the instructions and safety information before using this product. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the brace and be sure to include them when handing on the product to another person.

Medical device

Only intended for private use!

Note

The medical device is intended for adults who are able to read, understand and independently follow these instructions for use. A reason for use, as named under Indications, must be present at the user. The item is not intended for groups of persons who are excluded from its application by the sizes and contraindications listed below.

Scope of delivery

- 1 x posture trainer
- 1 x instructions for use

Technical Data

Size	Chest measurement
S – M	84 – 100 cm
L – XL	100 – 116 cm

Intended purpose

Can help to prevent poor posture and tense muscles. Regular use can help to maintain an upright posture.

Indications

To be used as support for:

- muscle tension
- muscular imbalance

Contraindications

 **Do not use – or use only after consultation with your doctor – in case of:**

- you are unsure about how to use the product
- acute injuries
- unforeseen problems arise
- circulatory disorders.

Safety instructions

Risk of injury

-  If problems occur while wearing the bandage, remove it immediately and consult a doctor.
-  Check the bandage before each use for tears and damage.
-  Use the product only in perfect condition, as described and for the specified area of use.
-  Wear a T-shirt or a long-sleeved shirt under the brace.
-  Never put the bandage on so tightly that the blood circulation is restricted.

-  Do not wear the bandage longer than 2 to 3 hours at a stretch and a maximum of 6 to 8 hours a day.
-  Do not wear the bandage while sleeping or during longer rest phases such as long periods of sitting.
-  The bandage may only be used on uninjured skin.
-  To avoid allergic skin reaction, the skin must be free of grease, gels, creams, oils or similar materials.

Putting the bandage on

- Before putting it on the hook-and-loop fasteners should be loosened, but the straps should still be fed through the buckles
- Put the shoulder belts on as you would a jacket by inserting your arms through both loops (**fig. 1**).
- Tension the brace by pulling fairly gently on the ends of the belts. Use the existing loops if necessary (**fig. 2**).
- Fasten the hook-and-loop fasteners on the respective shoulder belt (**fig. 3**).
- The bandage should sit comfortably and be slightly stretched.

Cleaning and care

- Before using it for the first time, the bandage should be thoroughly washed.
- Close the Velcro pieces during the washing process.
- Wash separately or with similar colours.

Instructions for disposal

Please dispose of the product in an environmentally-friendly way through an approved disposal centre or your local waste facility. Observe currently applicable regulations.

Obligation to notify

All serious incidents relating to the product must be reported immediately to the manufacturer and the competent authority.

3-year warranty

The product has been created with great care and under constant monitoring. You will receive a 3-year warranty for this product from the date of purchase. Please keep the receipt. The warranty is valid only for material and manufacturing faults and lapses in the event of misuse or improper handling. Your statutory rights, particularly the warranty rights, are not restricted by this warranty. If you have any complaints, please contact the service hotline below or contact us by email. Our service staff will discuss how to proceed further with you as soon as possible. In any case, we will personally advise you. The warranty period is not extended by any repairs under the terms of the warranty, statutory warranty or goodwill. This also applies to replaced and repaired parts. After expiry of the warranty, incurred repairs are chargeable.

Service Germany
Tel.: +49 (0)40 82 22 44-80
E-Mail: support@horizonte.de

-  Consult instructions for use
-  Attention: Always adhere to the stated warning information
-  Catalogue number
-  Batch designation
-  Use by
-  Date of manufacture (year): 2024
-  Manufacturer
-  CE Marking of conformity
-  Medical device

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