KNIEGELENKBANDAGE AIRCON AIRCON KNEE SUPPORT GENOUILLÈRE AIRCON

(DE) (AT) (CH)

KNIEGELENKBANDAGE AIRCON

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(GB) (IE) (NI) (MT)

AIRCON KNEE SUPPORT

Instructions for use (3)

(FR) (BE) (CH)

GENOUILLÈRE AIRCON

Mode d'emploi (4)

(NL) (BE)

KNIEBANDAGE AIRCON

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(CZ)

BANDÁŽ NA KOLENNÍ KLOUB AIRCON

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(PL)

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(SK)

KOLENNÁ BANDÁŽ AIRCON

Návod na použitie (8)

(ES)

RODILLERA AIRCON

Instrucciones de uso (9)

(DK)

KNÆLEDSBANDAGE AIRCON

Brugsanvisning (10)

(IT) (MT) (CH)

GINOCCHIERA ELASTICA AIRCON

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(HU)

AIRCON TÉRDBANDÁZS

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IAN 467438_2404

(DE) (AT) (FR) (NL) (BE)
(CZ) (PL) (SK) (DK) (IT) (HU)

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Congratulations on the purchase of your new product.

You have chosen a high-quality article. The instructions for use are part of this product. They contain important information on safety and use. Please familiarise yourself with the instructions and safety information before using this product. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the brace and be sure to include them when handing on the product to another person.

Medical device

Only intended for private use!

Note

The medical device is intended for adults who are able to read, understand and independently follow these instructions for use. A reason for use, as named under Indications, must be present at the user. The item is not intended for groups of persons who are excluded from its application by the sizes and contraindications listed below.

Scope of delivery

- 1 x knee brace
- 1 x instructions for use

Technical Data

- One size fits all Knee circumference
- 35 - 41 cm

Intended purpose

Stabilising, warming bandage to effectively support the joints and ligaments.

Indications

- irritation
- swelling
- post injury
- rheumatic conditions

Contraindications

Do not use – or use only after consultation with your doctor – in case of:

- acute injuries
- circulatory disorders.
- eczema or other skin diseases

Quit the application in case

- unforeseen problems arise
- you are unsure about how to use the product

Safety Advice

Risk of Injury

Use the product only in perfect condition, as described and for the specified area of use.

Do not wear the bandage longer than 2 to 3 hours at a stretch and a maximum of 6 to 8 hours a day.

Pay attention to a sufficient blood circulation.

The bandage may only be used on uninjured skin.

To avoid allergic skin reaction., the skin must be free of grease, gels, creams, oils or similar materials.

Instructions for disposal

Please dispose of the product in an environmentally-friendly way through an approved disposal centre or your local waste facility. Observe currently applicable regulations.

Obligation to notify

All serious incidents relating to the product must be reported immediately to the manufacturer and the competent authority.

3-year warranty

The product has been created with great care and under constant monitoring. You will receive a 3-year warranty for this product from the date of purchase. Please keep the receipt. The warranty is valid only for material and manufacturing faults and lapses in the event of misuse or improper handling. Your statutory rights, particularly the warranty rights, are not restricted by this warranty. If you have any complaints, please contact the service hotline below or contact us by email. Our service staff will discuss how to proceed further with you as soon as possible. In any case, we will personally advise you. The warranty period is not extended by any repairs under the terms of the warranty, statutory warranty or goodwill. This also applies to replaced and repaired parts. After expiry of the warranty, incurred repairs are chargeable.

Service Germany

Tel.: +49 (0)40 82 22 44-80

E-Mail: support@horizonte.de

Consult instructions for use

Attention: Always adhere to the stated warning information

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
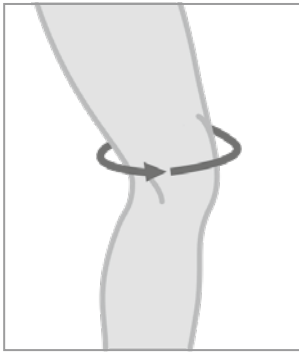
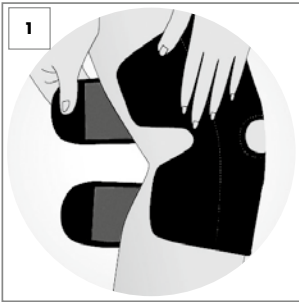
GB CY Putting the bandage on

The easiest way to put the bandage on is while sitting.

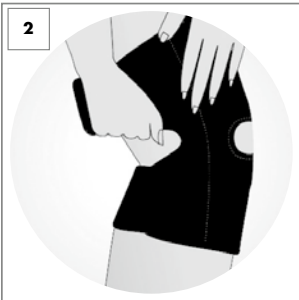- Place the opening of the bandage on your kneecap (Fig. 1).
- Pass the lower part of the bandage around the lower leg with a little tension and close the Velcro fastener.
- Then pass the upper part of the bandage around the thigh with a little tension and close the Velcro fastener here as well (Fig. 2).
- Adjust the position of the Velcro straps until the bandage is slightly taut but not uncomfortable.

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