www.lidl-service.com

## AIRCON WRIST SUPPORT

**GB** **IE** **NI**

### AIRCON WRIST SUPPORT

Instructions for use

**DK**

### HÅNDLEDSBANDAGE AIRCON

Brugsanvisning

**FR** **BE**

### BANDAGE DE POIGNET AIRCON

Mode d'emploi

**NL** **BE**

### POLSBANDAGE AIRCON

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**DE** **AT** **CH**

### HANDGELENKBANDAGE

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IAN 307367

**GB** **IE** **NI** **DK**

**BE** **NL** **DE** **AT** **CH**

Congratulations on the purchase of your new product. You have chosen a high-quality article. The instructions for use are part of this product. They contain important information on safety and use. Please familiarise yourself with the instructions and safety information before using this product. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the brace and be sure to include them when handing on the product to another person.

### Only intended for private use!

## Scope of delivery

- 1 x wrist support
- 1 x instructions for use

## Intended use

Stabilising, warming bandage to effectively support the joints and ligaments. The bandage can be used on both sides.

### Function:

- Increases physical resilience
- Can help to improve circulation
- Helps prevent temperature reduction and can decrease the risk of injuries
- Stabilises the joint and can alleviate aches and pain

## Usage restrictions

 **Do not use – or use only after consultation with your doctor – for:**

- you are unsure about how to use the product
- you have acute injuries
- unforeseen problems arise

-  Use the product only in perfect condition, as described and for the specified area of use.
-  Do not wear the bandage longer than 2 to 3 hours at a stretch and a maximum of 6 to 8 hours a day.
-  Pay attention to a sufficient blood circulation.
-  The bandage may only be used on uninjured skin.
-  To avoid allergic skin reaction., the skin must be free of grease, gels, creams, oils or similar materials.

## Putting the bandage on

- Insert your thumb into the thumb hole of the bandage. The bandage should rest on the back of your hand (**Fig. 1**).
- Put the bandage around the wrist with a little tension until it is slightly taut but not uncomfortable (**Fig. 2**).
- Close the Velcro fastener.

## Cleaning and care

- Before using it for the first time, the bandage should be thoroughly washed.
- Close the Velcro pieces during the washing process.
- Wash separately or with similar colours.

## Instructions for disposal

Please dispose of the product in an environmentally-friendly way through an approved disposal centre or your local waste facility. Observe currently applicable regulations.

## 3-year warranty

The product has been created with great care and under constant monitoring. You will receive a 3-year warranty for this product from the date of purchase. Please keep the receipt. The warranty is valid only for material and manufacturing faults and lapses in the event of misuse or improper handling. Your statutory rights, particularly the warranty rights, are not restricted by this warranty. If you have any complaints, please contact the service hotline below or contact us by email. Our service staff will discuss how to proceed further with you as soon as possible. In any case, we will personally advise you. The warranty period is not extended by any repairs under the terms of the warranty, statutory warranty or goodwill. This also applies to replaced and repaired parts. After expiry of the warranty, incurred repairs are chargeable.

Service Germany

Tel.: +49 (0)40 82 22 44-80

E-Mail: [info@horizonte.de](mailto:info@horizonte.de)

Consult instructions for use

Always adhere to the stated warning information

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