AIRCON KNEE SUPPORT

GB IE NI

AIRCON KNEE SUPPORT

Instructions for use

DK

KNÆBANDAGE AIRCON

Brugsanvisning

FR BE

GENOUILÈRE AIRCON

Mode d'emploi

NL BE

KNIEBANDAGE AIRCON

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DE AT CH

KNIEGELENKBANDAGE AIRCON

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IAN 307373

GB IE NI DK

BE NL DE AT CH

Congratulations on the purchase of your new product. You have chosen a high-quality article. The instructions for use are part of this product. They contain important information on safety and use. Please familiarise yourself with the instructions and safety information before using this product. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the brace and be sure to include them when handing on the product to another person.

Only intended for private use!

Scope of delivery

- 1 x knee support
- 1 x instructions for use

Intended use

Stabilising, warming bandage to support the joints and ligaments. The brace can be used on both sides.

Function:

- Increases physical resilience
- Can help to improve circulation
- Helps prevent temperature reduction and can decrease the risk of injuries
- Stabilises the joint and can alleviate aches and pain

Usage restrictions

Do not use – or use only after consultation with your doctor – for:

- you are unsure about how to use the product
- you have acute injuries
- unforeseen problems arise

-  Use the product only in perfect condition, as described and for the specified area of use.
-  Do not wear the bandage longer than 2 to 3 hours at a stretch and a maximum of 6 to 8 hours a day.
-  Pay attention to a sufficient blood circulation.
-  The bandage may only be used on uninjured skin.
-  To avoid allergic skin reaction, the skin must be free of grease, gels, creams, oils or similar materials.

Putting the bandage on

The easiest way to put the bandage on is while sitting.

- Place the opening of the bandage on your kneecap (**Fig. 1**).
- Pass the lower part of the bandage around the lower leg with a little tension and close the Velcro fastener.
- Then pass the upper part of the bandage around the thigh with a little tension and close the Velcro fastener here as well (**Fig. 2**).
- Adjust the position of the Velcro straps until the bandage is slightly taut but not uncomfortable.

Cleaning and care

- Before using it for the first time, the bandage should be thoroughly washed.
- Close the Velcro piece during the washing process.
- Wash separately or with similar colours.

Instructions for disposal

Please dispose of the product in an environmentally-friendly way through an approved disposal centre or your local waste facility. Observe currently applicable regulations.

3-year warranty

The product has been created with great care and under constant monitoring. You will receive a 3-year warranty for this product from the date of purchase. Please keep the receipt. The warranty is valid only for material and manufacturing faults and lapses in the event of misuse or improper handling. Your statutory rights, particularly the warranty rights, are not restricted by this warranty. If you have any complaints, please contact the service hotline below or contact us by email. Our service staff will discuss how to proceed further with you as soon as possible. In any case, we will personally advise you. The warranty period is not extended by any repairs under the terms of the warranty, statutory warranty or goodwill. This also applies to replaced and repaired parts. After expiry of the warranty, incurred repairs are chargeable.

Service Germany

Tel.: +49 (0)40 82 22 44-80

E-Mail: info@horizonte.de

-  Consult instructions for use
-  Always adhere to the stated warning information
-  Batch designation
-  Durability
-  Manufacturer
-  Complies with Directive 93/42/EEC

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