## **PREMIUM MASSAGE GUN / PISTOLET DE MASSAGE PREMIUM CMGB B2**

**GB IE NI MT**

### **PREMIUM MASSAGE GUN**

Operating instructions

**FR BE CH**

### **PISTOLET DE MASSAGE PREMIUM**

Mode d'emploi

**IT CH MT**

### **PISTOLA MASSAGGIANTE PREMIUM**

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**PT**

### **PISTOLA DE MASSAGEM PREMIUM**

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**DE AT BE CH**

### **PREMIUM MASSAGE GUN**

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**NL BE**

### **PREMIUM MASSAGE GUN**

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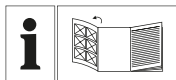
**ES**

### **PISTOLA DE MASAJE PREMIUM**

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**IAN 509807\_2507**

**GB IE NI DE AT BE  
CH FR NL ES IT PT**

GB IE NI MT

Before reading, unfold the page containing the illustrations and familiarise yourself with all functions of the device.

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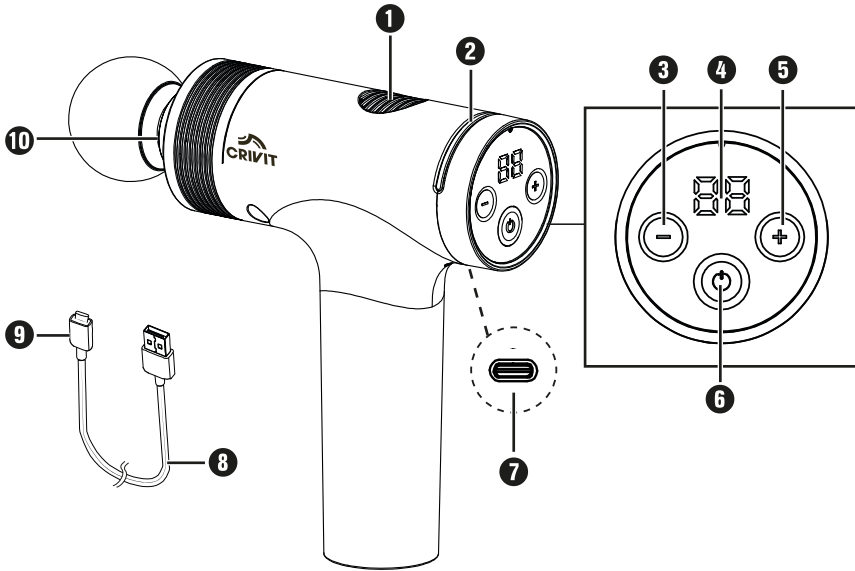
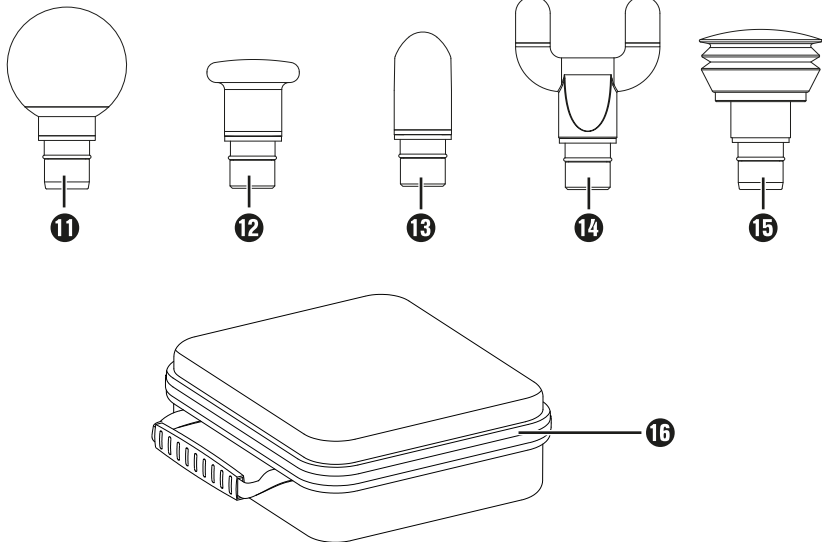
**A****B**

# Contents

<b>1. Introduction</b>	<b>2</b>
1.1. Intended use . . . . .	2
1.2. Warnings and symbols used. . . . .	2
<b>2. Safety information</b>	<b>3</b>
2.1. Safety guidelines for rechargeable batteries . . . . .	5
<b>3. Package contents</b>	<b>6</b>
<b>4. Parts description</b>	<b>6</b>
<b>5. Charging the integrated battery</b>	<b>7</b>
<b>6. Using the appliance</b>	<b>7</b>
6.1. Selecting/fitting attachments. . . . .	8
6.2. Examples of use . . . . .	8
6.3. Switching the appliance on/off and selecting intensity . . . . .	9
6.4. Intensity levels. . . . .	9
<b>7. Cleaning and care</b>	<b>10</b>
<b>8. Storage</b>	<b>10</b>
<b>9. Troubleshooting</b>	<b>10</b>
<b>10. Ordering replacement parts</b>	<b>10</b>
<b>11. Disposal</b>	<b>11</b>
11.1. Disposal of appliance and packaging . . . . .	11
<b>12. Technical specifications</b>	<b>11</b>
<b>13. Kompernass Handels GmbH warranty</b>	<b>11</b>
13.1. Service . . . . .	12
13.2. Importer . . . . .	12

# 1. Introduction

Congratulations on the purchase of your new appliance.

You have selected a high-quality product. The operating instructions are part of this product. They contain important information about safety, usage and disposal. Before using the product, please familiarise yourself with all operating and safety instructions. Use the product only as described and for the specified range of applications. Please also pass these operating instructions on to any future owner.

## 1.1. Intended use

This appliance and its attachments are intended exclusively for massaging the muscles of the neck, shoulders, arms, back (excluding the spine), thighs, calves and buttocks. Only use the attachments intended for this purpose. Never use the appliance directly on bones.

This appliance is intended solely for personal, private use. It is not to be used for medical or commercial applications and is not a substitute for medical treatment. It is not intended for use in medical/therapeutic or commercial fields. Any other use or use beyond that specified is considered improper use.

## 1.2. Warnings and symbols used

The following warnings and symbols are used in these operating instructions, on the packaging and on the appliance (where applicable):

	<p><b>DANGER!</b> A warning with this symbol and the signal word "DANGER" indicates an imminently hazardous situation that will result in death or serious injury if not avoided.</p>
	<p><b>WARNING!</b> A warning with this symbol and the signal word "WARNING" indicates a potentially hazardous situation that could result in death or serious injury if not avoided.</p>

	<p><b>CAUTION!</b> A warning with this symbol and the signal word "CAUTION" indicates a potentially hazardous situation that could result in a minor or moderate injury if not avoided.</p>
	<p><b>ATTENTION!</b> A warning with this symbol and the signal word "ATTENTION" indicates a potential situation that could result in property damage if not avoided.</p>
	<p><b>Note:</b> A note identifies additional information that facilitates the use of the appliance.</p>
	<p>Read the instructions.</p>
	<p>DC current/voltage</p>
	<p>Use the device indoors only.</p>
	<p>Protection class III (protection through low voltage)</p>
	<p>detachable supply unit</p>
	<p>USB® is a registered trademark of USB Implementers Forum, Inc.</p>
	<p>This product satisfies the requirements of the applicable European and national regulations.</p>
	<p>Do not dispose of electrical appliances in the household waste!</p>
	<p>Dispose of the packaging in an environmentally-responsible manner.</p>

	<p>The packaging is made of recyclable materials. Observe the labelling on the packaging material when sorting for disposal: The material is labelled with abbreviations (a) and numbers (b) with the following meanings: 1-7: plastics, 20-22: paper and cardboard, 80-98: composites.</p>
  <b>ES/PT</b>	<p>The packaging contains paper and/or cardboard components.</p> <p>The packaging contains plastic and/or metal components.</p>

	<p>This mark confirms that the product complies with the product safety requirements applicable in the UK.</p>
<div style="display: flex; justify-content: space-between;"> <div data-bbox="582 199 632 263">  </div> <div data-bbox="644 199 884 295"> <p><b>FR</b> Cet appareil et ses accessoires se recyclent</p> <p>A DÉPOSER EN MAGASIN    A DÉPOSER EN TRIERIE</p> <p>OU</p> <p>Points de collecte sur <a href="http://www.quefairedesdechets.fr">www.quefairedesdechets.fr</a> Privilégiez la réparation ou le don de votre appareil !</p> </div> </div> <div style="display: flex; justify-content: space-between; margin-top: 10px;"> <div data-bbox="582 303 632 367">  </div> <div data-bbox="644 303 845 375"> <p><b>FR</b> ÉLÉMENTS D'EMBALLAGE</p> <p>BAC DE TRI</p> </div> </div> <div style="display: flex; justify-content: space-between; margin-top: 10px;"> <div data-bbox="582 391 632 454">  </div> <div data-bbox="644 391 1037 531"> <p><b>FR:</b> The product, its packaging and the operating instructions are recyclable.</p> <p>They are subject to an extended manufacturer responsibility and will be collected separately.</p> </div> </div>	

## 2. Safety information

### **⚠ DANGER! RISK OF ELECTROCUTION!**

- Make sure that the charging cable does not become wet or damp. Route the cable in such a way that it cannot be caught or damaged.
- Keep the appliance and the charging cable away from hot surfaces.
- Do not perform any repairs to the appliance. All repairs must be performed by our Customer Service Unit or by a qualified specialist technician.
- Never open the appliance housing. There are no parts inside the appliance which require user maintenance. In addition, this will invalidate the warranty.
- Switch off the appliance after each use and before cleaning it.

### **⚠ WARNING! RISK OF INJURY!**

- This appliance may be used by children aged 8 years and above and by persons with limited physical, sensory or mental capabilities or lack of experience and knowledge, provided that they are under supervision or have been told how to use the appliance safely and are aware of the potential risks.
- Cleaning and user maintenance tasks must not be carried out by children unless they are supervised.

- Children must not be allowed to play with the appliance.
- Never leave children unattended with the packaging material. Keep children away from the packaging. There is a risk of suffocation!
- People with pacemakers, artificial joints or electronic implants should consult their doctor before using the appliance.
- Do not use the appliance around your heart, face or genital area.
- Do not use the appliance if you suffer from one or more of the following diseases or medical conditions: circulatory disorders, varicose veins, open wounds, bruises, cracked skin, phlebitis, thrombosis, diabetes or slipped disc.
- Never use the appliance on children, animals, during pregnancy, on your head, throat or other parts of the body that are exhibiting swelling, burns, inflammation, wounds or other sensitive areas.
- Do not use the appliance under blankets or pillows. Never use the appliance in the vicinity of petrol or other inflammable materials.
- Do not use the appliance while sleeping or after taking substances that may reduce your ability to react quickly (e. g. painkillers or alcohol).
- Never use the appliance directly on bones, e. g. on your spine, shins, the tops of your feet or similar places on your body.
- Use the appliance only on dry and clean parts of your body.
- Stop using the appliance immediately if you find the massage unpleasant or if you experience pain.
- Uninterrupted application of the appliance should not last any longer than 15 minutes. After use, switch off the appliance for at least 15 minutes and allow it to cool down.
- Do not insert your fingers or any other objects into the opening for the attachments. Keep long hair away from moving parts.
- Use the appliance only for its intended purpose.

## **CAUTION! PROPERTY DAMAGE!**

-  Use the appliance only indoors.
- Use the appliance only with the supplied accessories. Make sure that the attachments can always move freely. Use only the supplied charging cable to charge the appliance.
- If you are using a USB power supply, the mains socket must always be easily accessible so that the USB power supply can be quickly disconnected from the socket in a hazardous situation. You should also observe the operating instructions for the USB power supply.
- Protect the appliance and attachments from shocks, dust, chemicals or electromagnetic fields. Keep the appliance away from heat sources (radiators, ovens, etc.) and do not expose it to extreme fluctuations of temperature.
- Do not use any aggressive cleaning agents or solvents. They could damage the surface.
- Keep the ventilation slits free of dust and do not cover them.

### **2.1. Safety guidelines for rechargeable batteries**

The product is equipped or supplied with a lithium-ion battery. The battery in this appliance cannot be replaced.

## **DANGER! RISK OF EXPLOSION!**

- Do not open or short-circuit the battery. This could cause the battery to overheat and explode. There is a risk of fire!
- Never throw the battery into a fire or water.
- Do not subject the battery to mechanical stress.
- Never leave the appliance unattended during charging.

## **⚠ WARNING! RISK OF INJURY DUE TO BATTERY LEAKAGE!**

- High temperatures can affect the battery. Avoid direct sunlight and proximity to heat sources such as radiators.
- Under extreme conditions, the battery cells may leak. Avoid contact with skin, eyes and mucous membranes. In the event of accidental contact, rinse the affected areas with plenty of clean water and contact a doctor.
- Leaking or damaged batteries can cause chemical burns to the skin. Always wear protective gloves if you have to touch the batteries.

## **⚠ ATTENTION! PROPERTY DAMAGE!**

- Never store the device when it is discharged. Prolonged storage in a discharged state can cause permanent damage to the battery. If the device is stored for a longer period of time, the charge level of the battery must be checked regularly. The optimum charge level is between 50 % and 80 %.

### **3. Package contents**

The appliance is supplied with the following components as standard:

- Premium massage gun
  - 5 attachments (1 ball, 1 conical, 1 U-shaped, 1 flat and 1 air cushion head)
  - Storage bag
  - USB charging cable
  - Operating instructions
- 1) Remove all parts of the appliance and the operating instructions from the box.
  - 2) Remove all packaging materials and any stickers from the appliance.

#### **i Note**

- ▶ Check contents for completeness and damage immediately after unpacking. Contact Customer **13.1. Service** if necessary.

### **4. Parts description**

Figure A (see the fold-out page):

- 1 Ventilation slits
- 2 Battery level indicator
- 3 Button **—** Decrease intensity level
- 4 Display
- 5 Button **+** Increase intensity level
- 6 ON/OFF button **⏻**
- 7 Connection socket
- 8 USB charging cable (USB type A to USB type C)
- 9 USB plug (type C)
- 10 Socket for attachments

Figure B:

- 11 Ball head
- 12 Flat head
- 13 Conical head
- 14 U-shaped head
- 15 Air cushion head
- 16 Storage bag

## 5. Charging the integrated battery

The battery must be completely charged before using the appliance for the first time.

### **⚠ DANGER! ELECTRIC SHOCK!**

- ▶ Only charge the appliance indoors in dry locations away from water. For example, do not charge the appliance next to or above a sink filled with water.

### **i Note**

- ▶ The USB charging cable **8** and connection socket **7** are used only to charge the integrated battery. They are not suitable for data transmission.
- ▶ Only charge the appliance with a USB power adapter with protection class II that is approved for use with household appliances and has an output voltage of 5 V  $\overline{=}$  and an output current of max. 2 A.
- ▶ Use only the supplied USB charging cable **8** to charge the integrated battery. Disconnect the USB charging cable **8** from the appliance when charging is complete.
- ▶ The operating time with a fully charged battery is about 2.5–4 hours, depending on the selected intensity level and attachment.
- ▶  $\overline{LH}$  lights up on the display **4** during charging. The appliance cannot be used during charging.
- ▶ Charge the battery immediately as soon as  $\overline{LD}$  appears in the display **4** and/or the battery level indicator **2** lights up red.
  - 1) Make sure that the appliance is switched off.
  - 2) Insert the USB plug **9** into the USB connection socket **7**.
  - 3) Connect the USB charging cable **8** to a suitable mains adapter and plug the mains adapter into a mains socket.
    - ✓ When the battery level indicator **2** lights up permanently green, the appliance is fully charged.

The battery level indicator **2** has the following meanings:

Battery level indicator <b>2</b>		Battery charge
In operation	Flashes red	< 10 %
	Lights up red	10–40 %
	Lights up yellow	40–70 %
	Lights up green	70 – 100 %
During charging	Flashes slowly red	0–40 %
	Flashes slowly yellow	40–70 %
	Flashes slowly green	70–99 %
	Lights up green	100 %

- 4) Disconnect the appliance from the power supply and unplug the USB charging cable **8** from the appliance.

The battery can be recharged at any time and at any battery level.

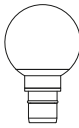
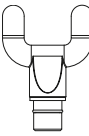
## 6. Using the appliance

You can use this appliance to give yourself or others a soothing and effective trigger point massage. Trigger point massages can relax or stimulate the muscles to relieve tension, pain and fatigue. The massage gun provides a powerful, intensive massage for the thighs, calves, shoulders, arms and the back region (excluding the spine).

Depending on the extent of the tension in the part of the body to be treated, a maximum treatment time of approx. 15 minutes is recommended.

## 6.1. Selecting/fitting attachments

1) Select the appropriate attachment for the desired treatment and body part as follows:

Attachment	Suitable for:
	Uniform and gentle massage to relax smaller muscle groups, e.g. upper arms, shoulders, waist and legs.
	Large areas all over the body, such as buttocks, thighs and back (excluding spine).
	Targeted and selective massage of deep muscle tissue, e.g. soles of the feet, palms of the hands, muscles of the shoulder blades.
	Uniform, 2-point stimulation of deep muscles in the lower back (except spine), neck, Achilles tendon.
	Gentle, soothing tapping massage of sensitive parts of the body, e.g. neck, shoulders, wrists and feet.

2) Insert the desired attachment into the socket **10** and press it as far as it will go (Fig. 1):

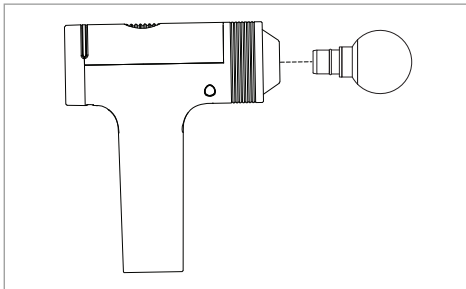
Fig. 1

3) To change the attachment, carefully pull it out of the socket **10** horizontally, turning it slightly if necessary.

## 6.2. Examples of use

### **i** Note

- ▶ Do not apply the attachments directly to a bone, only to muscle tissue.

Examples of application on arms (fig. 2), thighs (fig. 3), sole of the foot (fig. 4), Achilles tendon (fig. 5) and neck (fig. 6).

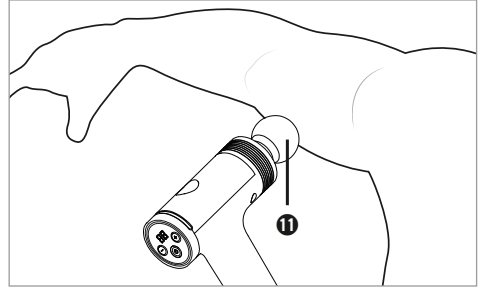
Fig. 2

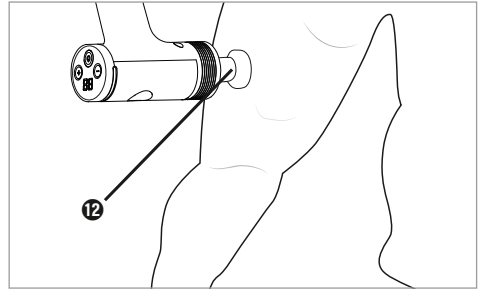
Fig. 3

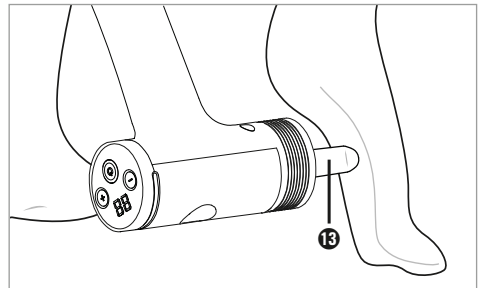
Fig. 4

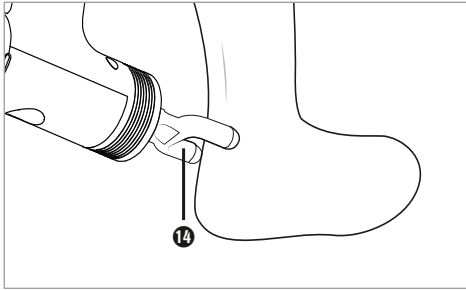
Fig. 5

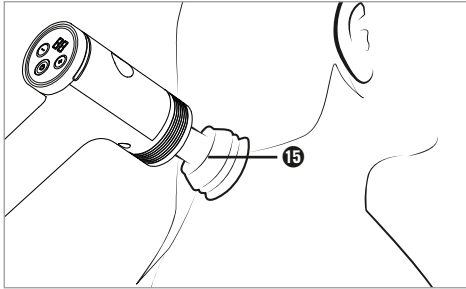
Fig. 6

### 6.3. Switching the appliance on/off and selecting intensity

- 1) Switch on the appliance by pressing and holding the ON/OFF button **⏻ 6** for about 2 seconds.
  - ✓ The battery level indicator **2** lights up. The display **4** and the buttons **3/5** light up. The appliance starts to operate at the lowest intensity level.
- 2) To increase the intensity level, briefly press the **+** button **5**. To decrease the intensity level, briefly press the **-** button **3**.
  - ✓ The intensity changes by one level each time you press it. The levels are shown on the display **4**.
- 3) Massage the desired body part by gently moving the attachment back and forth over the muscle area.

#### **i Note**

- ▶ Do not apply too much pressure, otherwise the appliance will automatically switch off after a few seconds to protect the motor from a possible overload.
- 4) To pause the massage, press the ON/OFF button **⏻ 6** briefly. The appliance switches to standby mode. You can press the ON/OFF button **⏻ 6** again to continue the massage; otherwise, the appliance will switch off after about 1 minute.
  - 5) If you no longer wish to use the appliance, switch it off by pressing and holding the ON/OFF button **⏻ 6** for about 2 seconds.

#### **i Note**

- ▶ The appliance is equipped with an automatic switch-off to prevent overheating. The appliance switches off automatically after 15 minutes of continuous operation. Allow the appliance to cool down for at least 15 minutes before using it again.

### 6.4. Intensity levels

The intensity of the massage varies depending on the level selected, as follows:

Display <b>4</b>	Intensity
01	1800 rpm
02	1935 rpm
03	2065 rpm
04	2200 rpm
05	2335 rpm
06	2470 rpm
07	2600 rpm
08	2735 rpm
09	2870 rpm
10	3000 rpm

## 7. Cleaning and care

### **⚠ DANGER! ELECTRIC SHOCK!**

- ▶ Do not clean the appliance while it is charging. Switch off the appliance before cleaning.
- ▶ Ensure that no moisture can penetrate the device during cleaning.

### **⚠ ATTENTION!**

- ▶ Do not use abrasive or aggressive cleaning materials. These can damage the surface of the appliance!
  - 1) Make sure that the appliance is switched off.
  - 2) Make sure that the appliance is not connected to a mains adapter. Unplug the USB charging cable **①** from the appliance if it is plugged in.
  - 3) Wipe off the appliance using a damp cloth. Do not use any solvents or cleansers which can damage plastic. If necessary, add a little mild detergent to the cloth and wipe off afterwards with fresh water. Make sure that no water or moisture can get into the appliance!
  - 4) To hygienically clean the attachments, wash them with water and a mild detergent and then rinse them with clear water.
  - 5) Allow everything to dry thoroughly before storing the appliance or putting it back into operation.

## 8. Storage

Store the cleaned appliance in the storage bag **①** in a dust-free and dry location out of direct sunlight. Charge the battery at regular intervals to ensure a long battery life.

### **⚠ ATTENTION!**

- ▶ Never store the device when it is discharged. Prolonged storage in a discharged state can cause permanent damage to the battery. If the device is stored for a longer period of time, the charge level of the battery must be checked regularly. The optimum charge level is between 50 % and 80 %.

## 9. Troubleshooting

Fault	Cause	Solution
The appliance is not working.	The battery is empty.	Charge the appliance.
	The appliance is charging and cannot be used.	Wait until the battery is fully charged.
	The appliance is defective.	Contact the Customer Service.
The appliance switches off briefly during operation.	The motor overload protection has been triggered.	Do not press too hard while massaging.

If the malfunction cannot be corrected using one of the suggestions above, or if you have found some other kind of malfunction, contact Customer

### **13.1. Service.**

## 10. Ordering replacement parts

You can order replacement parts for this product on the Internet at [www.kompernass.com](http://www.kompernass.com).

Scan the QR code with your smartphone/tablet. You can use this QR code to go directly to our website to view and order the available spare parts.

### **① Note**

- ▶ If you have problems with your online order, you can contact our service centre by phone or e-mail.
- ▶ Always quote the article number (IAN) 509807\_2507 in your order.
- ▶ Please note that online ordering of replacement parts is not possible for all countries.

## 11. Disposal

### 11.1. Disposal of appliance and packaging

The symbol of the crossed-out wheeled bin means that this appliance may not be disposed of in regular household waste at the end of its service life. The

appliance must be deposited at an established collection point, recycling centre or disposal company.

Please erase all personal data before returning an appliance.

Before returning an appliance, please remove any batteries that are not permanently enclosed within the old appliance as well as any light sources that can be removed without destroying the appliance. Then dispose of these items separately.

If the rechargeable batteries are permanently installed, you must indicate during disposal that the appliance contains a battery.

#### **The built-in rechargeable battery in this appliance cannot be removed for disposal.**

The packaging is made from environmentally friendly material which can be disposed of at your local recycling plant.

Dispose of the packaging in an environmentally friendly manner.

## 12. Technical specifications

Appliance	
Input voltage	5.0 V
Input current	2.0 A
Integrated battery (Li-Ion)	2500 mAh / 11.1 V  / 27.75 Wh
Protection class	III /

## 13. Kompersnass Handels GmbH warranty

Dear Customer,

This appliance has a 3-year warranty valid from the date of purchase. If this product has any faults, you, the buyer, have certain statutory rights. Your statutory rights are not restricted in any way by the warranty described below.

### **Warranty conditions**

The warranty period starts on the date of purchase. Please keep your receipt in a safe place. This will be required as proof of purchase.

If any material or manufacturing fault occurs within three years of the date of purchase of the product, we will either repair or replace the product for you or refund the purchase price (at our discretion).

This warranty service requires that you present the defective appliance and the proof of purchase (receipt) within the three-year warranty period, along with a brief written description of the fault and of when it occurred.

If the defect is covered by the warranty, your product will either be repaired or replaced by us. The repair or replacement of a product does not signify the beginning of a new warranty period.

### **Warranty period and statutory claims for defects**

The warranty period is not prolonged by repairs effected under the warranty. This also applies to replaced and repaired components. Any damage and defects present at the time of purchase must be reported immediately after unpacking. Repairs carried out after expiry of the warranty period shall be subject to a fee.

### **Scope of the warranty**

This appliance has been manufactured in accordance with strict quality guidelines and inspected meticulously prior to delivery.

The warranty covers material faults or production faults. The warranty does not extend to product parts subject to normal wear and tear or to fragile parts which could be considered as consumable parts such as switches or parts made of glass.

The warranty does not apply if the product has been damaged, improperly used or improperly maintained. The directions in the operating instructions for the product regarding proper use of the product are to be strictly followed. Uses and actions that are discouraged in the operating instructions or which are warned against must be avoided.

This product is intended solely for private use and not for commercial purposes. The warranty shall be deemed void in cases of misuse or improper handling, use of force and modifications/repairs which have not been carried out by one of our authorised Service centres.

### Warranty claim procedure

To ensure quick processing of your case, please observe the following instructions:

- Please have the till receipt and the item number (IAN) 509807\_2507 available as proof of purchase.
- You will find the item number on the type plate on the product, an engraving on the product, on the front page of the operating instructions (below left) or on the sticker on the rear or bottom of the product.
- If functional or other defects occur, please contact the service department listed either by **telephone** or by **e-mail**.
- You can return a defective product to us free of charge to the service address that will be provided to you. Ensure that you enclose the proof of purchase (till receipt) and information about what the defect is and when it occurred.

You can download these instructions along with many other manuals, product videos and installation software at [www.lidl-service.com](http://www.lidl-service.com).

This QR code will take you directly to the Lidl service page ([www.lidl-service.com](http://www.lidl-service.com)) where you can open your operating instructions by entering the item number (IAN) 509807\_2507.

## 13.1. Service

### GB Service Great Britain

Tel.: 0800 404 7657

E-Mail: [kompernass@lidl.co.uk](mailto:kompernass@lidl.co.uk)

### IE Service Ireland

Tel.: 1800 101010

E-Mail: [kompernass@lidl.ie](mailto:kompernass@lidl.ie)

### MT Service Malta

Tel.: 80062230

E-Mail: [kompernass@lidl.com.mt](mailto:kompernass@lidl.com.mt)

IAN 509807\_2507

## 13.2. Importer

Please note that the following address is not the service address. Please use the service address provided in the operating instructions.

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Lidl Great Britain Ltd