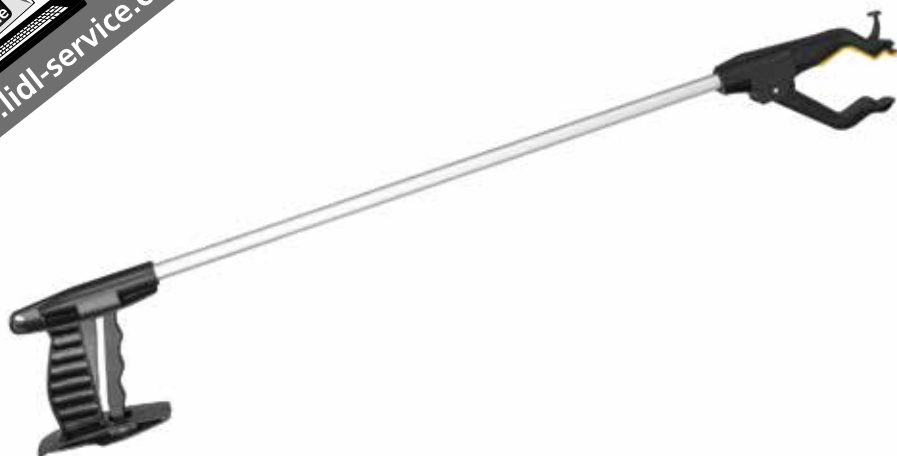
www.lidl-service.com

ALUMINIUM GRIPPER

(GB) (IE) (NI)

ALUMINIUM GRIPPER

Instructions for use

(FR) (BE)

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(DE) (AT) (CH)

ALUMINIUM-GREIFHILFE

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(DK)

GRIBETANG I ALUMINIUM

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(NL) (BE)

ALUMINIUM GRIJPHULP

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
IAN 282068

(GB) (IE) (NI) (DK) (FR)
(BE) (NL) (DE) (AT) (CH)

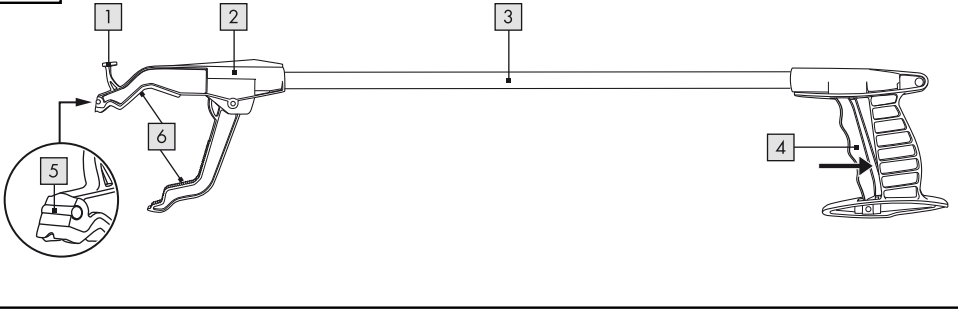A

GB IE NI

Included in Delivery	4
Technical Data	4
Symbols used	4
Intended use	4
Safety notes	4
Using the article	5
Care, Storage	5
Disposal Information	5
3-year warranty	5

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Congratulations!

With your purchase you have decided on a high-quality product. Get to know the product before you start to use it.

Carefully read the following instructions for use.

Use the product only as described and only for the given areas of application.

Keep these instructions safe. When passing the product on to a third party, always make sure that the documentation is included.

This product conforms to guideline 93/42/EEC on medical devices.

Please contact our service department if you have any questions regarding our product or if you have difficulties reading these instructions.

Included in Delivery

1 x Aluminium Gripper

1 x Instructions for use

Technical Data

Maximum permitted load:	1 kg
Length:	approx. 760mm
Weight:	approx. 0.199kg

Symbols used

Observe the instructions

Date of manufacture

Batch designation

Manufacturer

Max. user weight

Intended use

The article is designed as an aid for gripping, lifting or reaching smaller objects weighing up to 1kg for persons for whom illness or handicap has led to a limitation of the radius of action of the upper extremities. Some gripping action is still required.

The article is intended for private use and is not suitable for commercial purposes.

The expected lifespan of the article is up to 3 years with correct use and observance of the safety and care notes. Use of the article can continue beyond this time if it is still in order and in a safe condition.

Safety notes

WARNING!

Danger of strangulation for children!

Never allow children to play unsupervised with this article or its packaging.

- Use the article only for its stated use.
- The article should only be used by persons aware of these safety instructions.
- Check the article before every use for damage or signs of wear and tear.
- This article should only be used whilst in good order and condition! Do not use the article if you can see that it has been damaged or if you think it may be damaged!
- Use this article only when surface temperatures are suitable for its use. Hot water, direct sunlight, sauna conditions, extreme cold, etc. can cause the surface of the article to assume temperatures which could endanger health.
- Do not attempt to alter the product in any way and do not try to repair it yourself! Order spare parts only from the manufacturer. Repairs that are improperly carried out can cause unforeseen damage!
- Do not point the grip aid at people or animals and do not attempt to use it as a truncheon or baton!
- Keep the grip arm well away from the eyes!
- Do not attempt to pick up hot or fragile objects!

Using the article (fig. A)

Grasping Forceps (2)

Note: The grasping forceps (2) is able to swivel 360° around the aluminium shaft (3).

1. Position the grasping arm (6) around the object that you wish to pick up.
2. Pull the hand grip (4) in the direction of the arrow (fig. A) and hold it in this position. The grasping arms will then be pressed together, grabbing hold of the object.
3. Move the object to the required position and release the hand grip. The grasping arms will then move apart and the object will be released.

Magnet (5)

For picking up keys or other metal objects.

Hooks (1)

Use, for example, as an aid for opening drawers.

Care, Storage

ATTENTION!

Do not use any corrosive or abrasive cleansers or sponges. They may damage the surface.

1. Wipe the product with a damp cloth and some mild cleanser if necessary.
2. Dry the article thoroughly before you use it again.

Disposal Information

Dispose of the article and the packaging materials in accordance with current local regulations. Packaging materials such as foil bags are not suitable to be given to children. Keep the packaging materials out of the reach of children.

3-years warranty

The product was produced with great care and under constant supervision. You receive a three-year warranty for this product from the date of purchase. Please retain your receipt.

The warranty applies only to material and workmanship and does not apply to misuse or improper handling. Your statutory rights, especially the warranty rights, are not affected by this warranty.

With regard to complaints, please contact the following service hotline or contact us by e-mail. Our service employees will advise as to the subsequent procedure as quickly as possible. We will be personally available to discuss the situation with you.

Any repairs under the warranty, statutory guarantees or through goodwill do not extend the warranty period. This also applies to replaced and repaired parts. Repairs after the warranty are subject to a charge.

IAN: 282068

GB Service Great Britain

Tel.: 0871 5000 720
(£ 0.10/Min.)

E-Mail: deltasport@lidl.co.uk

IE Service Ireland

NI Tel.: 1890 930 034
(0,08 EUR/Min., (peak))
(0,06 EUR/Min., (off peak))

E-Mail: deltasport@lidl.ie

You can also find spare parts for your product at:
www.delta-sport.com, category Service - Lidl Spare Parts Service

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