WÄRME-/KÄLTE-KOMPRESSE-SET RÜCKEN HOT/COLD COMPRESS SET BACK COMPRESSE CHAUDE/FROIDE DOS

(DE) (AT)

WÄRME-/KÄLTE-KOMPRESSE-SET RÜCKEN

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(GB) (IE) (NI)

HOT/COLD COMPRESS SET BACK

Instructions for use (3-5)

(FR) (BE)

COMPRESSE CHAUDE/FROIDE DOS

Mode d'emploi (5-6)

(NL) (BE)

SET WARME/KOUDE COMPRESSEN RUG

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(PL)

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Hersteller

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Oberer Temperaturgrenzwert 50°C

Congratulations on the purchase of your new product. You have chosen a high-quality article. The instructions for use are part of this product. They contain important information on safety and use. Please familiarise yourself with the instructions and safety information before using this product. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the product and be sure to include them when handing on the product to another person.

Medical device

Only intended for private use!

Note:

The medical device is intended for adults who are able to read, understand and independently follow these instructions for use. A reason for use, as named under Indications, must be present at the user.

The item is not intended for groups of persons who are excluded from its application by the contraindications listed below.

Scope of delivery

1 x heat/cold multiple-use compress

1 x securing strap

1 x instructions for use

Intended purpose for cooling:

Cooling compress for external use. Can help alleviate pain.

Indications

For supporting use in:

- Contusions
- Sprains
- Bruises
- Inflammation
- Swelling

Cold treatment:

The compress must be stored in the refrigerator for approx. 1 hour at 5° C before use. Then insert the compress into the fastening strap and place it on the sore spot.
Application time: approx. 10 minutes.

Intended purpose for heat:

Warming compress for external use. Can help alleviate pain.

Indications

For supporting use in:

- Sore muscles
- Tense muscles

Heat treatment:

- Use in a water bath:
 - Bring water to the boil in a pot. Take the pot off the hob and place the compress in the water for about 1 – 3 minutes. Check the temperature after 3 minutes. Continue heating if necessary.
- Use in the microwave:
 - The compress must be placed in the middle of the microwave and heated at 600 W for 15 – 25 seconds. Please wait 20 seconds before opening the microwave, as delayed heating is possible
 - Take out the compress with a protective glove and knead it carefully so that the heat is evenly distributed.
 - Carefully check the temperature with the palm of your hand. If it is too warm, let it cool down. If the compress is not warm enough, repeat the warming process for another 10 seconds. The compress must not be warmed above 50°C, otherwise there is a risk of burns and damage to the product!
- If gel leaks, the compress is damaged and must be disposed of.
- Fasten the compress in the fastening belt on the area to be treated.
- Application time: approx. 20 – 40 Minuten

Contraindications

 **The multi-use compress must not be used, if you:**

- suffer from circulatory disorders
- have a limited sense of temperature
- take painkillers
- have open wounds or damaged skin areas

No heat application in the case of inflammation or increased temperature.

No cold application for acute kidney (pelvic) diseases and cystitis.

Warnings

-  Keep out of the reach of children and use only under adult supervision.
-  Do not exceed the specified wearing times.
-  Please check before each use that the compress is not damaged. Otherwise, please dispose of with your household waste.
-  Do not overtighten the strap, as this may impair your blood circulation.

-  Before using the compress, clean the skin thoroughly. The skin must be free of gels, greases, ointments, oils or similar substances.
-  Never place the cooled or heated compress directly on the skin. Use the fastening strap or wrap the compress in a cloth before use.
-  For external use only.
-  The compress must not be used while sleeping.
-  The ingredients can be harmful if swallowed or if they come into direct contact with the skin or eyes: In all cases, rinse immediately with water and seek medical advice immediately.
-  Keep away from open flames

If problems or complaints should occur, consult a doctor immediately!

Cold application

Under no circumstances must this be used without its fastening strap, as otherwise there is a risk of frostbite!

Heat application

Only mentally healthy people with full sensory capabilities are allowed to take over the warming process of the compress.

The compress must not be heated in the oven. The compress must not be used without the fastening strap.

Care

Securing strap:

The securing strap should be washed before first use. To do this, close the Velcro fastener. The article should be washed again after three uses.

Multi-use compress:

The compress should be washed under warm water with a mild soap (do not use other detergents or disinfectants) and dried with a soft cloth.

Storage – securing strap

 Keep dry / protect from moisture.

Instructions for disposal

Please dispose of the product in an environmentally-friendly way through an approved disposal centre or your local waste facility. Observe currently applicable regulations.

Obligation to notify

All serious incidents relating to the product must be reported immediately to the manufacturer and the competent authority.

3-year warranty

The product has been created with great care and under constant monitoring. You will receive a 3-year warranty for this product from the date of purchase. Please keep the receipt. The warranty is valid only for material and manufacturing faults and lapses in the event of misuse or improper handling. Your statutory rights, particularly the warranty rights, are not restricted by this warranty. If you have any complaints, please contact the service hotline below or contact us by email. Our service staff will discuss how to proceed further with you as soon as possible. In any case, we will personally advise you. The warranty period is not extended

by any repairs under the terms of the warranty, statutory warranty or goodwill. This also applies to replaced and repaired parts. After expiry of the warranty, incurred repairs are chargeable.

Service Germany
Tel.: +49 (0)40 82 22 44-80
E-Mail: info@horizonte.de

-  Consult instructions for use
-  Attention: Always adhere to the stated warning information
-  Keep dry / protect from moisture
-  Catalogue number
-  Batch designation
-  Use by
-  Date of manufacture (year): 2021
-  Manufacturer
-  CE Marking of conformity
-  Medical device

Cold treatment: multi-use compress

 Caution! Very cold surface!

Heat treatment: multi-use compress

 Caution! Hot surface!
 Maximum temperature limit 50°C

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Clean the product regularly/Wash separately or with similar colours/

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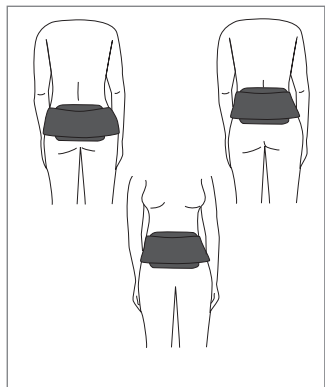
(GB) (IE) (NI)

Strapping on – securing strap:

CAUTION! The fastening strap should not be strapped on too tightly, since this could impair your blood circulation. The compress must be placed, crease-free on the body part, which is to be treated. Can also be used on your pelvis or abdomen.

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(GB) (IE) (NI)

With this QR code you can access **www.lidl-service.com** and open your instructions for use with the product number (IAN) 373256.

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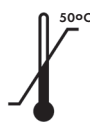
Tested Medical Device
Regular Production Surveillance
Safety
www.tuv.com
ID: 1111211771

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