## **ELLENBOGENBANDAGE EPI-ACTIVE ELBOW BRACE EPI-ACTIVE COUDIÈRE EPI-ACTIVE**

**DE AT CH**

### **ELLENBOGENBANDAGE EPI-ACTIVE**

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**GB IE**

### **ELBOW BRACE EPI-ACTIVE**

Instructions for use (3-4)

**FR BE**

### **COUDIÈRE EPI-ACTIVE**

Mode d'emploi (4-5)

**NL BE**

### **ELLEBOOGBANDAGE EPI-ACTIVE**

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**PL**

### **OPASKA NA ŁOKIEĆ EPI-ACTIVE**

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**CZ**

### **LOKETNÍ BANDÁŽ EPI-ACTIVE**

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**SK**

### **LAKŤOVÁ BANDÁŽ EPI-ACTIVE**

Návod na použitie (9-10)

**IAN 343818\_1910**

**DE AT CH GB IE FR**

**BE NL PL CZ SK**

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Congratulations on the purchase of your new product. You have chosen a high-quality article. The instructions for use are part of this product. They contain important information on safety and use. Please familiarise yourself with the instructions and safety information before using this product. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the brace and be sure to include them when handing on the product to another person.

### Medical Device

**Only intended for private use!**

### Scope of delivery

- 1 x elbow brace
- 1 x elastic forearm brace
- 1 x instructions for use

### Technical Data

Size	Forearm circumference
S	18 - 21 cm
M	21 - 24 cm
L	24 - 27 cm
XL	27 - 30 cm

To be worn on the left and right elbow

## Intended use

The bandage stabilises, supports and relieves strain on the elbow joint. The pads on the side of it and the elastic forearm brace help to stabilise the elbow joint and they promote circulation and improve muscle function. The napped pads produce the effect of a gentle massage.

### To be used as support for:

- Inflammatory joint conditions (rheumatoid arthritis, rheumatism)
- Degenerative joint disease (osteoarthritis)
- Epicondylitis (tennis elbow or golfer's elbow)
- Bruises
- Feeling of instability

### Usage restrictions

**Do not use – or use only after consultation with your doctor – for:**

- External damage
- Bone fractures
- Circulatory disorders
- Higher degree of instability in the elbow joint
- Neurological deficits such as paralysis, radiating nerve pain or similar.

## Safety instructions

### Risk of injury

-  If problems occur while wearing the bandage, remove it immediately and consult a doctor.
-  Check the bandage before each use for tears and damage.
-  The product should only be used in perfect condition.
-  Make sure the bandage is comfortable but not too tight.
-  Never put the bandage on so tightly that the blood circulation is restricted.
-  Do not wear the bandage longer than 2 to 3 hours at a stretch and a maximum of 6 to 8 hours a day.
-  Do not wear the bandage while sleeping or during longer rest phases such as long periods of sitting.
-  Wash the areas of skin covered by the bandage thoroughly and regularly.
-  To avoid allergic skin reaction., the skin must be free of grease, gels, creams, oils or similar materials.

### Possible side effects

-  If you experience symptoms (such as, e.g. tingling, pain, swelling) while wearing the bandage, remove the bandage immediately!
-  There is a risk of skin irritation and skin irritations! In individual cases, skin irritation and reactions and allergic reactions may occur. Make sure that the stated maximum wearing periods are not exceeded and the areas of skin covered by the bandage are thoroughly and regularly washed. Should any problems or complaints occur, consult a doctor immediately!

## Putting the bandage on

- Open the elastic brace and pull the article over your hand and forearm as far as your elbow (**Fig. 1**). If the bandage is fitted correctly the notches in the pads will lie over the bony protuberances at the side of the elbow (epicondyles).
- The bandage should fit nice and securely, but not too tightly.
- The elastic forearm brace that is supplied may be specifically fitted around the elbow bandage in order to provide additional stabilisation (**Fig. 2**).
- The forearm brace is placed over the front part of the side pads (**Fig. 3**).

**Note:** The explanations/pictures apply as examples of use for both the right and left elbow.

## Cleaning and care

- Thoroughly wash the brace before first use. Remove the elastic brace before washing the bandage.
- Clean the product regularly.
- After washing, thoroughly rinse your brace and reinsert the stabiliser bar after drying.
- It's advisable to wash the brace again every second or third use.
- Wash separately.

## Instructions for disposal

Please dispose of the product in an environmentally-friendly way through an approved disposal centre or your local waste facility. Observe currently applicable regulations.

## 3-year warranty

The product has been created with great care and under constant monitoring. You will receive a 3-year warranty for this product from the date of purchase. Please keep the receipt. The warranty is valid only for material and manufacturing faults and lapses in the event of misuse or improper handling. Your statutory rights, particularly the warranty rights, are not restricted by this warranty. If you have any complaints, please contact the service hotline below or contact us by email. Our service staff will discuss how to proceed further with you as soon as possible. In any case, we will personally advise you. The warranty period is not extended by any repairs under the terms of the warranty, statutory warranty or goodwill. This also applies to replaced and repaired parts. After expiry of the warranty, incurred repairs are chargeable.

Service Germany

Tel.: 0049 (0)40 8222448-0

E-Mail: info@horizonte.de

-  Consult instructions for use
-  Attention: Always adhere to the stated warning information
-  Order number (article number)
-  Batch designation
-  Use by
-  Date of manufacture (year): 2020
-  Manufacturer
-  CE Marking of conformity

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