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Artikelnummer

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Hersteller

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Congratulations on the purchase of your new product. You have chosen a high-quality article. The instructions for use are part of this product. They contain important information on safety and use. Please familiarize yourself with the instructions and safety information before using this product. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the product and be sure to include them when handing on the product to another person.

## Medical device

Only intended for private use!

### Note

The medical device is intended for adults who are able to read, understand and independently follow these instructions for use. A reason for use, as named under Indications, must be present at the user.

The item is not intended for groups of persons who are excluded from its application by the sizes and contraindications listed below.

## Scope of delivery

- 1 x back belt
- 1 x heat pad
- 1 x instructions for use

## Technical Data

Size	Waist circumference
M - L	80 - 98 cm
L - XXL	96 - 116 cm

## Intended Use

This bandage stabilizes, relieves and serves for external heat application in the lower back area.

## Indications

### To be used as support for:

- Chronic pain in the lower spine (lower back pain)
- First aid for lumbago
- Wear of the small vertebral joints (lumbar facet syndrome)
- Muscle tension
- Muscular imbalance
- Hypermobility of the lumbar spine

## Contraindications

 **Do not use – or use only after consultation with your doctor – in case of:**

- Circulatory disorders
- Limited Temperature Sensation
- Taking Pain-relieving Medication
- External injuries
- Bone fractures
- feverish diseases, Inflammations
- Disease patterns in which heat application can promote deterioration of the general state of health (e.g. diabetes mellitus, rheumatism)
- Heightened instability of the lumbar spine (e.g. spondylolisthesis)
- Neurological deficits such as paralysis of the lower extremities, radiating nerve pain etc.

## Warnings

### Heat Pad:

 Keep away from naked flames.

 Use it only along with the accompanying back strap.

**Caution: Never directly wear the heat pad on the skin.**

Otherwise, direct contact with the skin can cause burns!

### Risk of injury

 If problems occur while wearing the bandage, remove it immediately and consult a doctor.

 Check the back strap, as well as the heat pad for cracks and damage before each use.

 The product should only be used in perfect condition.

 Make sure the bandage is comfortable but not too tight.

 Never put the bandage on so tightly that the blood circulation is restricted.

 Do not wear the bandage longer than 2 to 3 hours at a stretch and a maximum of 6 to 8 hours a day.

 Do not wear the bandage while sleeping or during longer rest phases such as long periods of sitting.

 Wash the areas of skin covered by the bandage thoroughly and regularly.

 To avoid allergic skin reaction, the skin must be free of grease, gels, creams, oils or similar materials.

 The article may only be used by other people when it has been cleaned.

 The product is only for external use, and it must be kept out of the reach of children.

## Application

 The heat pad must not be heated in the oven or microwave.

- Before applying the bandage, activate the heat function of the heat pad immediately. For this purpose, lightly press (do not bend or break) the metal plate of the pad (Fig. 1). Therefore, the gel of the heat pad turns into a tarnished, more solid state and becomes warm.
- Put the heat pad in the pocket of the back strap. Thereby, the metal plate points upwards (Fig. 2).
- The side Velcro fasteners of the elastic compression straps must be loosened, before putting on the back strap. Place the back strap around the desired back area and then close the Velcro fastener. Close the side Velcro fasteners of the compression straps one after the other. The bandage should fit comfortably and slightly to be stretched in a moderate way (Fig. 3).

## Reactivating the heat pad

The heat pad should be completely cooled down. In order to reactivate it, place the heat pad in a pot of boiling water for approximately 10 (ten) minutes. Caution: In order to avoid damage to the heat pad or possible signs of wear on the pot, we recommend wrapping the heat pad in a cotton cloth (e.g. tea towel) before placing it in the pot (Fig. 4).

The filling material changes its consistency throughout the boiling time and becomes clear and liquid again.

Remove the heat pad from the pot using kitchen tongs for avoiding possible burns. Ensure that there are no more tarnished, crystal-like parts. If not, put the heat pad back into the boiling water. Afterwards, dry the heat pad completely.

## Cleaning and care

### Back Belt

- Before the first use, you should wash the back strap properly without the heat pad.
- Follow the care instructions on the product.
- After using it two or three times, it should be washed again.
- Close the Velcro pieces during the washing process.
- Wash separately.

### Heat Pad

- The heat pad can be wiped clean with a damp cloth when desired.
  - Do not use cleaning agents or disinfectants.
- Store the heat pad inserted in the pocket of the back strap.

## Instructions for disposal

Please dispose of the product in an environmentally-friendly way through an approved disposal centre or your local waste facility. Observe currently applicable regulations.

## Obligation to notify

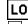
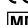
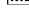
All serious incidents relating to the product must be reported immediately to the manufacturer and the competent authority.

## 3-year warranty

The product has been created with great care and under constant monitoring. You will receive a 3-year warranty for this product from the date of purchase. Please keep the receipt.

The warranty is valid only for material and manufacturing faults and lapses in the event of misuse or improper handling. Your statutory rights, particularly the warranty rights, are not restricted by this warranty. If you have any complaints, please contact the service hotline below or contact us by email. Our service staff will discuss how to proceed further with you as soon as possible. In any case, we will personally advise you. The warranty period is not extended by any repairs under the terms of the warranty, statutory warranty or goodwill. This also applies to replaced and repaired parts. After expiry of the warranty, incurred repairs are chargeable.

Service Germany  
Tel.: +49 (0)40 82 22 44-80  
E-Mail: info@horizonte.de

-  Consult instructions for use
-  Attention: Always adhere to the stated warning information
-  Catalogue number
-  Batch designation
-  Use by
-  Date of manufacture (year): 2023
-  Manufacturer
-  CE Marking of conformity
-  Medical device

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