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Congratulations on the purchase of your new product. You have chosen a high-quality article. Please familiarise yourself with the instructions and safety information before using this product. The instructions for use are part of this product. They contain important information on safety and use. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the product and be sure to include them when handing on the product to another person.

## Medical device

### Only intended for private use!

### Note:

The medical device is intended for adults who are able to read, understand and independently follow these instructions for use. A reason for use, as named under Indications, must be present at the user. The item is not intended for groups of persons who are excluded from its application by the sizes and contraindications listed below.

## Scope of delivery

- 1 x ankle brace
- 2 x cooling pads
- 1 x instructions for use

## Technical Data

Size	Ankle circumference
S	21 – 23 cm
M	23 – 25 cm
L	25 – 27 cm
XL	27 – 29 cm

Wear on the left and right

## Intended purpose

The brace supports, relieves and cools the ankle joint.

## Indications

To be used as support for:

- ankle injuries (such as strains, bruises and sprains)
- inflammations and swellings
- irritations

## Contraindications

**⚠ Do not use – or use only after consultation with your doctor – for:**

- external injuries
- complex chronic ligament instabilities
- circulatory disorders
- lymphatic drainage disorders
- eczema or other skin diseases
- reduced temperature sensation
- ingestion of painkillers

## Safety instructions

### Risk of injury

- ⚠ If problems occur while wearing the bandage, remove it immediately and consult a doctor.
- ⚠ Check the bandage before each use for tears and damage.
- ⚠ The product should only be used in perfect condition.
- ⚠ Make sure the bandage is comfortable but not too tight.
- ⚠ Never put the bandage on so tightly that the blood circulation is restricted.
- ⚠ Do not wear the bandage longer than 2 to 3 hours at a stretch and a maximum of 6 to 8 hours a day.
- ⚠ Do not wear the bandage while sleeping or during longer rest phases such as long periods of sitting.
- ⚠ Wash the areas of skin covered by the bandage thoroughly and regularly.
- ⚠ To avoid allergic skin reaction, the skin must be free of grease, gels, creams, oils or similar materials.
- ⚠ The article may only be used by other people when it has been cleaned.

### Cooling pads:

- ⚠ The cooling pads are for external use only and must be kept out of the reach of children.
- ⚠ Please check that the cooling pads are not damaged before each use. Otherwise, please dispose of them in the household waste.
- ⚠ Keep away from open flames.
- ⚠❄ When cooling the pads in the freezer, a thin garment must be worn under the bandage. Avoid direct skin contact with the cooling pads, otherwise there is a risk of cold burns!

## Possible side effects

- ⚠ If you experience symptoms (such as, e.g. tingling, pain, swelling) while wearing the bandage, remove the bandage immediately!
- ⚠ There is a risk of skin inflammation and skin irritation! In individual cases, skin irritation and reactions as well as allergic reactions may occur. Make sure that the stated maximum wearing periods are not exceeded and the areas of skin covered by the bandage are thoroughly and regularly washed. Should any problems or complaints occur, consult a doctor immediately!

## Cooling the cooling pads

- Remove the cooling pads from the pockets on the inside of the bandage. To do this, it is recommended to turn the bandage inside out.
  - Depending on the desired degree of coldness, the cooling pads can be cooled in the refrigerator or freezer.
- Alternatively, the bandage can be cooled with inserted cooling pads in the refrigerator or freezer compartment.
- To do this, place the bandage in a closed freezer bag.
  - Make sure that the bandage is completely dry.
  - Stop cooling the items directly before use.
- ⚠❄ After the bandage has been cooled in the freezer compartment, a garment must be worn under it to reduce the risk of cold burns!

## Inserting the cooling pads

- Turn the article inside out and insert the cooling pads into the pockets provided (Fig. 1). Smooth out the cooling pads after they are fully inserted.
- Make sure that the cooling pads are completely in the pocket (Fig. 2).
- Then turn the bandage over again so that the cooling pad pockets are on the inside of the bandage.

## Putting the bandage on

- Pull the article from the front over the foot to the ankle (Fig. 3).
- To check the correct fit, fold over the upper part of the bandage. The cooling pads should be positioned over the ankle protrusions (Fig. 4).
- Make sure that the article fits without wrinkles

## Using the cooling bandage

- It is recommended to repeat the cooling several times.
- Maximum 20 min per application

## Cleaning and care

- Before using it for the first time, the bandage should be thoroughly washed.
- To do this, remove the cooling pads before washing.
- After using it two or three times, it should be washed again.
- Wash separately.

## Instructions for disposal

Please dispose of the product in an environmentally-friendly way through an approved disposal centre or your local waste facility. Observe currently applicable regulations.

## Obligation to notify

All serious incidents relating to the product must be reported immediately to the manufacturer and the competent authority.

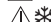
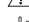
## 3-year warranty

The product has been created with great care and under constant monitoring. You will receive a 3-year warranty for this product from the date of purchase. Please keep the receipt. The warranty is valid only for material and manufacturing faults and lapses in the event of misuse or improper handling. Your statutory rights, particularly the warranty rights, are not restricted by this warranty. If you have any complaints, please contact the service hotline below or contact us by email. Our service staff will discuss how to proceed further with you as soon as possible. In any case, we will personally advise you. The warranty period is not extended by any repairs under the terms of the warranty, statutory warranty or goodwill. This also applies to replaced and repaired parts. After expiry of the warranty, incurred repairs are chargeable.

Service Germany

Tel.: +49 (0)40 82 22 44-80

E-Mail: support@horizonte.de

-  Consult instructions for use
-  Attention: Always adhere to the stated warning information
-  Attention! Cold surface
-  Temperature limit / -18 °C to 37 °C  
(recommended temperature during application)
-  Catalogue number
-  Batch designation
-  Use by
-  Date of manufacture (year): 2023
-  Manufacturer
-  CE Marking of conformity
-  Medical device

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