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List of pictograms used	Page 39
Introduction	Page 42
Intended use.....	Page 42
Description of parts.....	Page 43
Scope of delivery.....	Page 43
Technical data.....	Page 43
Warning and safety notes	Page 45
General warnings.....	Page 45
Information on electromagnetic compatibility.....	Page 46
Battery safety information.....	Page 48
Prior to first use	Page 49
Inserting / replacing batteries.....	Page 49
Basic settings	Page 49
Activating / deactivating sound	Page 51
What you should know before taking a measurement	Page 51
Taking a measurement	Page 53
Measuring body temperature from the forehead.....	Page 54
Measuring the temperature of objects.....	Page 55
Displaying saved measured values.....	Page 55
Restoring the device to its factory settings	Page 56
Cleaning and care	Page 56

Storage Page 57

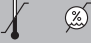
Troubleshooting Page 58

Disposal Page 59

Warranty / service Page 60

List of pictograms used

	WARNING Warning of risks of injury or health hazards
	ATTENTION This shows that caution should be exercised if devices or controllers are being operated near this symbol or that the current situation requires attentiveness or intervention from the operator to avoid unintended consequences.
	Safety notices
	Instructions for use
	Observe the instructions Read the instructions before starting to work and/or operating devices or machines.
Storage & transport 	Permissible storage and transport temperature and humidity
Operating 	Permissible operating temperature and humidity
	Air pressure limit
	Manufacturer
	Date of manufacture
	Authorised Representative in the European Community

List of pictograms used

	Applied part, type BF
	Serial number
	Unique Device Identifier (UDI) Unique identifier for product identification
	CE mark This product fulfils the requirements of the applicable European and national directives.
	Direct current
	Do not dispose of batteries containing hazardous substances with household waste.
IP22	Device protected against foreign matter ≥ 12.5 mm and against water falling diagonally.
	The (electric) appliance may not be disposed of with the usual domestic waste.
	Labelling to identify the packaging material. A = material abbreviation, B = material number: 1-7 = plastics, 20-22 = paper and cardboard
	Remove the product from all packaging, then dispose of the packaging in accordance with local regulations.
	Remove all packaging, then dispose of it in accordance with local regulations.

List of pictograms used

	Measuring body temperature from the forehead
	Measuring the temperature of objects
	Memory: for 30 measurements
	Includes batteries
	Importer
	Medical product
	Item number
	Model number of product
	Batch reference
	EXPLOSION HAZARD! Never recharge disposable batteries. Never short-circuit and / or open batteries.
	WEAR SAFETY GLOVES! Leaking or damaged batteries can cause burns upon contact with the skin. Wear suitable protective gloves.

Infrared Thermometer

● Introduction

Read these instructions for use carefully. Observe the warnings and safety instructions. Keep these instructions for use for future reference. Make these instructions for use accessible to other users. Also include these instructions for use when passing the device on to others.

● Intended use

This infrared thermometer was specially developed to be safely used on the forehead. This device is used to measure the body temperature of people by recording the intensity of infrared light that is emitted from their forehead. It converts the heat that was measured into a temperature value that can be read on the display. When used correctly, it quickly and accurately calculates your temperature.

Purpose: The infrared thermometer is used for intermittent measurement of human body temperature from the skin on the forehead. The thermometer is only intended for private use at home and is not suitable for commercial purposes.

Target group: The thermometer is provided for medical assistance in a home environment, not in professional facilities for the healthcare sector. The fever thermometer can be used on new-born babies heavier than 2.5 kg as well as on infants, toddlers and children under supervision. It can be used by everyone from the age of 12 upwards.

Clinical benefit: The thermometer can be used to measure body temperature from the forehead at home. Temperature monitoring is in °C (or °F) and is accurate to one decimal point.

Indicator: The infrared thermometer is used to measure body temperature via the forehead.

Contraindication: The measurement must not be carried out in the healing phase after surgical procedures.

● Description of parts

- 1 Memory button
- 2 Battery cover
- 3 Display
- 4 Measurement button
- 5 Mode button (forehead or object measurement)
- 6 Sound button
- 7 Measurement sensor

Display indicators

- 8 Date/time display
- 9 Hour mode (AM/PM)
- 10 Battery level 
- 11 Memory mode **M**
- 12 Celsius/Fahrenheit unit display °E
- 13 Speaker symbol 
- 14 Temperature value
- 15 Forehead mode 
- 16 Object mode 

● Scope of delivery


- | | |
|----------------------------------|-------------------------------|
| 1 infrared thermometer | 1 set of instructions for use |
| 2 batteries 1.5 V type AAA, LR03 | 1 quick start guide |

● Technical data

The accuracy of this thermometer has been thoroughly tested and it was designed to have a long usable life.

If the device is used in medicine then the relevant national regulations apply if technical inspections are to be carried out with suitable equipment.

Type:	DET-3021
Model:	SFT 88
Operating mode:	calibration mode
Measurement method:	infrared measurement
Measurement range:	forehead thermometer mode: 34.0 °C–43.0 °C (93.2 °F–109.4 °F) Object temperature mode: 0.0 °C–100.0 °C (32.0 °F–212.0 °F)
Laboratory measurement accuracy:	forehead thermometer mode: ± 0.2 °C (± 0.4 °F) from 35.5 °C– 42.0 °C (95.9 °F–107.6 °F), outside this measurement range, it is ± 0.3 °C (± 0.5 °F), Object temperature mode: ± 4 % or ± 2.0 °C (± 4.0 °F)
Time interval between two measurements:	approx. 1 second
Clinical accuracy:	0–1 year: Clinical systematic error: -0.19 °C (-0.34 °F); Clinical repeatability: 0.06 °C (0.11 °F); Match threshold value: 0.68 °C (1.22 °F) 1–5 years: Clinical systematic error: -0.25 °C (-0.45 °F); Clinical repeatability: 0.07 °C (0.13 °F); Match threshold value: 0.73 °C (1.31 °F)

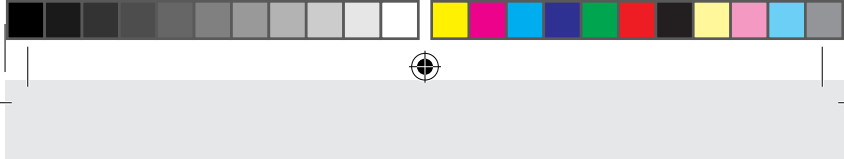Over 5 years:

Clinical systematic error: $-0.25\text{ }^{\circ}\text{C}$ ($-0.45\text{ }^{\circ}\text{F}$);
 Clinical repeatability: $0.17\text{ }^{\circ}\text{C}$ ($0.31\text{ }^{\circ}\text{F}$);
 Match threshold value: $0.60\text{ }^{\circ}\text{C}$ ($1.08\text{ }^{\circ}\text{F}$)
 $^{\circ}\text{Celsius (}^{\circ}\text{C)}$ or $^{\circ}\text{Fahrenheit (}^{\circ}\text{F)}$
 Operating conditions: $5.0\text{ }^{\circ}\text{C}$ to $40.0\text{ }^{\circ}\text{C}$ ($41.0\text{ }^{\circ}\text{F}$ to $104.0\text{ }^{\circ}\text{F}$)
 at a relative humidity of 15% to 95%
 (non-condensing) 700 - 1060 hpa ambient
 pressure

Measurement units:
 Operating conditions:

Storage
 conditions:

$-25.0\text{ }^{\circ}\text{C}$ to $55.0\text{ }^{\circ}\text{C}$ ($-13.0\text{ }^{\circ}\text{F}$ to $131\text{ }^{\circ}\text{F}$)
 at a relative humidity of 15% to 95%
 (non-condensing)

Dimensions:

approx. $145.1 \times 90.9 \times 42\text{ mm}$

Weight:

approx. 131 g (with batteries)

Battery:

2 x 1.5V AAA (LR03)

Battery life span:

approx. 6000 measurements

Memory:

for 30 measurements

Expected service life:

3 years

IP-protection class:

IP 22: Protected against solid foreign objects
 with a diameter over 12.5 mm and water
 falling diagonally at an angle of 15° .

● Warning and safety notes

General warnings

- There are no restrictions on gender or age for users of the infrared thermometer.
- Do not touch the measurement sensor with your hands.
- Use of this thermometer is not a substitute for consulting your doctor.

- 
- 
- Do not allow children to measure their temperature unsupervised as some parts are small enough to be swallowed.
 - Children may not use the device. Medical devices are not toys.
 - Never immerse this device in water or other liquids.
 - Do not make any modifications to this appliance without permission from the manufacturer.
 - Do not subject the thermometer to extreme temperatures (below $-25\text{ }^{\circ}\text{C}/-13\text{ }^{\circ}\text{F}$ or above $55\text{ }^{\circ}\text{C}/131\text{ }^{\circ}\text{F}$) or excessive humidity ($>95\%$).
 - Do not expose the thermometer to direct sunlight or to cotton as this impairs accuracy.
 - Electrical medical appliances must not be cleaned during operation.
 - Every serious incident that is linked to the device should be reported to the manufacturer as well as the relevant authorities in the member state in which the user and/or patient is domiciled.
 - The measurement sensor on the electrical medical device must not be serviced or repaired while the device is being used on a patient.
 - The thermometer is only designed for use in the measuring point on the human body stated in the instructions for use.
 - If you have further questions on using the device then please contact your specialist dealer or the Customer Service Department.

Information on electromagnetic compatibility

- This device must not be used near or on top of other electronic devices such as mobile phones, transceivers or radio-controlled devices. If you have to do this, then observe the device to check that it is working properly.
- Using components and power cables other than those specified may result in increased emissions or reduce the immunity of the device or system. Exceptions to these are cables that are sold by

the manufacturer of the device or system as replacement parts for internal components.

- Do not operate this device alongside or on top of other devices to avoid malfunctions.
- Using components, converters or cables that are not specified or provided by the manufacturer of this device may result in increased electromagnetic emissions or reduce the electromagnetic immunity of this device, which may lead to malfunctions.
- Portable HF communications equipment (including peripheral equipment such as antenna cables and external antennas) should not be used within 30 cm (12 inches) from any part of the device, including cables specified by the manufacturer. Otherwise the performance of this device may be impaired.
- Portable and mobile HF communications equipment may impair the device. Specific precautionary measures regarding electromagnetic compatibility are required for this device as per the EMC information in the support documentation.
- Do not use the device in environments that contain lint or dust.
- The operator must not use the system and must make the Customer Service Department aware if the FUNDAMENTAL PURPOSE of the device cannot be carried out or is impaired due to ELECTROMAGNETIC FAULTS.
- **PRECAUTION:** The functionality of the device can be impaired by one or more of the following conditions:
 - operation outside the temperature and humidity ranges specified by the manufacturer.
 - Storage outside the temperature and humidity ranges specified by the manufacturer.
 - Mechanical shock (e.g. drop test) or a damaged measurement sensor.
 - The patient's body temperature is lower than that of the surrounding environment.

Battery safety information

- **DANGER TO LIFE!** Keep batteries out of the reach of children. Seek immediate medical attention if swallowed!
-  **EXPLOSION HAZARD!** Never recharge disposable batteries. Never short-circuit and/or open batteries. This could cause batteries to overheat, catch fire or explode.
- Never throw batteries into fire or water.
- Never exert mechanical stress on batteries.

Risk of battery leakage

- Avoid extreme conditions and temperatures which may affect batteries (e.g. radiators / direct sunlight).
- If batteries have leaked, do not allow the chemicals to come into contact with skin, eyes or mucous membranes! Flush the affected areas immediately with clean water and seek medical attention!
-  **WEAR SAFETY GLOVES!** Leaking or damaged batteries can cause burns upon contact with the skin. Wear suitable protective gloves.
- In the event of a battery leak, immediately remove the batteries from the product to prevent damage.
- Only use the same type of batteries. Do not mix old and new batteries!
- Remove the batteries if the product will not be used for an extended period.

Risk of product damage

- Only use the specified battery type!
- Insert the batteries according to the polarity marking (+) and (-) on the battery and the product.
- Clean the contacts on the battery and inside the battery compartment with a dry cloth before inserting the battery!

- Immediately remove drained batteries from the product.
- Do not use rechargeable batteries.

● Prior to first use

● Inserting / replacing batteries

- Open the battery compartment by sliding forward and flipping open the battery cover **2** as shown in Figure C.
- Insert both batteries into the battery compartment in accordance with the correct polarity. Only use 2x 1.5V AAA (LR03) batteries.
- Then close the battery compartment again.

Replace the batteries as follows:

- Replace the battery if  appears in the top right-hand corner of the display.
- When replacing the batteries, follow the previous actions and also note the additional, following step:
- Remove the used batteries from the battery compartment.


● Basic settings

Set the basic settings when using the thermometer for the first time.

You can configure the following basic settings: temperature unit (Celsius/Fahrenheit), date and time. Press and hold down the mode button **5** when the thermometer is switched off to enter the settings mode.


Setting the temperature unit:

- Press the memory button **1** to select the desired unit.
- Press the mode button **5** to confirm the unit. The time format display appears.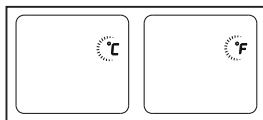


Setting the time format:

- The device can show the time either in 12-hour format (AM/PM) or in 24-hour format (24:00). Press the memory button **1** to select the format.
- Press the mode button **5** to confirm the unit. The hour display automatically starts to flash.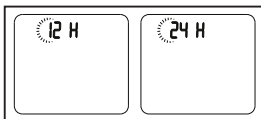

Set hour:

- Press the memory button **1** to move one hour forwards. Keep pressing the memory button **1** until the correct hour is reached.
- Press the mode button **5** to confirm the hour setting. The minute display will automatically begin to flash.

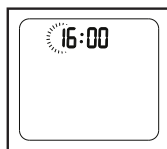
Set minute:

- Press the memory button **1** to move one minute forwards. Keep pressing the memory button **1** until the correct minute is reached.
- Press the mode button **5** to confirm the minute setting. The year display will automatically begin to flash.


Set year:

- Press the memory button **1** to move one year forwards.
- Keep pressing the memory button **1** until the correct year is reached.
- Press the mode button **5** to confirm the year setting. The month display will automatically begin to flash.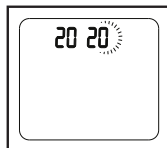


Set month:

- Press the memory button **1** to move one month forwards.
- Keep pressing the memory button **1** until the correct month is reached.
- Press the mode button **5** to confirm the month setting. The day display will automatically begin to flash.

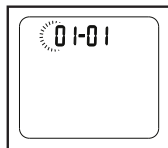Set day:

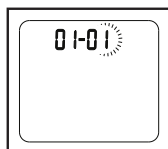- Press the memory button **1** to move one day forwards.
- Keep pressing the memory button **1** until the correct day is reached.
- After setting the correct day, press the mode button **5** to leave the settings mode. The device will save the basic settings and switch off automatically.

● Activating/deactivating sound

You can activate/deactivate sound on the device. When the sound is activated, a beep will be emitted after every measurement.

- To activate the sound, briefly press the sound button **6** when the device is switched on. A loudspeaker symbol  appears on the display **3**.
- To deactivate the sound, press the sound button **6** again. A changed loudspeaker symbol  appears on the display **3**.

● What you should know before taking a measurement

There are different thermometers to measure different parts of the body:

- 
- 
- Ear/forehead thermometer (this thermometer is for measuring on the forehead area)
- Rod thermometer (for rectal [in the anus], axial [under the armpit] or oral measurement [in the mouth])

Note: The temperature measurement shows a value that provides information on an individual's current body temperature.

If you are unsure how to interpret the results or abnormal values appear, you should consult your physician. This also applies for lower temperature changes if other symptoms are present i.e. restlessness, excessive sweating, skin redness, high pulse rate, tendency to collapse, etc.

Temperatures measured using different thermometers cannot be compared with each other. Therefore, you should tell your doctor (or take into account, if diagnosing yourself) which thermometer you used to measure your body temperature and in which part of the body.

The temperature of a healthy person is influenced by many different factors: their individual metabolism which varies from person to person, age (body temperature is higher for infants and toddlers and decreases with age. For children, higher temperature fluctuations occur more readily and more frequently due to growth spurts, for example), clothing, external temperature, time of day (body temperature is lower in the morning and increases throughout the day and in the evening), physical, and to a limited extent, mental activity.

The temperature value varies depending on the part of the body it is being measured from. For healthy people, this deviation can lie between 0.2 °C (0.4 °F) – 1 °C (1.8 °F).

The normal temperature range for

- forehead measurement: 35.8 °C (96.4 °F) – 37.6 °C (99.7 °F) using a forehead thermometer,
- ear measurement: 36.0 °C (96.8 °F) – 37.8 °C (100 °F) using an ear thermometer,

- rectally measured temperature: 36.3°C (97.3°F) – 37.8°C (100°F) using a conventional thermometer,
- orally measured temperature: 36.0°C (96.8°F) – 37.4°C (99.3°F) using a conventional thermometer.

To be able to measure a temperature progression you must always measure from the same part of the body.

● Taking a measurement

Note: Check before each use to see if the measurement sensor 7 has been damaged. If it is damaged, contact your specialist dealer or the service address. Please note that the infrared thermometer must be placed in the room in which the measurement will be taken for at least 30 minutes before the measurement is carried out.

- Wait approx. 30 minutes after sporting activity, after a bath or if you were outdoors before taking a temperature from the forehead.
- Take your temperature regularly to discover your typical average temperature. If you suspect you have a fever, compare your typical average temperature with the current temperature that was measured.
- Ensure before every measurement that the device is in the mode which you want to use to take a measurement.

Mode	Forehead	Object
Display		

● Measuring body temperature from the forehead

Note:


Take into account that

- physical activity, increased formation of sweat on the forehead in general, taking vasoconstrictive medication and skin irritation can falsify the measurement result,
- the forehead must be free from sweat, hair, cosmetics and dirt. If the forehead area is covered, clean it and wait for 10 minutes to improve the accuracy of measurements.

We recommend taking three measurements from the forehead.

If the three results vary, take the highest value.

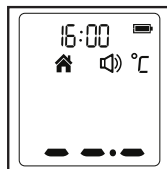- In order to switch on the thermometer, press the measurement button **4**. The display activates and shows all segments as illustrated in Figure B. The adjacent display appears after the self-test, along with a beep. At this point the measurement can start.

- Point the thermometer towards the middle of the forehead (see illustration) at a distance of less than 5 cm, then press the measurement button **4**. The measured value appears on the display **3** and the end of the measuring process is indicated by a short beep (if sound is activated).

- Read the temperature off the display.
The colours on the display alongside the temperature also show if there is a fever:
up to 37.5 °C (99.5 °F) = green,
37.6–37.9 °C (99.6–100.2 °F) = orange,

from 38.0 °C (100.4 °F) = red

If the temperature is in the fever range (red display) then a fever alarm is activated (10 beeps).

- The device automatically switches off after 60 seconds.
- Clean the device after every use as described in the chapter "Cleaning and care".

● Measuring the temperature of objects

- Press the measurement button **4** to switch on the thermometer.
- Press the mode button **5** to switch between forehead and object mode. After switching over to the  measurement mode, you can measure the temperature of an object.
- Point the thermometer towards the middle of the object or liquid that is to be measured (never immerse in liquids!) at a distance of less than 5cm, then press the measurement button **4**.
- The measured value appears on the display **3** and the end of the measuring process is indicated by a short beep (if sound is activated). The device automatically switches off after 60 seconds.
- Clean the device after every use as described in the chapter "Cleaning and care".
- Please note that the displayed temperature is the temperature which has been determined and is not the adjusted surface temperature. It cannot be compared with the forehead temperature.

● Displaying saved measured values

The device saves the measured values of the last 30 measurements automatically. The date, time, and mode are saved with each measurement. If the 30 memory spaces have been exceeded, the respective oldest measured value will be deleted.

The saved measured values can be requested as follows:

- Press the measurement button **4** to switch on the thermometer.

- 
- 
- Press the memory button **1**. Every time the memory button **1** is pressed, the display shows previous measurements taken, which are consecutively numbered from 1 to 30. The most recent measurement is labelled with the number 1 and the oldest has the number 30.
- The user can press the measurement button **4** to carry out new measurements. The device automatically switches off after 60 seconds.

● Restoring the device to its factory settings

You can delete all basic settings stored on the device by taking the batteries out of it. Saved measurements will still be stored.

● Cleaning and care

ATTENTION!

- 
- 
- Possible property damage! Never hold the product under water as liquid could enter the product and damage it.
 - The measurement sensor **7** must always be dry, clean and intact to ensure accurate measurements. Accuracy of temperature measurements can be impaired by damage to the measurement sensor **7** or the presence of dirt, fingerprints, dust and other contaminants on the measurement sensor **7**. Damaged measurement sensors **7** can impair the performance or cause other issues.
 - Never immerse the thermometer directly in water.
 - The measurement sensor **7** is the most sensitive part of the thermometer. Wipe the measurement sensor **7** and the thermometer with a soft cloth that is slightly dampened with 75% isopropyl alcohol solution. Do not use abrasive cleaning agents. Leave the appliance to dry for at least 10 minutes before taking temperatures.
- Note:** Only use isopropyl alcohol to clean the measurement sensor **7**.

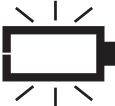
- 
- 
- Clean the display **3** and the outside of the thermometer with a soft, damp cloth.
- Store the thermometer in a dry location, free from dust and dirt, as well as being protected from direct sunlight.
- Put the thermometer back in its original packaging after use.

● **Storage**

- If you intend to store the device for an extended period, please remove the batteries.
- The device must not be stored or used at temperatures which are too high or too low, at a humidity which is too high or too low (see technical data), in sunlight, connected to an electrical current, or in a dusty location. Otherwise, there could be measurement inaccuracies.

● Troubleshooting

Error message	Cause	Solution
	The temperature determined is higher than 1) Forehead thermometer mode: 43 °C (109.4 °F), 2) Object temperature mode: 100 °C (212 °F).	Only use the thermometer within the temperature ranges stated. Clean the measurement sensor  if necessary. If the same error message pops up repeatedly, please contact your specialist dealer or customer service.
	The temperature determined is lower than 1) Forehead thermometer mode: 34 °C (93.2 °F), 2) Object temperature mode: 0 °C (32 °F).	
	The thermometer is not working properly.	Take out the batteries, wait a minute, then switch the appliance on again. If the message appears again then contact your specialist dealer for assistance.
	The ambient temperature is not between 5 °C and 40 °C (41 °F - 104 °F).	Store the thermometer for at least 30 minutes in a room with a temperature between 5 °C and 40 °C (41 °F - 104 °F).

Error message	Cause	Solution
	The thermometer is working properly.	Use the thermometer as usual.
	If the battery display flashes then it means that the batteries are low, but you can continue with the measurement.	The thermometer will measure properly, but the batteries must be replaced soon.
	The thermometer is not working as the batteries are low.	Insert two new AAA alkaline batteries.

● Disposal

Please note the markings on the packaging when sorting waste, these are marked with abbreviations (a) and numbers (b) meaning: 1-7: plastics / 20-22: paper and cardboard / 80-98: composites.

To help protect the environment, please dispose of your product properly when it has reached the end of its useful life, and not in the household waste. Please contact your municipality for information on collection facilities and their opening hours.

The product incl. accessories and packaging materials are recyclable and are subject to extended producer responsibility. Dispose them

separately, following the illustrated Info-tri (sorting information), for better waste treatment. The Triman logo is valid in France only.

Faulty or used batteries/rechargeable batteries must be recycled in accordance with Directive 2006/66/EC and its amendments. Please return the batteries/rechargeable batteries and/or the product to the available collection points.

Environmental damage through incorrect disposal of the batteries/rechargeable batteries!

Batteries/rechargeable batteries may not be disposed of with the usual domestic waste. They may contain toxic heavy metals and are subject to hazardous waste treatment rules and regulations. The chemical symbols for heavy metals are as follows: Cd = cadmium, Hg = mercury, Pb = lead. That is why you should dispose of used batteries/rechargeable batteries at a local collection point.

● Warranty / service

Hans Dinslage GmbH, Riedlinger Straße 28, 89524 Uttenweiler (hereafter referred to as "HaDi") provides a guarantee for this product under the following prerequisites and the product warranty described hereafter.

In case of defects, warranty claims do not affect the legal warranty rights of the customer which emerge from the purchase contract. The customer may exercise their legal warranty rights free of charge. Furthermore, the warranty also applies in spite of obligatory legal liability provisions.

HaDi guarantees the fault-free function and completeness of this product.

The worldwide warranty is 3 years starting with the purchase of the new, unused product by the customer.

This warranty only applies to products that the customer has purchased and used for personal purposes in domestic contexts as a consumer. Subject to German law.

If the product proves to be incomplete or defective in terms of its function during the warranty period and according to the following stipulations, HaDi will provide a free replacement or repair in accordance with these warranty conditions.

If the customer wishes to make a warranty claim, they should first contact HaDi customer service:

Service Hotline (toll-free):

GB Tel.: 0800 931 0319

IE Tel.: 1800 939 235

Email address:

GB service-uk@sanitas-online.de

IE service-ie@sanitas-online.de

Then, the customer will receive additional information about the processing of the warranty claim, e.g. where to send the product free of charge and which documents are required.

If we ask the customer to return the defective product, please send it to the following address:

NU Service GmbH
Lessingstraße 10b
89231 Neu-Ulm
GERMANY

Warranty claims are only considered if the customer can submit

- a copy of the invoice/receipt and
- the original product

to HaDi or an authorised HaDi partner.

The following is not covered by this warranty:

- wear, which is the result of normal product use
- accessories included with this product, which become worn while used as intended (e.g. batteries, cuffs, seals, electrodes, illuminants, attachments);
- products that have been used improperly and/or used, cleaned, stored, or serviced in violation of the guidelines of the instructions for use, as well as products that were opened, repaired, or retrofitted by the customer or a service centre not authorised by HaDi;
- damage that occurs in transit from the manufacturer and the customer or between the service centre and the customer;
- products purchased used or as factory seconds;
- consequential damage based on a defect of this product (in this case, however, only product liability claims or other mandatory liability provisions may arise).

Repairs or replacements never extend the warranty period.

Directives

This appliance complies with the following norms:

ISO 80601-2-56 Medical electrical equipment – Part 2-56: Particular requirements for basic safety and essential performance of clinical

thermometers for body temperature measurement, IEC 60601-1-11 Medical electrical equipment – Part 1-1 1: General requirements for basic safety and essential performance - Collateral Standard: Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment and also satisfies the requirements set out in norms IEC 60601-1-2 (EMC) (group 1, class B, in line with CISPR 11, IEC61000-4-2, IEC61000-4-3, IEC61000-4-8) as well as IEC 60601-1 (safety). The manufacturer is also certified in accordance with ISO 13485.

Note on reporting incidents

For users / patients in the European Union and identical regulation systems (Medical Device Regulation, MDR (EU) 2017/745), the following applies: if, during or as a result of the application of the product, a serious incident occurs, report this to the manufacturer and /or their authorised representative and to the relevant national authority of the member state, where the user/patient is.

The CE declaration of conformity for this product is available at:
www.sanitas-online.de/web/de/landingpages/cedclarationofconformity.php

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