WÄRMEGÜRTEL/ LOWER BACK HEAT WRAP / CEINTURE CHAUFFANTE

(DE) (AT) (CH)

WÄRMEGÜRTEL

Gebruiksaanweisung (2–3)

(GB) (IE) (NI) (MT)

LOWER BACK HEAT WRAP

Instructions for use (3–4)

(FR) (BE) (CH)

CEINTURE CHAUFFANTE

Mode d'emploi (4–6)

(NL) (BE)

WARMTEGORDEL

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(PL)

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(CZ)

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(SK)

TERMOOPASOK

Návod na použitie (11–12)

(ES)

FAJA LUMBAR TÉRMICA

Instrucciones de uso (12–14)

(DK)

VARMEBÆLTE

Brugsanvisning (14–15)

(IT) (MT) (CH)

FASCIA TERMICA

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(HU)

MELEGÍTŐ ÖV

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IAN 460434_2401

(DE) (AT) (FR) (NL) (BE) (PL)

(CZ) (SK) (DK) (IT) (HU)

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GB **IE** **NI** **MT**

Congratulations on the purchase of your new product. You have chosen a high-quality article. Please familiarise yourself with the instructions and safety information before using this product. The instructions for use are part of this product. They contain important information on safety and use. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the product and be sure to include them when handing on the product to another person.

Medical device

Only intended for private use!

Note:

The medical device is intended for adults between the ages of 18 and 65 who can read, understand and follow these instructions for use on their own.

A reason for use, as named under Indications, must be present at the user.

The item is not intended for groups of persons who are excluded from its application by the contraindications listed below.

Scope of delivery

1 x textile wrap

4 x heating pads

1 x instructions for use

Intended purpose

Item for potential pain relief through heat therapy

Indications

Back pain.

Contraindications

Please do not use the item or consult your doctor or pharmacist before using this product if you:

- have poor blood circulation or heart disease,
- have diabetes,
- have nerve damage or a sensory function disorder,
- have rheumatoid arthritis,
- are pregnant,
- have menstrual cramps or irritable bowel syndrome,
- have allergies to medicines or cosmetics.

Do not use:

- whilst sleeping
- in combination with creams, lotions or sprays that are applied to the skin and also not in combination with or near other sources of heat.
- in combination with or near other sources of heat.
- in places where redness, inflammation, skin cracks, bruising or skin irritation may be present.
- in body parts where warmth is not perceptible.
- for bedridden patients and patients with a tendency to develop skin ulcers.

Use:

- Open the sachet by hand (do not use scissors/knife) and remove the heating pad.
- Place one heating pad in each of the two pockets of the warming belt so that the printed side faces the body.
- Insert no more than one pad per pocket
- Position the pockets filled with the heating pads on the painful area around the lumbar spine or hip.
- Wrap the warming belt exactly around the body and secure it with the Velcro fastener.
- After use, remove the heating pads from the warming belt and dispose of them.
- The warming belt is available for the next application.
- Since the heating pads are individually packaged, it is possible to use only one heating pad to relieve localised pain in the lumbar vertebra or hip area.

Notes:

- The heating pad are activated by air.
- Therefore do not open the sachet until just before using it.
- The heating pad will be activated within one minute and will warm up within 5 minutes after the sachet has been opened.
- For external use only.
- Every heating pad is designed for a single application.
- It is normal to observe a slight redness of the skin after use. If the skin is still reddened after a few hours, the application should be stopped until the redness has completely disappeared.

 **If used incorrectly, the product may cause burns even at low temperatures.**

 Follow the instructions and check the application site regularly.

 Allow air to circulate over the heating pad and do not apply pressure, for example by lying or sitting on the heating pad or wearing a tight belt/clothes over the warming belt or leaning against a hard surface.

 Do not use longer than 8 hours per day (=24-hour period) in the same place.

 If the warming belt becomes uncomfortably hot or a skin rash develops, stop use immediately.

 Use only with careful supervision for persons who cannot remove the product by themselves, such as elderly or physically disabled persons.

 If symptoms persist or worsen, seek medical attention.

 Do not use the warming belt while sleeping.

 Do not use on any part of the body other than the lumbar region or lower back.

 For sensitive or mature skin, wear a thin garment under the warming belt.

 **The product may not**

- come into contact with water or other liquids.
- be kneaded, cut, ripped or folded.
- be re-warmed.

Ingredients of the warming plaster:

iron, vermiculite, activated charcoal, sodium chloride, water

 **The ingredients can be harmful if swallowed or come into direct contact with the skin or eyes:**

- If contents have been swallowed, rinse mouth with water.
- If the contents come into direct contact with the skin or eyes, immediately rinse with water.
- In both cases, immediate medical advice should be sought.

Storage/minimum shelf life:

- Store in a cool, dry place away from direct sunlight.
- Keep out of the reach of children and pets.
- Use only until the best before date indicated on the sachet.

Notes on disposal

- Dispose of with normal household waste
- The product must not be incinerated.

Obligation to notify

All serious incidents relating to the product must be reported immediately to the manufacturer and the competent authority.

Service Germany

Tel.: +49 (0)40 82 22 44-80

E-Mail: support@horizonte.de

Consult instructions for use

Attention: Always adhere to the stated warning information

Not to be re-used

Protect against getting wet

Protect from light

Do not use if the packaging is damaged

Catalogue number

Batch designation

Use by

Date of manufacture (year): 2024

Manufacturer

Authorized Representative in the European Community

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INDUSTRY CO., LTD.**