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DE AT CH

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IT CH

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E-Mail: [deltasport@lidl.ch](mailto:deltasport@lidl.ch)

Congratulations!

You have chosen to purchase a high-quality product. Familiarise yourself with the product before using it for the first time.

**Read the following instructions for use carefully.**

Use the product only as described and only for the given areas of application. Store these instructions for use carefully. When passing the product on to third parties, please also hand over all accompanying documents.

This product conforms to medical device guideline 93/42/EEC.

## Scope of delivery

2 x finger bandages

2 x toe bandages

1 x instructions for use

## Symbols Used

Follow instructions for use

Manufacturing date (year - month)

Lot identification

Manufacturer

Instructions for use as PDF on the Lidl Service website.

## Technical Data

Material: 100% TPE gel; 85% polyamide, 15% elastane

## Designated Use

This product is intended for private use.

### Intended use:

- Alleviates pressure and friction discomfort

### Do not use if you have any

- Circulatory disorders
- Open wounds
- Skin diseases

## Safety Advice

### Risk of injury!

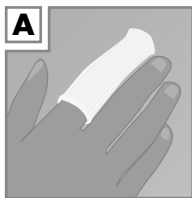
- If you experience any problems when wearing the bandage, remove it immediately and seek the advice of a doctor.
- Ensure that the bandage fits snugly but is not too tight. Ask your doctor if you are unsure!
- Check the bandage for tears or damage before every use.
- This bandage should only be used when in perfect condition.
- Do not wear this bandage during sleep.
- Do not wear the bandage continually for longer than 1-2 hours and no longer than 6-8 hours a day.

### Risk of skin reaction and irritation!

- In some cases, this bandage can cause skin to become irritated or cause an allergic reaction.
- Wash the areas of the skin which are to be covered by the bandage regularly in order to minimise skin reaction and irritation.
- In order to minimise skin reaction and irritation, the skin must be free of gel, grease, creams, balms, oils or similar. If problems persist, seek the advice of a doctor immediately.

## Using this article

Note: The bandage can be cut to the desired size as required.

### **WARNING!**

Ensure that the bandage fits snugly but is not too tight. Ensure that the blood circulation in the finger / toe is not affected. Do not wear the

bandage continually for longer than 1-2 hours and no longer than 6-8 hours a day. Pull the bandage over the fingers and toes as shown.

## Cleaning and Care

- Before using the bandage for the first time, wash it thoroughly.
- It is advisable to carry out a further wash after 2 or 3 times of use.
- After washing, you should rinse your bandage thoroughly.
- Wash with similar colours only.

## Disposal

Dispose of the article and the packaging materials in accordance with current local regulations. Packaging materials such as foil bags are not suitable to be given to children. Keep the packaging materials out of the reach of children.

Dispose of the products and the packaging in an environmentally friendly manner.

The recycling code is used to identify various materials for recycling. The code consists of the recycling symbol – which is meant to reflect the recycling cycle – and a number which identifies the material.

## Notes on the guarantee and service handling

The product was produced with great care and under continuous quality control. DELTA-SPORT HANDELSKONTOR GmbH gives private end customers a three-year guarantee on this product from the date of purchase (guarantee period) in accordance with the following provisions. The guarantee is only valid for material and manufacturing defects. The guarantee does not cover parts subject to normal wear and tear that are thus considered wear parts (e.g. batteries) or fragile parts such as switches, rechargeable batteries, or parts made of glass.

Claims under this guarantee are excluded if the product has been used incorrectly, improperly, or contrary to the intended purpose, or if the provisions in the instructions for use were not observed, unless the end customer proves that a material or manufacturing defect exists that was not caused by one of the aforementioned circumstances.

Claims under the guarantee can only be made within the guarantee period by presenting the original sales receipt. Please therefore keep the original sales receipt. The guarantee period is not extended by any repairs carried out under the guarantee, under statutory guarantees, or as a gesture of goodwill. This also applies to replaced and repaired parts.

If you wish to make a claim please first contact the service hotline mentioned below or contact us by e-mail. If there is a guarantee case, then the product will be repaired or replaced free of charge to you or the purchase price will be refunded, depending on our choice.

Your legal rights, in particular guarantee claims against the respective seller, are not limited by this guarantee.

IAN: 353736\_2010

 Service Great Britain  
Tel.: 0800 404 7657  
E-Mail: [deltasport@lidl.co.uk](mailto:deltasport@lidl.co.uk)

**DELTA-SPORT HANDELSKONTOR GMBH**

Wragekamp 6 • DE-22397 Hamburg  
GERMANY

2021-03

02021353736

Druckversion / Version imprimable / Versione stampata / Printable Version:  
2021-03

Delta-Sport-Nr.: BG-1417

01.15.2021 / PM 12:33

IAN 353736\_2010

1