BATH SEAT

(GB) (IE) (NI)

BATH SEAT

Instructions for use

(FR) (BE)

SIÈGE DE BAIN

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(DE) (AT) (CH)

BADEWANNENSITZ

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(DK)

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(NL) (BE)

BADKUIP- ZITTING

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IAN 315318

(GB) (IE) (NI) (DK) (FR)
(BE) (NL) (DE) (AT) (CH)

Congratulations!

You have chosen to purchase a high-quality item. Familiarise yourself with the item before using it for the first time.

 **Read the following instructions for use carefully.**

Only use this item in the manner described and for the purpose for which it is intended. Store these instructions for use carefully. When passing the item on to third parties, please also hand over all accompanying documents.

Package contents

1 x bath seat

1 x instructions for use

Technical data

Max. user weight: 150kg

Overall dimensions: approx. 75 x 23 x 18cm
(W x D x H)

 Date of manufacture (month/year):
05/2019

Intended use

This item is to be used exclusively as a bath seat, to facilitate easier washing.

The item has been developed for standard bathtubs with an inner dimension of 580–620mm. The item is not suitable for use by people with impaired mental abilities. It is intended for use in the bathing and shower area of a private bathroom and only by one person weighing a maximum of 150kg.

This item is not a medical device and was not developed for the rehabilitation and medical care of individuals.

Safety information

Risk of injury!

WARNING! Suffocation risk for children! Do not let children play unsupervised with the packaging material or the product.

- Keep children away from the product. They could try to stand on it or pull themselves up on it and topple over with it in the process.
- The item is not a toy!

- Use the item for its intended purpose only.
- The seat may only be used on a bathtub.
- The product may only be used by persons who are familiar with the safety instructions.
- The item may not be loaded by more than 150kg.
- Check the item for damage or wear before each use. Before each use, check that the item is sitting correctly and in the right position on the rim of the bathtub! The item should only be used if it is in perfect condition! Do not use the item if it is visibly damaged or you suspect it is damaged!
- Do not modify the item and do not attach any additional elements to it.
- The item may only be used on bathtubs where you are sure the support is stable and non-slip!
- Ensure that the item is lying securely on the rim of the bathtub!
- Only use the item when its surface temperature is quite safe. The surface of the item can acquire temperatures injurious to health due to hot water, sun rays, sauna conditions, extreme cold etc.
- Slip hazard! Check the seat is fitted securely before each use and make sure that the surface inside and outside the bath tub is non-slip.
- Smooth or damp surfaces can impair the non-slip properties.
- Do not lift with a person on it.
- Do not step on the seat.

Beware of damaging the item!

- Do not use caustic or abrasive detergents.
- No liability will be accepted for damages of any kind to the bath seat.
- Examine the item regularly every 3 months to check the screws and all connections are fitting tightly and correctly and retighten if necessary. Contact the product hotline if you are in doubt.

Putting the seat on (Fig. A)

Caution!

- The item may only be used on bathtubs where you are sure the support is stable and non-slip!

Caution!

- Make sure that all four handles of the item are lying securely on the rim of the bath!
- Place the item on the rim of the bathtub as shown in Fig. A.

Use

Get into the bath and sit down on the seat carefully.

Repairs

Do not modify the item and do not repair it yourself! Purchase spare parts from the manufacturer or a specialist shops only. Improper repairs can cause unforeseen harm!

Maintenance

Clean the item as needed with a damp cloth and use a mild detergent if required. Afterwards, dry the item completely before using it again.

Disposal

Dispose of the article and the packaging materials in accordance with current local regulations. Packaging materials such as foil bags are not suitable to be given to children. Keep the packaging materials out of the reach of children.

Dispose of the products and the packaging in an environmentally friendly manner.

The recycling code is used to identify various materials for recycling.

The code consists of the recycling symbol – which is meant to reflect the recycling cycle – and a number which identifies the material.

Notes on the guarantee and service handling

The product was produced with great care and under constant supervision. You receive a three-year warranty for this product from the date of purchase. Please retain your receipt.

The warranty applies only to material and workmanship and does not apply to misuse or improper handling. Your statutory rights, especially the warranty rights, are not affected by this warranty.

With regard to complaints, please contact the following service hotline or contact us by e-mail. Our service employees will advise as to the subsequent procedure as quickly as possible. We will be personally available to discuss the situation with you.

Any repairs under the warranty, statutory guarantees or through goodwill do not extend the warranty period. This also applies to replaced and repaired parts.

Repairs after the warranty are subject to a charge.

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