## HANDGELENKBANDAGE/ WRIST BRACE / BANDAGE POIGNET

DE AT CH

### HANDGELENKBANDAGE

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GB IE NI MT

### WRIST BRACE

Instructions for use (3-4)

FR BE CH

### BANDAGE POIGNET

Mode d'emploi (5-6)

NL BE

### POLSBANDAGE

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PL

### OPASKA NA NADGARSTEK

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CZ

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SK

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Návod na použitie (11-12)

ES

### MUÑEQUERA

Instrucciones de uso (13-14)

DK

### HÅNDELSBANDAGE

Brugsanvisning (14-16)

IT MT CH

### POLSIERA ELASTICA

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HU

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Congratulations on the purchase of your new product. You have chosen a high-quality article. The instructions for use are part of this product. They contain important information on safety and use. Please familiarise yourself with the instructions and safety information before using this product. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the product and be sure to include them when handing on the product to another person.

## Medical device

### Only intended for private use!

### Note:

The medical device is intended for adults who are able to read, understand and independently follow these instructions for use. A reason for use, as named under Indications, must be present at the user.

The item is not intended for groups of persons who are excluded from its application by the contraindications listed below.

## Scope of delivery

- 1 x wrist brace
- 1 x stabilising bar
- 1 x elastic wrist strap
- 1 x instructions for use

## Technical Data

Size	Wrist circumference
S-M	15 – 17 cm
L	17 – 19 cm
XL-XXL	19 – 21 cm

## Intended purpose

The brace stabilises, supports and relieves.

The ergonomically shaped joint rod and the elastic wrist strap serve to stabilise the wrist, the built-in pad promotes the blood circulation and improves the muscle function.

## Indications

To be used as support for:

- Inflammatory arthropathies (rheumatoid arthritis, rheumatism)
- Joint pain due to degradation (osteoarthritis)
- Irritation from overworking
- Typist's (tendovaginitis)
- Bruises
- Feeling of instability
- Carpal tunnel syndrome
- Distorsion

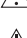
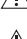
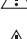
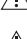
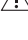
## Contraindications

 **Do not use – or use only after consultation with your doctor – for:**

- External damage
- Bone fractures
- Circulatory disorders
- Higher degree of instability in wrist
- Neurological deficits such as paralysis, radiating nerve pain or similar.

## Safety instructions

### Risk of injury

-  If problems occur while wearing the bandage, remove it immediately and consult a doctor.
-  Check the bandage before each use for tears and damage.
-  The product should only be used in perfect condition.
-  Make sure the bandage is comfortable but not too tight.
-  Never put the bandage on so tightly that the blood circulation is restricted.
-  Do not wear the bandage longer than 2 to 3 hours a stretch and a maximum of 6 to 8 hours a day.
-  Do not wear the bandage while sleeping or during longer rest phases such as long periods of sitting.
-  Wash the areas of skin covered by the bandage thoroughly and regularly.
-  To avoid allergic skin reaction, the skin must be free of grease, gels, creams, oils or similar materials.
-  The article may only be used by other people when it has been cleaned.

### Possible side effects

-  If you experience symptoms (such as, e.g. tingling, pain, swelling) while wearing the bandage, remove the bandage immediately!
-  There is a risk of skin irritation and skin irritations! In individual cases, skin irritation and reactions and allergic reactions may occur. Make sure that the stated maximum wearing periods are not exceeded and the areas of skin covered by the bandage are thoroughly and regularly washed. Should any problems or complaints occur, consult a doctor immediately!

## Positioning of the product

- Pull the article on from below through your fingers and stick your thumb gently into the thumb hole (**Fig. 1**). When the bandage is correctly applied, the thumb-hole sits over the base of the thumb without tension. The brace is pleasantly firm, but not too tight.
- The included elastic wrist strap can be individually wrapped around the wrist brace for additional stabilisation (**Fig. 2**).
- The stabiliser bar is in the brace and can be removed from it, if necessary (**Fig. 3**).

**Note:** The statements/illustrations shown are for the wrist brace on both the left or the right.

## Cleaning and care

- Thoroughly wash the brace before first use. Remove the stabiliser bar before cleaning (**Fig. 3**). Remove the elastic brace before washing the bandage.
- It's advisable to wash the brace again every second or third use.
- Reinsert the stabilising rod after washing.
- Wash separately.

## Instructions for disposal

Please dispose of the product in an environmentally-friendly way through an approved disposal centre or your local waste facility. Observe currently applicable regulations.

## Obligation to notify

All serious incidents relating to the product must be reported immediately to the manufacturer and the competent authority.

## 3-year warranty

The product has been created with great care and under constant monitoring. You will receive a 3-year warranty for this product from the date of purchase. Please keep the receipt. The warranty is valid only for material and manufacturing faults and lapses in the event of misuse or improper handling. Your statutory rights, particularly the warranty rights, are not restricted by this warranty. If you have any complaints, please contact the service hotline below or contact us by email. Our service staff will discuss how to proceed further with you as soon as possible. In any case, we will personally advise you. The warranty period is not extended by any repairs under the terms of the warranty, statutory warranty or goodwill. This also applies to replaced and repaired parts. After expiry of the warranty, incurred repairs are chargeable.

Service Germany

Tel.: +49 (0)40 82 22 44-80

E-Mail: support@horizonte.de

-  Consult instructions for use
-  Attention: Always adhere to the stated warning information
-  Catalogue number
-  Batch designation
-  Use by
-  Date of manufacture (year): 2024
-  Manufacturer
-  CE Marking of conformity
-  Medical device

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