WÄRMEUMSCHLAG HEAT WRAP COMPRESSE CHAUFFANTE

DE AT CH

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GB IE

HEAT WRAP

Instructions for use

FR BE

COMPRESSE CHAUFFANTE

Mode d'emploi

NL BE

WARMTECOMPRES

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PL

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CZ

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SK

TERMOOBKLAD

Návod na použitie

IAN 324657_1901

DE AT CH GB IE FR

BE NL PL CZ SK

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Congratulations on the purchase of your new product. You have chosen a high-quality article. The instructions for use are part of this product. They contain important information on safety and use. Please familiarise yourself with the instructions and safety information before using this product. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the product and be sure to include them when handing on the product to another person.

Only intended for private use!

Scope of delivery

8 x heat wrap

1 x instructions for use

Intended use

Can be used to relieve pain in muscles and joints

Use:

- Open the sachet by hand (do not use scissors/knife) and remove the heat wrap.
- Carefully remove the protective film.
- Apply the plaster over the painful area with the sticky side to the skin.
- Apply the wrap with the sticky side to the skin over the painful area.
- After use, gradually remove the heat wrap and dispose of it.

Notes:

- The heat wrap is activated by air.
- Therefore do not open the sachet until just before using it.
- The heat wrap will be activated within one minute and will warm up within 5 minutes after the sachet has been opened.
- For external use only.
- Every heat wrap is designed for a single application.
- It is normal to observe a slight redness of the skin after use. If the skin is still reddened after a few hours, the application should be stopped until the redness has completely disappeared.

 **If used incorrectly, the product may cause burns even at low temperatures.**

 Follow the instructions and check the application site regularly.

 Allow air to circulate over the heat wrap and do not apply pressure, for example by lying or sitting on the heat wrap or wearing tight-fitting clothing over the heat wrap.

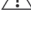
 Do not use longer than 8 hours per day (=24-hour period) in the same place.

 If the heat wrap becomes uncomfortably hot or a skin rash develops, stop use immediately.

 Use only with careful supervision for persons who cannot remove the product by themselves, such as elderly or physically disabled persons.

 If symptoms persist or worsen, seek medical attention.

 Do not use the heat wrap while sleeping.

 Do not use on any part of the body other than the back.

 For sensitive or mature skin, apply the product to clothing.

Restrictions on use

 **Please consult your doctor or pharmacist before using this product if you:**

- suffer from poor circulation or heart disease,
- suffer from diabetes,
- suffer from nerve damage or a disorder of the sensory functions,
- suffer from a skin condition,
- suffer from rheumatoid arthritis or are pregnant.

 **Do not use:**

- for children under 12;
- whilst sleeping
- for menstrual complaints or irritable bowel syndrome
- in combination with creams, lotions or sprays that are applied to the skin and also not in combination with or near other sources of heat.
- in combination with or near other sources of heat.
- in places where redness, inflammation, skin cracks, bruising or skin irritation may be present.
- in body parts where warmth is not perceptible.
- for patients who have experienced an allergic reaction to medication or cosmetics in the past
- for bedridden patients and patients with a tendency to develop skin ulcers.

The product may not

- come into contact with water or other liquids.
- be kneaded, cut, ripped or folded.
- be re-warmed.

Ingredients of the heat wrap:

iron, vermiculite, activated charcoal, sodium chloride, water

The ingredients can be harmful if swallowed or come into direct contact with the skin or eyes:

- If contents have been swallowed, rinse mouth with water.
- If the contents come into direct contact with the skin or eyes, immediately rinse with water.
- In both cases, immediate medical advice should be sought.

Storage/minimum shelf life:

- Store in a cool, dry place away from direct sunlight.
- Keep out of the reach of children and pets.
- Use only until the best before date indicated on the sachet.

Notes on disposal

- Dispose of with normal household waste
- The product must not be incinerated.

Service Germany

Tel.: +49 (0)40 82 22 44-80

E-Mail: info@horizonte.de

-  Consult instructions for use
-  Always adhere to the stated warning information
-  Not to be re-used
-  Protect against getting wet
-  Protect from light
-  Do not use if the packaging is damaged
-  Batch designation
-  Durability
-  Manufacturer
-  Authorized Representative in the European Community
-  Complies with Directive 93/42/EEC

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INDUSTRY CO., LTD.**