SPORTS KNEE BRACE / SPORTBANDAGE KNIEGELENK / GENOUILLÈRE DE MAINTIEN

GB / IE / NI / MT

SPORTS KNEE BRACE

Instructions for use

(2-3)

DE / AT / BE / CH

SPORTBANDAGE KNIEGELENK

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(3-4)

FR / BE / CH

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Mode d'emploi

(4-6)

NL / BE

SPORT KNIEBRACE

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(6-7)

IT / CH / MT

GINOCCHIERA ELASTICA

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(8-9)

ES

RODILLERA DEPORTIVA

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(9-10)

PT

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(11-12)

DK

SPORTSBANDAGE KNÆLED

Bruksanvisning

(12-13)

PL

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(13-14)

HU

TÉRDBANDÁZS

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(15-16)

SI

ŠPORTNI POVOJ ZA KOLENO

Navodila za uporabo

(16-17)

CZ

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(17-18)

SK

ŠPORTOVÁ BANDÁŽ NA KOLENO

Návod na použitie

(19-20)

RO

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(21-24)

GB / IE / NI / MT

The instructions for use are part of this product. They contain important information on safety, use and disposal. Please familiarise yourself with the instructions and safety information before using this product. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging, including this leaflet, for as long as you use the product and be sure to include it when handing the product on to another person.

Medical device

Only intended for private use!

Note

The medical device is intended for adults who are able to read, understand and independently follow these instructions for use. A reason for use, as named under Indications, must be present at the user. The item is not intended for groups of persons who are excluded from its application by the sizes and contraindications listed below.

Scope of delivery

1 x knee brace
1 x instructions for use

Technical Data

Size	knee circumference
S	35 - 37 cm
M	37 - 39 cm
L	39 - 41 cm
XL	41 - 42,5 cm
To be worn on the left and right	

Intended purpose

Gentle compression and support for joints

Indications

- irritation
- swelling
- post injury
- rheumatic conditions

Contraindications

 **Do not use - or use only after consultation with your doctor - for:**

- acute injuries
- circulatory disorders
- eczema or other skin diseases

Discontinue use if

- unforeseen problems arise
- you are unsure about how to use the product

Safety Advice

Risk of Injury

-  Use the product only in perfect condition, as described and for the specified area of use.
-  Do not wear the bandage longer than 2 to 3 hours at a stretch and a maximum of 6 to 8 hours a day.
-  Pay attention to a sufficient blood circulation.
-  The bandage may only be used on uninjured skin.
-  To avoid allergic skin reaction, the skin must be free of grease, gels, creams, oils or similar materials.

Instructions for disposal

Please dispose of the product in an environmentally-friendly way through an approved disposal centre or your local waste facility. Observe currently applicable regulations.

Obligation to notify

All serious incidents relating to the product must be reported immediately to the manufacturer and the competent authority.

3-year warranty

The product has been created with great care and under constant monitoring. You will receive a 3-year warranty for this product from the date of purchase. Please keep the receipt. The warranty is valid only for material and manufacturing faults and lapses in the event of misuse or improper handling. Your statutory rights, particularly the warranty rights, are not restricted by this warranty. If you have any complaints, please contact the service hotline below or contact us by email. Our service staff will discuss how to proceed further with you as soon as possible. In any case, we will personally advise you. The warranty period is not extended by any repairs under the terms of the warranty, statutory warranty or goodwill. This also applies to replaced and repaired parts. After expiry of the warranty, incurred repairs are chargeable.

Service Germany
Tel.: +49 (0)40 82 22 44-80
E-Mail: support@horizonte.de

-  Consult instructions for use
 Attention: Always adhere to the stated warning information
 Catalogue number
 Batch designation
 Use by
 Date of manufacture (year): 2026
 Manufacturer
 Indicates the authorised representative in Switzerland (for products purchased in Switzerland)
 Importer
 CE Marking of conformity
 Medical device

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Lidl Schweiz DL AG