PDF ONLINE
www.lidl-service.com

RÜCKENBANDAGE LUMBO-ACTIVE/ BACK BRACE LUMBO-ACTIVE / CEINTURE LOMBAIRE LUMBO-ACTIVE

DE AT CH

RÜCKENBANDAGE LUMBO-ACTIVE

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GB IE NI

BACK BRACE LUMBO-ACTIVE

Instructions for use (3-4)

FR BE

CEINTURE LOMBAIRE LUMBO-ACTIVE

Mode d'emploi (4-6)

NL BE

RUGBANDAGE LUMBO-ACTIVE

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PL

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CZ

ZÁDOVÁ BANDÁŽ LUMBO-ACTIVE

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SK

BANDÁŽ NA CHRÁT LUMBO-ACTIVE

Návod na použitie (11-12)

ES

CINTURÓN LUMBAR LUMBO-ACTIVE

Instrucciones de uso (12-14)

DK

RYGBANDAGE LUMBO-ACTIVE

Brugsanvisning (14-15)

IT

FASCIA ELASTICA PER SCHIENA LUMBO-ACTIVE

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HU

HÁTÖV LUMBO-ACTIVE

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SI

BANDAŽA ZA HRBET LUMBO-ACTIVE

Navodila za uporabo (19-20)

IAN 377430_2110

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Artikelnummer

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Hersteller

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Congratulations on the purchase of your new product. You have chosen a high-quality article. The instructions for use are part of this product. They contain important information on safety and use. Please familiarise yourself with the instructions and safety information before using this product. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the brace and be sure to include them when handing on the product to another person.

Medical device

Only intended for private use!

Note:

The medical device is intended for adults who are able to read, understand and independently follow these instructions for use. A reason for use, as named under Indications, must be present at the user.

The item is not intended for groups of persons who are excluded from its application by the sizes and contraindications listed below.

Scope of delivery

1 x back brace

1 x pad

1 x instructions for use

Technical Data

Size	Waist circumference
M-L	80 - 98 cm
L-XXL	96 - 116 cm

Intended purpose

The brace stabilises, supports and relieves.

The height-adjustable and individually positioned lumbar pad also guarantees targeted stabilizing support, promotes blood circulation and improves muscle function.

Indications

To be used as support for:

- Chronic pain in the lower spine (lower back pain)
- First aid for lumbago
- Wear of the small vertebral joints (lumbar facet syndrome)
- Muscle tension
- Muscular imbalance
- Hypermobility of the lumbar spine
- Sports and physical strains

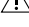
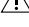
Contraindications

 **Do not use – or use only after consultation with your doctor – in case of:**

- External injuries
- Bone fractures
- Circulatory disorders.
- Heightened instability of the lumbar spine (e.g. spondylolisthesis)
- Neurological deficits such as paralysis of the lower extremities, radiating nerve pain etc.

Safety instructions

Risk of injury

-  If problems occur while wearing the bandage, remove it immediately and consult a doctor.
-  Check the bandage before each use for tears and damage.
-  The product should only be used in perfect condition.
-  Make sure the bandage is comfortable but not too tight.
-  Never put the bandage on so tightly that the blood circulation is restricted.
-  Do not wear the bandage longer than 2 to 3 hours at a stretch and a maximum of 6 to 8 hours a day.
-  Do not wear the bandage while sleeping or during longer rest phases such as long periods of sitting.
-  Wash the areas of skin covered by the bandage thoroughly and regularly.
-  To avoid allergic skin reaction, the skin must be free of grease, gels, creams, oils or similar materials.
-  The article may only be used by other people when it has been cleaned.

Possible side effects

-  If you experience symptoms (such as, e.g. tingling, pain, swelling) while wearing the bandage, remove the bandage immediately!
-  There is a risk of skin irritation and skin irritations! In individual cases, skin irritation and reactions and allergic reactions may occur. Make sure that the stated maximum wearing periods are not exceeded and the areas of skin covered by the bandage are thoroughly and regularly washed. Should any problems or complaints occur, consult a doctor immediately!

Putting the bandage on

- Lay the bandage around the desired area of the back and close the Velcro strip **(A)** of the bandage (**fig. 1**). The bandage should sit comfortably and be slightly stretched.

Lumbar pad

- The included lumbar pad **(B)** is removable and can be individually positioned (**fig. 2**).
- The bandage is placed correctly if the lumbar pad **(C)** sits on the buttocks in a palpable manner (**fig. 3**).
- If the bandage is correctly positioned, the blood circulation is stimulated, the lumbar spine is supported in a focused manner and the muscle function is improved.

Cleaning and care

- Before using it for the first time, the bandage should be thoroughly washed.
- After washing, you should thoroughly rinse your bandage.
- After using it two or three times, it should be washed again.
- Close the Velcro pieces during the washing process.
- Wash separately.
- Wash pad separately in a laundry net bag.
- Wash separately.

Instructions for disposal

Please dispose of the product in an environmentally-friendly way through an approved disposal centre or your local waste facility. Observe currently applicable regulations.

Obligation to notify

All serious incidents relating to the product must be reported immediately to the manufacturer and the competent authority.

3-year warranty

The product has been created with great care and under constant monitoring. You will receive a 3-year warranty for this product from the date of purchase. Please keep the receipt. The warranty is valid only for material and manufacturing faults and lapses in the event of misuse or improper handling. Your statutory rights, particularly the warranty rights, are not restricted by this warranty. If you have any complaints, please contact the service hotline below or contact us by email. Our service staff will discuss how to proceed further with you as soon as possible. In any case, we will personally advise you. The warranty period is not extended by any repairs under the terms of the warranty, statutory warranty or goodwill. This also applies to replaced and repaired parts. After expiry of the warranty, incurred repairs are chargeable.

Service Germany

Tel.: +49 (0)40 82 22 44-80

E-Mail: info@horizonte.de

Consult instructions for use

Attention: Always adhere to the stated warning information

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