WÄRMFLASCHE HOT WATER BOTTLE BOUILLOTTE

(DE) (AT) (CH)

WÄRMFLASCHE

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(FR) (BE)

BOUILLOTTE

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(PL)

TERMOFOR

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(SK)

OHRIEVACIA FĽAŠA

Navod na používanie

(DK)

VARMEDUNK

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(HU)

MELEGVIZES PALACK

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(GB) (IE)

HOT WATER BOTTLE

Instructions for use

(NL) (BE)

KRUIK

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(CZ)

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(ES)

BOLSA DE AGUA CALIENTE

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(IT)

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Istruzioni d'uso

(SI)

TERMOFOR

Navodilo za uporabo

IAN 498149_2204

(DE) (AT) (CH) (GB) (IE) (FR) (BE) (NL)
(PL) (CZ) (SK) (ES) (DK) (IT) (HU) (SI)

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 Service Schweiz
Tel.: 0800 56 44 33
E-Mail: deltasport@lidl.ch

Congratulations!

You have chosen to purchase a high-quality product. Familiarise yourself with the product before using it for the first time.

 **Read the following instructions for use carefully.**

Use the product only as described and only for the given areas of application. Store these instructions for use carefully. When passing the product on to third parties, please also hand over all accompanying documents.

 Class I medical device – This product is compliant with the Regulation (EU) 2017/745 (MDR) concerning medical devices. Please use our service if you have questions about the product or have trouble reading these instructions.

The product is intended for partial filling pursuant to B.S. 1970:2012.

 UK Conformity Assessed

Package contents

1 x hot water bottle
1 x cap
1 x instructions for use

Technical data

Dimensions: approx. 33 x 20cm (H x W)
Capacity: 2,0l
Usage volume: 1,7l
Material: 100% natural rubber

Symbols used

Follow the instructions for use

Date of manufacture (year-month):
2022-08

Manufacturer

Batch number

Product number

Medical device

Keep away from sunlight

Please ensure that you take note of the warning notices or safety information in the instructions for use

Indicates the presence of dry natural rubber or natural rubber latex as a design material in the medical product.

Instructions for use as PDF on the Lidl Service website.

Intended use

This product is intended for your well-being and comfort. It was developed for personal use and is suitable for alleviating pain (where heat therapy is recommended).

Indication:

- Relieves pain stemming from gut motility disorders
- Helps relieve muscle tension
- Relieves menstrual pain

Do not use for

- feverish illnesses, inflammations
- open wounds
- extensive skin injuries or skin diseases, chronic venous insufficiency
- itchy and weeping areas
- skin diseases of any sort
- allergic reactions to the material
- fresh surgical wounds
- fractures (broken bones)
- reduced nerve sensitivity or temperature perception disorders
- chronic venous insufficiency

Contact your doctor or pharmacist if you are uncertain.

Safety information

Important: read these instructions for use carefully and keep them in a safe place!

Life-threatening hazard!

- Do not allow children to play unsupervised with the product or the packaging materials. There is a risk of suffocation!

Risk of injury!

- Do not use boiling water. Hot tap water can also be scalding.
- Do not heat up the product in the oven or microwave!
- The product can cause burns. For this reason, avoid direct contact with the skin or use a suitable hot water bottle cover. Remember that sensitive skin can burn in temperatures 42°C and higher!
- Adult supervision is required when the product is used by children, helpless persons, sick persons or persons insensitive to heat.
- Do not use on your chest, especially near the heart.
- Only use water cooler than 42°C if you are pregnant.
- Repeated use of the product on a specific area of the skin can result in the skin condition erythema ab igne. This heals up once the heat treatment ends. Because children under 36 months of age have considerably more sensitive skin, a surface temperature lower than 42°C must be maintained and direct contact with the skin must be avoided.
- Never sit on, lie on, or use the product as a pillow.
- Inspect the product and cap for damage and wear before each use. For hygienic reasons, we recommend replacing the product after 5 years.
- Only use the original cap.
- Never use the product while asleep.
- Do not use the product uninterrupted for longer than 8–12 hours.

- Check the product for damage or wear before each use. Only use the product if it is in perfect condition!
- Use the product as described in the instructions for use.

Damage to the product

- Do not use water from a domestic hot water supply device. This may significantly reduce the service life of the product.
- Do not fill the product more than 2/3 full, otherwise the product could burst.
- Clean the product if it becomes dirty.
- Replace the product if it is damaged.
- Do not place any additives in the water, such as table salt or similar.
- Avoid contact with hot, pointed or sharp surfaces, oil, grease and sunlight.
- Do not place any objects on the hot water bottle.

Use

1. Unscrew and remove the cap.
2. Hold the product vertically by the neck and fill it up to no more than 2/3 full with hot, but not boiling water.

Note: by holding the product vertically you prevent the water from splashing.

3. Remove the remaining air by carefully laying the bottle on a level surface until the water is visible at the filling opening.
4. Firmly tighten the cap and check that the product is sealed.

Note: there must be no residual water on the cap. Remove this before using the product.

5. Place the product on the part of your body of your choosing.

It is recommended that users, especially those with heat-sensitive skin, use a suitable hot water bottle cover to control the heat output. You must stop using the product immediately if you feel discomfort and/or pain.

Storage, cleaning

When not in use, always store the product clean, dry, unsealed and at room temperature.

 The product must not be exposed to direct sunlight when not in use.

Completely empty the product after each use.

Leave the product unsealed after use so the inside can dry completely.

Clean only with water and wipe dry afterwards with a cloth.

IMPORTANT! Never clean the product with harsh cleaning agents.

Disposal

 Dispose of the product and packaging materials in accordance with current local regulations. Store the packaging materials (foil bags, for example) out of the reach of children. For further information about disposal of the product no longer needed, contact your local council. Dispose of the product and the packaging in an environmentally friendly manner.

 The Recycling Code distinguishes different materials to be returned for recycling. The Code consists of the recycling symbol for the recycling process and a number that identifies the material.

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