## COOLING TUBULAR BANDAGE / SCHLAUCHBANDAGE KÜHLEND / BANDAGE TUBULAIRE REFRROIDISSANT

<b>(GB)</b>	<b>COOLING TUBULAR BANDAGE</b>		<b>(PL)</b>	<b>BANDAŻ CHŁODZĄCY</b>	
<b>(IE)</b>	Instructions for use	(2–4)		Instrukcja użytkowania	(18–20)
<b>(NI)</b>			<b>(HU)</b>	<b>HŰSÍTŐ CSŐBANDÁZS</b>	
<b>(MT)</b>				Használati utasítás	(20–22)
<b>(DE)</b>	<b>SCHLAUCHBANDAGE KÜHLEND</b>		<b>(SI)</b>	<b>HLADILNI CEVASTI POVOJ</b>	
<b>(AT)</b>	Gebrauchsanweisung	(4–6)		Navodila za uporabo	(23–25)
<b>(BE)</b>			<b>(CZ)</b>	<b>CHLADICÍ BANDÁŽ</b>	
<b>(CH)</b>	<b>BANDAGE TUBULAIRE REFRROIDISSANT</b>			Návod k použití	(25–27)
	Mode d'emploi	(6–8)	<b>(SK)</b>	<b>CHLADIACA BANDÁŽ</b>	
<b>(FR)</b>				Návod na použitie	(27–29)
<b>(BE)</b>			<b>(RO)</b>	<b>BANDAJ TUBULAR, RĂCORITOR</b>	
<b>(NL)</b>	<b>SLANGBANDAGE VERKOELEND</b>			Instrucțiuni de utilizare	(29–32)
<b>(BE)</b>	Gebruiksaanwijzing	(8–11)	<b>(BG)</b>	<b>ОХЛАЖДАЩ БАНДАЖ</b>	
<b>(IT)</b>				Инструкция за употреба	(32–34)
<b>(CH)</b>	<b>FASCIA TUBOLARE REFRIGERANTE</b>				
<b>(MT)</b>	Istruzioni per l'uso	(11–13)			
<b>(ES)</b>					
<b>(DK)</b>	<b>VENDAJE REFRIGERANTE</b>				
	Instrucciones de uso	(13–15)			
<b>(DK)</b>					
<b>(DK)</b>	<b>KØLENDE SLANGEBANDAGE</b>				
	Brugsanvisning	(16–18)			

Congratulations on the purchase of your new product. You have chosen a high-quality article. Please familiarise yourself with the instructions and safety information before using this product. The instructions for use are part of this product. They contain important information on safety and use. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the product and be sure to include them when handing on the product to another person.

## Medical device

### Only intended for private use!

#### Note

The medical device is intended for adults who are able to read, understand and independently follow these instructions for use. A reason for use, as named under Indications, must be present at the user. The item is not intended for groups of persons who are excluded from its application by the sizes and contraindications listed below.

## Scope of delivery

- 1 x tubular bandage
- 1 x instructions for use

## Technical Data

Size	Circumferences
S	24 – 30 cm
M	30 – 37 cm
L	37 – 45 cm

Wear on the left and right

## Intended purpose for cooling:

Cooling tubular bandage for external use.  
Can help alleviate pain.

## Indications

### For supporting use in:

- Contusions
- Sprains
- Bruises
- Inflammation
- Swelling

## Cold treatment:

- The resealable packaging is required for storage and pre-cooling of the product.
- The article has a refreshing effect even when not cooled.
- Depending on requirements, the product can be pre-cooled for up to 2 hours at approx. 5 °C in the refrigerator.

- Pre-cooling in the freezer compartment should not take longer than 1 hour.
- Before use – especially after pre-cooling – check whether the temperature is appropriate for your own sensation of cold.
- Otherwise, leave the product at room temperature until it has reached the desired temperature and reduce the duration of pre-cooling before the next use.
- If the product has been stored in the refrigerator or freezer for longer than the recommended time, leave it at room temperature for at least 30 minutes and check that the temperature is appropriate before use.
- If the item is damaged, it must not be used.
- Pull the product over the affected area of the body.
- Application time: approx. 20 minutes.
- Remove the product after application and wait at least 20 minutes before reapplying to the same area.
- Check the skin regularly for signs of irritation during use.

In case of excessive cold sensation, redness or blistering, discontinue use immediately.

- Store the item in the resealable packaging after each use

## Intended purpose for heat:

Warming tubular bandage for external use.  
Can help alleviate pain.

## Indications

### For supporting use in:

- Sore muscles
- Tense muscles

## Heat treatment:

### Use in the microwave:

- The article must be placed in the middle of the microwave and heated at 600 W for approx. 30 seconds. Please wait 20 seconds before opening the microwave, as delayed heating is possible
- Carefully check the temperature with the palm of your hand. If it is too warm, let it cool down. If the product is not warm enough, repeat the warming process for another 10 seconds.
- The product must not be warmed above 42 °C, otherwise there is a risk of burns and damage to the product!
- Otherwise, leave the product at room temperature until it has reached the desired temperature and reduce the duration of heating before the next use.
- If the item is damaged, it must not be used.
- Pull the product over the affected area of the body.
- Application time: approx. 20–40 minutes
- Remove the product after application and wait at least 20 minutes before reapplying to the same area.

- Check the skin regularly for signs of irritation during use.

 In case of excessive heat sensation, redness or blistering, discontinue use immediately.

- Store the item in the resealable packaging after each use

## Contraindications

 **The article must not be used, if you:**

- suffer from circulatory disorders
- have a limited sense of temperature
- take painkillers
- have open wounds or damaged skin areas

No cold application for osteoarthritis.

No heat application in the case of inflammation or increased temperature.

## Warnings

 Keep out of the reach of children and use only under adult supervision.

 Do not exceed the specified wearing times.

 Please check before each use that the product is not damaged. Otherwise, please dispose of with your household waste.

 Select the correct item size according to circumference so that the product is not too tight, otherwise your circulation could be impaired.

 Before using the product, clean the skin thoroughly. The skin must be free of gels, greases, ointments, oils or similar substances.

 For external use only.

 The product must not be used while sleeping.

 Do not exert any external pressure on the product during use!

 Keep away from open flames.

If problems or complaints should occur, consult a doctor immediately!

## Cold application

Only mentally healthy people with full sensory capabilities are allowed to use the product.

## Heat application

Only mentally healthy people with full sensory capabilities are allowed to take over the warming process of the product. The product must not be heated in the oven.

## Care

The tubular bandage should not be cleaned under running water! The bandage may be wiped with a damp cloth and should then be allowed to dry.

## Storage

 Keep dry / protect from moisture

 Protect from sunlight

The resealable packaging is required for storing the item. Do not dispose of packaging! Always store the article in the closed packaging!

## Instructions for disposal

Please dispose of the product in an environmentally-friendly way through an approved disposal centre or your local waste facility. Observe currently applicable regulations.

## Obligation to notify

All serious incidents relating to the product must be reported immediately to the manufacturer and the competent authority.

## Legal warranty

Your warranty rights are governed by local legal provisions. Typically, you will receive a legal warranty of 24 months from the date of purchase for this item. Please keep the original receipt as proof of purchase. If a product defect is detected within 24 months of the purchase date for an unused item and reported to the retailer, the defective item will be replaced or the purchase price refunded under the legal warranty. The warranty covers defects that the product had at the time of purchase and requires that the defective item can be presented to the manufacturer.

Service Germany

Tel.: +49 (0)40 82 22 44-80

E-Mail: support@horizonte.de

Consult instructions for use

Attention: Always adhere to the stated warning information

Keep dry / protect from moisture

Protect from sunlight

Catalogue number

Batch designation

Use by

Date of manufacture (year): 2025

Manufacturer

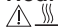
Indicates the authorised representative in Switzerland (for products purchased in Switzerland)

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Lidl Schweiz DL AG