## GEL FOOT CUSHIONS FUSSSOHLENKISSEN COUSSINETS PLANTAIRE

(GB) (IE) (NI) (MT)

### GEL FOOT CUSHIONS

Instructions for use

(3-5)

(DE) (AT) (BE) (CH)

### FUSSSOHLENKISSEN

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(6-8)

(FR) (BE) (CH)

### COUSSINETS PLANTAIRE

Mode d'emploi

(8-12)

(NL) (BE)

### VOETZOOKUSSEN

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(12-15)

(IT) (CH) (MT)

### CUSCINETTO PLANTARE

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(15-17)

(ES)

### COJÍN PLANTAR

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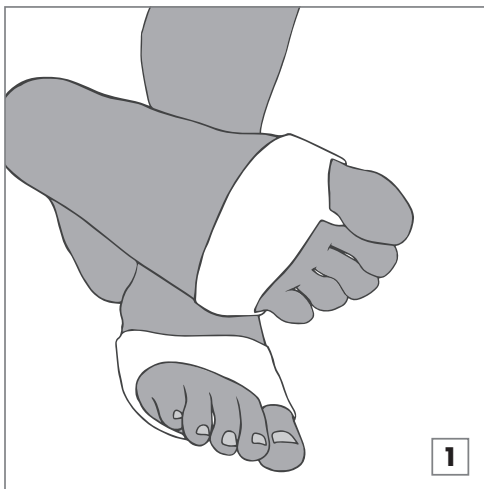
(18-20)

(PT)

### ALMOFADA PARA A PLANTA DO PÉ

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(20-22)

GB IE NI MT

Congratulations on the purchase of your new product. You have chosen a high-quality article. The instructions for use are part of this product. They contain important information on safety and use. Please familiarise yourself with the instructions and safety information before using this product. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the product and be sure to include them when handing on the product to another person.

## Medical device

### Only intended for private use!

## Note

The medical device is intended for adults who are able to read, understand and independently follow these instructions for use. A reason for use, as named under Indications, must be present at the user.

The item is not intended for groups of persons who are excluded from its application by the sizes and contraindications listed below.

## Scope of delivery

- 2 x gel foot cushions (1 pair)
- 1 x laundry bag
- 1 x instructions for use

## Technical Data

Size	Dimensions
S/M	6 x 9,5 cm
L/XL	6,5 x 10,5 cm

## Intended purpose

Reduces the pain caused by pressure on the frontal ball of the foot and the metatarsal head

## Indications

To be used as support for:

- Calluses and warts on the frontal ball of the foot
- Pressure applied to the metatarsal head by shoes
- Splay feet

## Contraindications

**Do not use – or use only after consultation with your doctor – for:**

- External injuries
- Circulatory disorders
- Eczema or other skin diseases

## Safety instructions

### Risk of injury

- ⚠ If problems occur while wearing the bandage, remove it immediately and consult a doctor.
- ⚠ Check the bandage before each use for tears and damage.
- ⚠ The product should only be used in perfect condition.
- ⚠ Never put the bandage on so tightly that the blood circulation is restricted.
- ⚠ Do not wear the bandage longer than 2 to 3 hours at a stretch and a maximum of 6 to 8 hours a day.
- ⚠ Do not wear the bandage while sleeping or during longer rest phases such as long periods of sitting.
- ⚠ Wash the areas of skin covered by the bandage thoroughly and regularly.
- ⚠ To avoid allergic skin reaction, the skin must be free of grease, gels, creams, oils or similar materials.
- ⚠ The article may only be used by other people when it has been cleaned.

### Possible side effects

- ⚠ If you experience symptoms (such as, e.g. tingling, pain, swelling) while wearing the bandage, remove the bandage immediately!
- ⚠ There is a risk of skin irritation and skin irritations! In individual cases, skin irritation and reactions and allergic reactions may occur. Make sure that the stated maximum wearing periods are not exceeded and the areas of skin covered by the bandage are thoroughly and regularly washed. Should any problems or complaints occur, consult a doctor immediately!

### Positioning of the product

- Pull the footbed cushion over the foot as shown in the figure (**Fig. 1**).
- Make sure that the product is firmly in place and feels comfortable but not too tight.
- Make sure that the blood flow in the front of the foot and toes is not impaired.

### Cleaning and care

- You should launder the product thoroughly prior to its first use.
- After using it two or three times, it should be washed again.
- Flush and drain the product well after it has been laundered.
- Wash in the laundry bag.
- Wash separately.

### Instructions for disposal

Please dispose of the product in an environmentally-friendly way through an approved disposal centre or your local waste facility. Observe currently applicable regulations.

## Obligation to notify

All serious incidents relating to the product must be reported immediately to the manufacturer and the competent authority.

## 3-year warranty

The product has been created with great care and under constant monitoring. You will receive a 3-year warranty for this product from the date of purchase. Please keep the receipt. The warranty is valid only for material and manufacturing faults and lapses in the event of misuse or improper handling. Your statutory rights, particularly the warranty rights, are not restricted by this warranty. If you have any complaints, please contact the service hotline below or contact us by email. Our service staff will discuss how to proceed further with you as soon as possible. In any case, we will personally advise you. The warranty period is not extended by any repairs under the terms of the warranty, statutory warranty or goodwill. This also applies to replaced and repaired parts. After expiry of the warranty, incurred repairs are chargeable.

Service Germany

Tel.: +49 (0)40 82 22 44-80

E-Mail: support@horizonte.de

Consult instructions for use

Attention: Always adhere to the stated warning information

Catalogue number

Batch designation

Use by

Date of manufacture (year): 2025

Manufacturer

Indicates the authorised representative in Switzerland (for products purchased in Switzerland)

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Lidl Schweiz DL AG