SILVERCREST®

OBERARM-BLUTDRUCKMESSGERÄT / UPPER ARM BLOOD PRESSURE MONITOR / TENSIO MÈTRE DE BRAS

DE AT CH

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www.sanitas-online.de/web/de/landingpages/cedeclarationofconformity.php

List of pictograms used	Page 40
Introduction	Page 41
Intended use	Page 42
Improper use	Page 42
Parts description	Page 43
Scope of delivery	Page 44
Technical data	Page 45
Important safety notes	Page 47
Battery safety information	Page 50
Initial use	Page 52
Inserting / Replacing batteries	Page 52
App / web version of "HealthForYou"	Page 53
Configuring the date / time / <i>Bluetooth</i> [®]	Page 53
Measuring blood pressure	Page 54
Attaching the cuff	Page 55
Adopting the correct posture	Page 56
Selecting a memory	Page 56
Carrying out the blood pressure measurement	Page 57
Transferring measurements via <i>Bluetooth</i>[®]	Page 58
Evaluating measurements	Page 60
Risk indicator	Page 60
Saving, requesting and deleting measurements	Page 62
Cleaning and care	Page 63
Storage	Page 64
Troubleshooting	Page 65
Disposal	Page 66
Warranty / Service	Page 67

List of pictograms used

 WARNING!	Warning of risks of injury or health hazards
	Safety note related to possible damage to the device / components
	Safety notes
	Instructions
	Observe instructions for use
	Direct current
Storage / Transport 	Permissible storage and transport temperature and humidity
Operating 	Permissible operating temperature and humidity
	Permissible storage, transport and operating air pressure
	Manufacturer
	Authorized EU representative for manufacturers of medical devices
	Applied part, type BF
	Serial number

 0123	The CE mark certifies compliance with the basic requirements of Medical Devices Directive 93/42/EEC.
	Improper disposal of batteries can harm the environment!
IP21	Device protected against foreign matter ≥ 12.5 mm and against water falling vertically
	Dispose of the packaging and the blood pressure monitor in an environmentally friendly manner!
	Data transfer via <i>Bluetooth</i> [®] wireless technology

Upper Arm Blood Pressure Monitor

● Introduction

Please make yourself familiar with the upper arm blood pressure monitor before preparing it for use or using it for the first time. Please carefully read the following instructions for use and the important safety notes. Failure to observe the notes may result in personal injury or property damage. The product must only be used as described in these instructions for use and for the specified fields of application. Keep these instructions for

use in a safe place. If you pass the upper arm blood pressure monitor on to anyone else, please ensure that you also pass on all the documentation.

● Intended use

This upper-arm blood pressure monitor is intended to measure and monitor the arterial blood pressure values and the heart rates of adults with an arm circumference of 22 to 42 cm, in a non-invasive way.

Using this device, you can quickly and easily measure your blood pressure, save the measurements and display the progression and average of these measurements. You are warned in cases of pre-existing cardiac arrhythmia. The values determined are classified and evaluated graphically. Any use other than previously mentioned or any product modification is prohibited and may result in injuries and/or product damage. The manufacturer assumes no liability for damage arising out of improper use. The product is only intended for use in private environments (at home).

● Improper use

- ⚠ WARNING!** Do not use this product on newborns and preeclampsia patients. We recommend consulting your doctor before using the product if you are pregnant.
- The device is not suitable for use by patients with implanted electrical devices such as pacemakers or defibrillators.

- The measurements determined by you are only intended for your information - they do not replace a medical examination. Discuss the measurements with your doctor. Do not self-diagnose or self-treat based on the measurement results without consulting your attending physician. Do not start new medication without consulting a doctor and do not modify the type and/or dosage of current medications.
- The product may not be used in conjunction with a high-frequency surgical unit.
- Using the blood pressure monitor outside the domestic environment or under the influence of movement (e.g. whilst riding in cars, ambulances or helicopters as well as whilst practising physical activities such as sport) can impact the accuracy of the measurement and lead to measurement errors.

● Parts description

- 1 Cuff
- 2 Cuff tubing
- 3 Cuff plug
- 4 Display
- 5 Memory button M
- 6 Settings button SET
- 7 Function button -/+
- 8 START/STOP button ⓘ
- 9 Risk indicator

- 10 Connection for cuff plug
- 11 Battery compartment

Display indicators

- 12 Time and date
- 13 Systolic pressure
- 14 Diastolic pressure
- 15 Pulse value determined
- 16 Symbol for cardiac arrhythmia  Symbol for pulse 
- 17 Symbol for replace battery 
- 18 Memory display: Average value (A), morning (AM), evening (PM), number of the memory space
- 19 Risk indicator
- 20 Release air
- 21 User memory
- 22 Symbol for *Bluetooth*[®] transfer

● Scope of delivery

- 1 Upper arm blood pressure monitor
- 1 Upper arm cuff (Material no. 164.136) with cuff tubing
- 4 Batteries 1.5 V type AAA LR03
- 1 Instructions for use
- 1 Quick start guide

● Technical data

Type:	SBM 69
Measuring method:	Oscillometric, non-invasive blood pressure measurement on the upper arm
Measurement range:	Cuff pressure 0–299 mmHg, systolic 60–230 mmHg, diastolic 40–130 mmHg, pulse 40–199 beats per minute
Display accuracy:	systolic ± 3 mmHg, diastolic ± 3 mmHg, pulse $\pm 5\%$ of the displayed value
Measurement uncertainty:	max. permissible standard deviation according to clinical testing: systolic 8 mmHg / diastolic 8 mmHg
Memory:	4 x 30 memory spaces
Dimensions:	L 138,2 mm x W 100,2 mm x H 51,0 mm
Cuff:	suitable for an arm circumference between 22 and 42 cm
Weight:	approx. 395 g (with cuff, without batteries)
Permissible operating conditions:	+5 °C to +40 °C, 15 % to 90 % relative humidity (non-condensing), 700-1060 hPa ambient pressure
Permissible storage	

conditions:	-20 °C to +60 °C, ≤ 93 % relative humidity
Power supply:	4 x 1.5V AAA batteries
Product expected lifespan:	5 years
Battery life:	Approx. 225 measurements, depending on blood pressure level and inflation pressure / <i>Bluetooth</i> ® transfer instructions for use, 4 x 1.5V AAA LR03 batteries, storage pouch
Components:	Internal supply, IP21, no AP or APG, continuous operation, applied part Type BF
Classification:	
Data transfer via <i>Bluetooth</i> ® wireless technology:	The product uses <i>Bluetooth</i> ® low energy technology, frequency band 2.402–2.480 GHz, maximum transmission power radiated in the frequency band < 4 dBm, compatible with <i>Bluetooth</i> ® ≥ 4.0 smart phones / tablets

For a list of compatible smart phones, information about the “HealthForYou” app and software, as well as further details about the devices, please visit: www.healthforyou.lidl

System requirements

of the „HealthForYou“

web version:

- MS Internet Explorer
 - MS Edge
 - Mozilla Firefox
 - Google Chrome
 - Apple Safari
- (in the respective updated version)

System requirements
of the “HealthForYou“

app

iOS from Version ≥ 10.0 , Android™
from Version ≥ 5.0 , Bluetooth® ≥ 4.0 .

Technical specifications subject to change without notice
due to updates.

Important safety notes

⚠ WARNING! This device is not intended to be used by persons with limited physical, sensory or mental abilities or lacking experience and/or knowledge unless supervised or instructed on its use by an person responsible for their safety. Children should be supervised to ensure they do not play with the device.

- Cardiovascular system diseases may result in incorrect measurements and/or impairments to the device's measurement accuracy. This is also the case when it comes to low blood pressure, diabetes, circulatory problems and dysrhythmia, as well as chills and shaking.

- Only use the product on individuals who have the upper arm range stated for the product.
 - Ensure that the limb is not functionally impaired during inflation.
 - Blood circulation should not be restricted by the blood pressure measurement for an unnecessary amount of time. Remove the cuff from the arm if the device malfunctions.
 - Avoid mechanically restricting, compressing or kinking the cuff tubing.
 - Avoid sustained pressure to the cuff, as well as frequent measurements. The resulting curtailing of the blood flow can lead to injuries.
 - Ensure that the cuff is not placed on an arm whose arteries or veins are undergoing medical treatment, i.e. intravascular access or intravascular therapy or an arteriovenous (A-V) shunt.
 - Do not place the cuff on individuals who have had a mastectomy.
 - Do not place the cuff over wounds as this could lead to further injuries.
 - Only place the cuff on the upper arm. Do not place the cuff on other body parts.
- Attention:** The device may only be used in conjunction with the original cuff provided.
- Do not use rechargeable batteries.
 - If no button is pressed, the automatic shutoff will switch off the blood pressure monitor within 1 minute to save the batteries.

⚠ Attention! Potential damage to the device!
The blood pressure monitor is made up of precision components and electronic components. The accuracy of the measurements and the service life of the device depend on the device being used with a due amount of care:

- Protect the device from impacts, moisture, dirt, severe temperature fluctuations and direct sunlight.
- Do not drop the device.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.
- Do not use the device in the vicinity of strong electromagnetic fields, keep it away from radio equipment and mobile telephones.

- Only use the cuff provided or an original replacement cuff. Otherwise, measurements could be miscalculated.
- Remove the batteries from the device if it is not used over a longer period of time.
- Routinely inspect the device to ensure the device is not visibly damaged before use and the battery level is sufficient. When in doubt, do not use and contact service or an authorised retailer. Do not use additional parts not recommended or sold as components by the manufacturer.
- Never open or repair the device or proper function cannot be guaranteed. Non-compliance will void the warranty.
- Please contact service or an authorised retailer for repairs. However, before making a complaint, firstly check the batteries and replace them if necessary.

Battery safety information

- **DANGER TO LIFE!** Keep batteries out of the reach of children. Seek immediate medical attention if swallowed!
-  **EXPLOSION HAZARD!** Never recharge disposable batteries. Never short-circuit and/or open batteries. The battery may overheat, ignite or explode.
- Never throw batteries into fire or water.
- Do not exert mechanical stress on batteries.

Risk of battery leakage

- Avoid extreme conditions and temperatures which may affect batteries, e.g. radiators / direct sunlight.
- If batteries have leaked, avoid contact with skin, eyes and mucous membranes with the chemicals! Flush immediately the affected areas with fresh water and seek medical attention!
-  **WEAR PROTECTIVE GLOVES!** Leaked or damaged batteries can cause burns on contact with the skin. Therefore wear suitable safety gloves in this case.
- In the event of a battery leak, immediately remove the batteries from the product to prevent damage.
- Only use the same type of batteries. Do not mix old and new batteries!
- Remove the batteries if the product will not be used for an extended period.

Risk of product damage

- Only use the specified battery type!
- Insert the batteries according to the polarity marking (+) and (-) on the battery and the product.
- Clean the contacts on the battery and inside the battery compartment with a dry cloth before inserting the battery!
- Immediately remove drained batteries from the product.
- Do not use rechargeable batteries.

● Initial use

- Verify the parts listed in the scope of delivery are complete.
- Remove all packaging material.

● Inserting / Replacing batteries

- Slide the battery cover off the battery compartment 11 (fig. C).
- Insert the four type 1.5 V AAA (Alkaline type LR03) batteries (with the correct polarity) into the device (fig. C). Please note the marking inside the battery compartment.
- Carefully close the battery cover on the battery compartment 11.
- Do not use rechargeable batteries.

Note: if the change battery indicator 17 is displayed continuously, measurement will not be possible until all batteries have been replaced. As soon as the batteries have been removed from the device, the date and time will be reset. Saved measurements will still be stored.

- All display elements will be displayed briefly and "24h" will flash on the display. Configure the date, time and Bluetooth® (see "Configuring date/time/Bluetooth®").

● App / web version of "HealthForYou"

Download the free app "HealthForYou" from the Apple App Store and Google Play.

- Follow the installation instructions on the screen.
- Alternatively, you can also visit the website www.healthforyou.lidl. Here you will be able to enter measurements and find guidance for assessment.

● Configuring the date / time / Bluetooth®

- Configure the date, hour format and time. It is only in this way that you will be able to correctly save these measurements with the date and time and request them later on.

① The settings menu can be accessed in two different ways:

Before first use and after every battery change:

- When you insert batteries into the device, you will automatically arrive at the relevant menu.

If batteries have already been inserted:

- On the **switched-off** device, hold the settings button SET  pressed for approx. 5 seconds.

Now proceed as follows:

- The time format will flash. Use the function buttons $-/+$ **7** to set the format to 12-hour or 24-hour. Press the SET button **6** to confirm. The year will flash.
- Use the function buttons $-/+$ **7** to set the year. Press the SET button **6** to confirm. The month will flash.
- Use the function buttons $-/+$ **7** to set the month. Press the SET button **6** to confirm.
- Configure the day, hour and minute consecutively as described above. Press the SET **6** button to confirm each entry.
The *Bluetooth*[®] symbol will be shown on the display. "On" and "Off" will flash at the same time.
- Using the function buttons $-/+$, choose whether you want to activate ("On" flashes) or deactivate ("Off" flashes) *Bluetooth*[®] data transfer. Press the SET button **6** to confirm.
If *Bluetooth*[®] is activated ("On") data transfer will begin automatically after the measurement has been taken.
Note: The battery life is reduced by transfer via *Bluetooth*[®].

● Measuring blood pressure

Note: To guarantee that you will be able to compare values adequately, always measure your blood pressure at the same time of day.

- You should not eat, drink, smoke or be physically active in the 30 minutes before each measurement.
- Before taking the first blood pressure measurement, always rest for 5 minutes!
- Furthermore, if you wish to take multiple measurements one after the other, always wait at least 1 minute between each measurement.
- Repeat the measurement process if the measured values seem incorrect.
- Bring the device to room temperature before the measurement.

● Attaching the cuff

⚠ ATTENTION! The device may only be used in conjunction with the original cuff provided. The cuff is suitable for an upper arm circumference of between 22 and 42 cm.

- Place the cuff **1** on a bare upper arm (fig. D). The blood flow of the arm should not be constricted by clothing etc. which is too tight.
- Position the cuff **1** on the upper arm so the lower edge is 2–3 cm up from the elbow and over the artery. The cuff tubing **2** points towards the middle of the palm.
- Place the free end of the cuff **1** closely, but not too tightly, around the arm and close the Velcro strip. The cuff **1** should not be too tight (fig. E). Check that two fingers can still fit beneath the cuff **1** (fig. F).

- This cuff is suitable for you if, after putting it on, the index mark on the outside of the cuff (▼) is within the OK range on the cuff (Fig. G).
- Now insert the cuff tubing [2] with the cuff plug [3] into the connector (fig. H).

● Adopting the correct posture

Note: Stay still during the measurement process and do not talk to prevent the measurement result from being falsified.

- You can carry out the measurement while sitting or lying down. Always ensure that the cuff [1] is at the same level as the heart (fig. I-K).
- Sit comfortably for the blood pressure measurement. Recline your back and arms. Do not cross your legs. Put your feet flat on the ground.
- In order not to falsify the measurement result, it is important that you stay still and do not talk.

● Selecting a memory

- With the device off, press the SET button [6]. Select the desired user memory M_1 ... M_4 [21] by pressing the function buttons -/+ [7].
You have 4 memories with 30 memory spaces for saving the measurement results of 4 different people separately.

- Confirm your choice using the START / STOP button **① 8** or wait 3 seconds. Your choice will then be saved automatically.

● Carrying out the blood pressure measurement

- Place the cuff on your arm as previously described and assume the position in which you wish to carry out the measurement.
- Switch on the blood pressure monitor using the START / STOP button **① 8**.

After the full-screen display, the last measurement result saved will be displayed. If no measurement has been saved, the device will display the value of "0".

After 5 seconds, the blood pressure monitor will begin with the measurement by pumping up the cuff. The measurement is taken while the cuff is being pumped up. As soon as a pulse has been detected, the pulse symbol  will be displayed. After this, the cuff air pressure will be released again.

The measurement results for systolic pressure, diastolic pressure and pulse will be displayed.

- You can stop the measurement at any time by pressing the START / STOP button **① 8**.

If the measurement has not been carried out correctly, "Er_" will be displayed (see "Troubleshooting" also). Repeat the measurement process in the event that this happens.

The measurement result will be saved automatically.

If you have activated *Bluetooth*[®] data transfer, the data will be transferred. The blood pressure monitor displays the symbol for *Bluetooth*[®] transfer during transfer. Switch the blood pressure monitor off by pressing the START / STOP button once more.

- To power down press the START / STOP button ① [8].
Note: The device switches off automatically after one minute if no button has been pressed. Wait for at least one minute before taking another measurement!

● Transferring measurements via *Bluetooth*[®]

You have the option of transferring the measured values and values saved on your device to your smartphone using *Bluetooth*[®] low energy technology.

- Activate *Bluetooth*[®] in your smartphone settings and open the “HealthForYou” app.
Note: The “HealthForYou” app must be open for transfer to occur.
- Add “SBM69” in the app.
- When connecting for the first time, a randomly generated six-digit PIN code will be displayed on the device over two lines. At the same time, an entry field will appear on your smartphone where you should enter this six digit code. When you have entered it correctly, the device is connected with your smartphone and you can begin to transfer your measurement values.

Warning! The displayed values are intended merely as a visualisation and may not be used as the basis for therapeutic measures. The software is not part of a diagnostic medical system.

We wish to expressly indicate that the software at hand is not a medical product in accordance with the EU directive 93/42/EEC.

Note: During data transfer, button functions cannot be executed.

To transfer measurements to your smartphone via *Bluetooth*[®] proceed as follows:

Manual transfer

Go to save mode (see "Saving, requesting and deleting measurements"). As soon as the average value of all measurements is displayed, a *Bluetooth*[®] connection will automatically be established. The data will be transferred and the *Bluetooth*[®] symbol will be shown on the display.

- Switch the blood pressure monitor off by pressing the START / STOP button  .

Automatic transfer

If you forget to switch off the blood pressure monitor, it will automatically switch off after one minute. In this case, the value will be saved in the selected user memory and the data will be transferred if *Bluetooth*[®] data transfer has been activated. The *Bluetooth*[®] symbol will be shown on the display during transfer.

● Evaluating measurements

Cardiac arrhythmia

This device can identify any potential cardiac arrhythmia disorders and, if necessary, indicates this after the measurement process with the symbol . This can be an indicator for arrhythmia.

Arrhythmia is an illness in which the heart rhythm is abnormal due to errors in the bioelectric system, which controls the heartbeat. The symptoms (skipped or premature heartbeats, slow or rapid pulse) can stem from heart conditions, age, physical predispositions, an excess of luxury foods, stress or a lack of sleep. Arrhythmia can only be diagnosed by a doctor after a physical examination.

- Repeat the measurement process if the symbol  is shown on the display after the measurement. Please make sure that you rest for a minute before the measurement and do not talk or move during it.

 **WARNING!** If the symbol  is displayed repeatedly, contact your doctor. Self-diagnosis and self-treatment on the basis of measurement results can be dangerous. Follow your doctor's instructions.

● Risk indicator

In accordance with the guidelines / definitions provided by the World Health Organisation (WHO) and the latest findings, the measurement results can be classified and evaluated according to the following table:

Range of blood pressure values	Systole (in mmHg)	Diastole (in mmHg)	Remedies
Stage 3: severe hypertension	≥ 180	≥ 110	seek medical attention
Stage 2: moderate hypertension	160–179	100–109	seek medical attention
Stage 1: mild hypertension	140–159	90–99	regular monitoring by a doctor
High, normal	130–139	85–89	regular monitoring by a doctor
Normal	120–129	80–84	Self-monitor
Ideal	< 120	< 80	Self-monitor

Source: WHO, 1999

Note: These standard values merely serve as a general guideline as blood pressure deviates depending on the individual, different age groups, etc.

It is important that you consult your doctor at regular intervals. Your doctor will give you your particular values for normal blood pressure as well as the value for when your blood pressure is classified as dangerous.

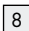
The bar chart on the display and the scale on the device state which range the blood pressure measured is in.

If the systole and diastole values are in two different ranges (i.e. systole is “high, normal” and diastole is “normal”) the graphic categorisation on the device will always show the higher range i.e. in the above example this would be “high, normal”.

● Saving, requesting and deleting measurements



The results of each successful measurement are saved together with the date and time. If there are over 30 measurement values, the oldest measurement value will be overwritten.

- With the device off, press the SET button [6]. Press the function buttons -/+ [7] to select the desired user memory $\text{M}_1 \dots \text{M}_4$. Confirm your choice using the START / STOP button ① [8].
- Press the memory button M [5] to display the average value “A” of all saved measurements pertaining to the user memory. Press the memory button M [5] again to display the average value of the last 7 days for the morning measurement. (Morning: 5.00 am – 9.00 am, display “AM”). Press the memory button M [5] again to display the average value of the last 7 days for the evening measurement. (Evening: 6 pm – 8 pm, display “PM”). By pressing the memory button M again, the last individual measurement will be displayed with the date and time.

- Press the START / STOP button   to switch off the device.

Note: The device switches off automatically after one minute when no button has been pressed.


Deleting data

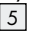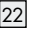- To delete the memory of the selected user, press the memory button M  when the device is switched off.
- The average value of all measurements will appear on the display. Next to this, "A" and the Bluetooth® transfer symbol  will flash.
- Keep the function buttons -/+  pressed simultaneously for 5 seconds.
- After successful deletion, "CL 00" will appear in the display.

The data has been deleted.

● Cleaning and care

 **ATTENTION!** Possible property damage! Never hold the product under water as liquid could enter the product and damage it.

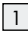- Clean the device and cuff  carefully using a slightly damp cloth.
- Do not use cleaners or solvents.

● Storage

- ⚠ ATTENTION!** Possible property damage! Do not place any heavy objects on the device and cuff **1**.
- If the product will not be used for more than one month, remove both batteries from the device to prevent possible battery leakage.
 - Do not kink the cuff tubing **2**.
 - Store the product in a dry location (relative humidity $\leq 93\%$). High humidity can shorten the service life or damage the product. Store the product in a location with ambient temperatures between $-20\text{ }^{\circ}\text{C}$ and $+60\text{ }^{\circ}\text{C}$.

● Troubleshooting

Trouble-shooting	Problem	Solution
Er1	The pulse cannot be measured correctly.	Repeat the measurement process. Be sure not to move or talk.
Er2	You have been talking or moving during the measurement process.	
Er3	The cuff was attached too tightly or too loosely.	
Er4	An error occurred during the measurement process.	
Er5	The pump-up pressure is higher than 300 mmHg or the blood pressure measurement falls outside of the measuring range.	
Er6 	The batteries are almost dead.	Reinsert the batteries or replace them with new batteries.

Trouble-shooting	Problem	Solution
Er7	Data cannot be sent via Bluetooth®.	Activate Bluetooth® on your smartphone and start the "HealthForYou" app.
Er8	A device error has occurred.	Repeat the measurement process. Be sure not to move or talk.

● Disposal

The packaging is made entirely of recyclable materials, which you may dispose of at local recycling facilities.

Observe the marking of the packaging materials for waste separation, which are marked with abbreviations (a) and numbers (b) with following meaning: 1-7: plastics / 20-22: paper and fibreboard / 80-98: composite materials.

Contact your local refuse disposal authority for more details of how to dispose of your worn-out product.

To help protect the environment, please dispose of the product properly when it has reached the end of its useful life and not in the household waste. Information on collection points and their opening hours can be obtained from your local authority.

Per Directive 2006 / 66 / EC and its amendments, defective or used batteries must be recycled. Return the batteries and/or the product to a collection site.

Improper disposal of batteries can harm the environment!

Never dispose of batteries in your household bin. They may contain toxic heavy metals and are subject to hazardous waste regulations. The chemical symbols of the heavy metals are: Cd = cadmium, Hg = mercury, Pb = lead. Therefore dispose of used batteries through your local collection facilities.

● Warranty / Service

We provide a 3-year warranty covering faults in materials or manufacturing of the blood pressure monitor from the purchase date.

The warranty does not cover:

- Damage due to improper use.
- Wear parts.
- Damage caused by the customer.
- If the device has been opened by an unauthorised repair shop.

This warranty does not affect the customer's legal rights. The customer must provide proof of purchase in order for any warranty claims within the warranty period to be honoured. The warranty claim must be filed with Hans Dinslage

GmbH, Uttenweiler, Germany, within 3 years from the date of purchase.

In the event of a complaint, please contact our Service Department listed below:

Service Hotline (free of charge):

GB

☎ **0800 93 10 319 (Mon – Fri: 8 am – 6 pm)**

✉ **service-uk@sanitas-online.de**

IE

☎ **1800 93 92 35 (Mon – Fri: 8 am – 6 pm)**

✉ **service-ie@sanitas-online.de**

If we ask you to return the defective blood pressure monitor, please send it to the following address:

send to:

NU Service GmbH

Lessingstraße 10b

89231 Neu-Ulm

GERMANY

IAN 322244

This device complies with European Standard EN 60601-1-2 (in accordance with CISPR 11, IEC 61000-4-2, IEC 61000-4-3, IEC 61000-4-8) and is subject to special precautions with

respect to electromagnetic compatibility. The device is suitable for use in all environments listed in these instructions for use, including domestic environments. Please note, portable and mobile HF communication devices may interfere with this device. This device complies with EU Medical Devices Directive 93/42/EEC, the Medical Devices Act and EN 1060-3 standards (non-invasive blood pressure monitors Part 3: Supplementary requirements for electro-mechanical blood pressure measuring systems) and standard IEC 80601-2-30 (Medical electrical equipment Part 2-30: Particular requirements for basic safety and essential performance of automated non-invasive blood pressure monitors).

The accuracy of this blood pressure monitor has been thoroughly tested and was developed with regard to a long utilizable service life. If the device is used in medicine, technical inspections using the suitable equipment are to be carried out. Precise information on how to test the accuracy of the device can be requested from our service address.

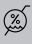
We hereby affirm that this product complies with the European RED Directive 2014/53/EU.

The CE declaration of conformity for this product is available at:

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www.sanitas-online.de/web/de/landingpages/cedeclarationofconformity.php

The *Bluetooth*[®] word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Hans Dinslage GmbH is under license. Other trademarks and trade names are those of their respective owners.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries.

Google Play and the Google Play logo are trademarks of Google LLC. Android is a trademark of Google LLC.

Notes on electromagnetic compatibility

The ME EQUIPMENT or ME SYSTEM is suitable for home healthcare environments and so on. Warning: Don't near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.

Warning: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Warning: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the equipment (SBM69), including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

If any: a list of all cables and maximum lengths of cables (if applicable), transducers and other ACCESSORIES that are replaceable by the RESPONSIBLE ORGANIZATION and that are likely to affect compliance of the ME EQUIPMENT

or ME SYSTEM with the requirements of Clause 7 (EMIS- SIONS) and Clause 8 (IMMUNITY). ACCESSORIES may be specified either generically (e.g. shielded cable, load impedance) or specifically (e.g. by MANUFACTURER and EQUIPMENT OR TYPE REFERENCE). If any: the performance of the ME EQUIPMENT or ME SYSTEM that was determined to be ESSENTIAL PERFORMANCE and a description of what the OPERATOR can expect if the ESSENTIAL PERFOR- MANCE is lost or degraded due to EM DISTURBANCES (the defined term “ESSENTIAL PERFORMANCE” need not be used).

Table 1:
Guidance and manufacturer’s declaration –
electromagnetic emissions:

Emissions test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR 11	Group B
Harmonic emissions IEC 61000-3-2	Not application
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not application

Table 2:
Guidance and manufacturer's declaration –
electromagnetic immunity:

Immunity test	IEC 60601-2 test level	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air
Electrical fast transient/burst IEC 61000-4-4	power supply lines: ±2 kV input/output lines: ±1 kV	Not application
Surge IEC 61000-4-5	line(s) to line(s): ±1 kV line(s) to earth: ±2 kV 100 kHz repetition frequency	Not application
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0% U_T ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0% U_T ; 1 cycle and 70% U_T ; 25/30 cycles Single phase: at 0° 0% U_T ; 300 cycle	Not application
Power frequency (50Hz/60Hz) magnetic field IEC 61000-4-8	30 A/m 50 Hz/60 Hz	30 A/m 50 Hz/60 Hz

Immunity test	IEC 60601-2 test level	Compliance level
Conducted RF IEC 61000-4-6	150 KHz to 80 MHz: 3 Vrms 6 Vrms (in ISM and amateur radio bands) 80% Am at 1 kHz	Not application
Radiated RF IEC 61000-4-3	10V/m 80 MHz-2,7 GHz 80% AM at 1 kHz	10V/m 80 MHz-2,7 GHz 80% AM at 1 kHz
NOTE U_T is the a.c. mains voltage prior to application of the test level.		

Table 3:
Guidance and manufacturer's declaration -
electromagnetic immunity:

Radiated RF IEC 61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	Test Frequency	Band (MHz)	Service
	385	380-390	TETRA 400
	450	430-470	GMRS 460: FRS 460
	710	704-787	LTE Band 13, 17
	745		
	780		
	810	800-960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5
	870		
	930		

Radiated RF IEC 61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	Test Frequency	Band (MHz)	Service
	1720	1700-1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS
	1845		
	1970		
	2450	2400-2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7
	5240	5100-5800	WLAN 802.11 a/n
	5500		
	5785		

Table 3:
Guidance and manufacturer's declaration -
electromagnetic immunity:

Radiated RF IEC 61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equip- ment)	Modu- lation	Modu- lation (W)	Dis- tance (m)	IMMU- NITY TEST LEVEL (V/m)
	Pulse modula- tion b) 18Hz	1.8	0.3	27
	FM c) \pm 5kHz de- viation 1 kHz sine	2	0.3	28
	Pulse modula- tion b) 217Hz	0.2	0.3	9
	Pulse modula- tion b) 18Hz	2	0.3	28

Radiated RF IEC 61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equip- ment)	Modu- lation	Modu- lation (W)	Dis- tance (m)	IMMU- NITY TEST LEVEL (V/m)
	Pulse modula- tion b) 217Hz	2	0.3	28
	Pulse modula- tion b) 217Hz	2	0.3	28
	Pulse modula- tion b) 217Hz	0.2	0.3	9

Guangdong Transtek Medical Electronics Co., Ltd.

Zone A, No.105, Dongli Road, Torch Development District,
Zhongshan, 528437, Guangdong, CHINA

MDSS - Medical Device Safety Service GmbH,

Schiffgraben 41, 30175 Hannover, GERMANY

Distributed by **Hans Dinslage GmbH,**
Riedlinger Straße 28, 88524 Uttenweiler, GERMANY

Stand der Informationen · Last Information
Update · Version des informations · Stand
van de informatie · Stan informacii
Stav informáci · Stav informáci: 09 / 2019 · Ident.-No.:
749.489_SBM69_2019-09-12_02_IM_Lidl_LB8_OS

IAN 322244