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Riedlinger Straße 28  
88524 Uttenweiler  
GERMANY

Last Information Update · Version des informations · Stand van  
de informatie · Stand der Informationen: 06/2023 · Ident.-No.:  
748.553\_2304\_SPO55\_2023-04-13\_01\_IM\_Lidl\_LB6\_BG/IE/NL/BE

CE 0483

IAN 437992\_2304

6

HealthForYou  
by **SILVERCREST**<sup>®</sup>

## PULSE OXIMETER

GB IE NI

### PULSE OXIMETER

Instructions for use and safety notes

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GB IE BE NL

GB IE NI

Before reading, unfold both pages containing illustrations and familiarise yourself with all functions of the pulse oximeter.

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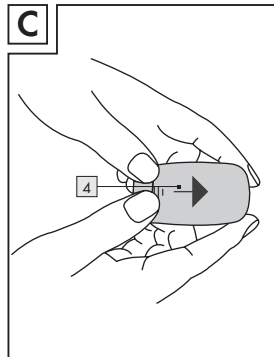
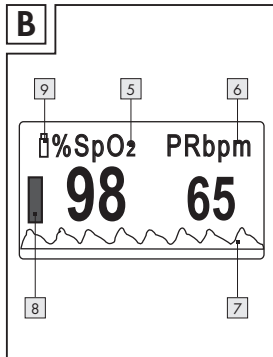
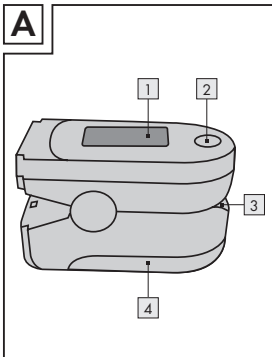
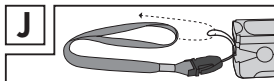
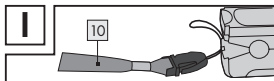
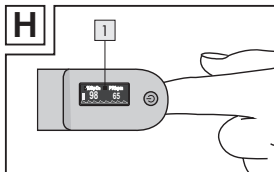
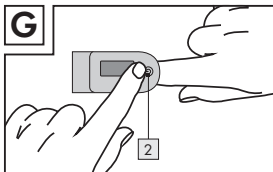
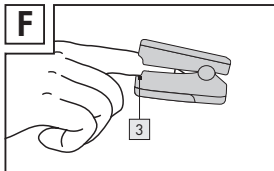
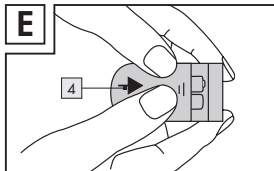
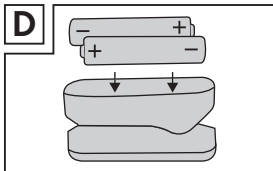
Scope of delivery..... Page 18

Technical data..... Page 18

**Important safety instructions** ..... Page 22

Battery safety information..... Page 27

**Prior to first use** ..... Page 29

Inserting / replacing batteries..... Page 29

Securing the strap ..... Page 30

"HealthForYou" app..... Page 30

**Use**..... Page 31

Measurements..... Page 35

Transferring measurements to the  
"HealthForYou" app via

*Bluetooth*<sup>®</sup> ..... Page 36

Evaluating measurements..... Page 39

Altitude-Related drop in  
oxygen saturation ..... Page 40

**Cleaning and care**..... Page 42

**Storage** ..... Page 43

**Troubleshooting** ..... Page 44

**Disposal**..... Page 47

**Warranty / service**..... Page 51

**Information on  
electromagnetic  
compatibility** ..... Page 54

### List of pictograms used

#### **WARNING**

Warning of risks of injury or health hazards

#### **Observe the instructions**

Read the instructions before starting to work and/or operating devices or machines.

### List of pictograms used

	<b>ATTENTION</b> This symbol shows that caution should be exercised if devices or controllers are being operated near this symbol or that the current situation requires attentiveness or intervention from the operator to avoid unintended consequences.	 %SpO <sub>2</sub>	Arterial oxygen saturation of haemoglobin (in percent)
	Product Information Note on important information	 PR bpm	Pulse rate (beats per minute)

8 | GB/IE/NI

### List of pictograms used

	Instructions for use		Permissible storage, transport and operating air pressure
Storage & Transport 	Permissible storage and transport temperature and humidity		Applied part, type BF
Operating 	Permissible operating temperature and humidity		Serial number

### List of pictograms used

	Manufacturer		Mute alarm
	This pulse oximeter complies with the requirements of the relevant European and national guidelines.	<b>IP22</b>	Pulse oximeter protects against foreign matter $\geq 12.5$ mm and against water falling diagonally
	Improper disposal of batteries can harm the environment!		Dispose of the packaging and pulse oximeter in an environmentally friendly manner!

10 | GB/IE/NI

### List of pictograms used

	Data transfer via <i>Bluetooth</i> <sup>®</sup> low energy technology		Recyclable
	Batteries included (2 x AAA)		Medical product
	Batch reference		Item number
	Model number		Date of manufacture

### List of pictograms used

	Unique Device Identifier (UDI) Unique identifier for product identification		Remove the product from all packaging components, then dispose of the packaging in accordance with local regulations.
	UKCA mark indicates conformity with relevant Great Britain regulations applicable for this product. (The UKCA mark logo is valid in Great Britain only.)		

## Pulse oximeter

### ● Introduction

Familiarise yourself with the pulse oximeter before using it for the first time. Please carefully read the following instructions for use and the important safety notes. Failure to observe the notes may result in personal injury or property damage. Only use the pulse oximeter as described and for the indicated

purposes. Keep these instructions for use in a safe place. Include all documentation when passing the pulse oximeter on to third parties.

### ● Intended use

The oxygen saturation indicates the percentage of haemoglobin in the arterial blood loaded with oxygen. The respiratory function is therefore an important parameter of assessment. The pulse oximeter uses two rays with different wavelengths which strike the finger inside the housing to take measurements.

### **Intended purpose**

The pulse oximeter serves for the non-invasive measurement of the arterial oxygen saturation (SpO<sub>2</sub>) and the heart rate (pulse rate) at home and in hospitals. This pulse oximeter is not suitable for implementing long-term measurement.

### **Target group**

The pulse oximeter is suitable for people who have a need for its diagnostic function. It is designed for users with a fingertip size in the range of approx.

10–20 mm in width and approx. 5–15 mm in thickness and who have no contraindications.

### **Indication**

The pulse oximeter is particularly suited for high-risk patients such as persons with heart disease, asthma, as well as for athletes and healthy persons in high altitudes (e.g. mountain climbers, skiers or amateur pilots). The pulse oximeter is also suitable for people without existing illnesses who wish to

measure their oxygen saturation or demonstrate the symptoms of a reduced oxygen saturation.

### **Clinical use**

The user can quickly and easily determine their oxygen saturation rate and determine a reduced oxygen saturation rate.

People with a low oxygen saturation rate often experience the following symptoms: shortness of breath, increased heart rate, impaired performance, nervousness and perspiration. Chronic and known low oxygen saturation requires monitoring with

your pulse oximeter under medical supervision. An acutely low oxygen saturation, with or without symptoms, requires immediate medical attention. This could be a sign of a life-threatening condition.

### ● **Function**

The SPO 55 pulse oximeter serves for the non-invasive measurement of the arterial oxygen saturation (SpO<sub>2</sub>) and the heart rate (pulse rate).

The oxygen saturation indicates the percentage of haemoglobin in the arterial blood loaded with

oxygen. The respiratory function is therefore an important parameter of assessment.

The pulse oximeter uses two rays with different wavelengths which strike the finger inside the housing to take measurements. A low oxygen saturation rate is primarily caused by illness (respiratory illnesses, asthma, heart failure, etc.). People with a low oxygen saturation rate often experience the following symptoms: shortness of breath, increased heart rate, impaired performance, nervousness and perspiration. Chronic and known

low oxygen saturation requires monitoring with your pulse oximeter under medical supervision. An acute low oxygen saturation, with or without symptoms, requires immediate medical attention. This could be a sign of a life-threatening condition. The pulse oximeter is therefore particularly suited for high-risk patients such as persons with heart disease, asthma, as well as for athletes and healthy persons in high altitudes (e.g. mountain climbers, skiers or amateur pilots).

## ● Improper use

**⚠ WARNING!** The pulse oximeter shows an instant measurement but cannot be used for continuous monitoring.

The pulse oximeter does not have an alarm and is therefore not suitable for analysing medical results.

Do not self-diagnose or self-treat based on the measurement results without consulting your attending physician. In particular, do not start new medication without consulting a doctor and do not modify the type and/or dosage of current medications.

The pulse wave and the pulse bar do not allow an assessment of the intensity of the pulse or circulation at the measured site but solely visualise the current signal visualisation at the site measured, but do not allow for a confident pulse diagnosis.

## ● Parts description

- 1 Display
- 2 Function key 
- 3 Finger opening
- 4 Battery compartment cover

## Display

- 5 Oxygen saturation (value in percent)
- 6 Pulse rate (value in beats per minute)
- 7 Pulse wave (plethysmographic waveform)
- 8 Pulse bar
- 9 Batty indicator
- 10 Strap

## ● Scope of delivery

- 1 SPO 55 pulse oximeter
- 1 strap
- 2 1.5V AAA batteries
- 1 set of instructions
- 1 belt pouch
- for use

18 | GB/IE/NI

1 quick start guide

## ● Technical data

- Type: CMS50D-BT
- Model no.: SPO 55
- Measuring method: non-invasive measurement of the arterial oxygen saturation of haemoglobin and pulse rate of the finger
- Measuring range: SpO<sub>2</sub> 0-100%,  
Pulse 30-250 beats/minute

Accuracy: SpO<sub>2</sub> 70–100 %, ± 2 %,  
Pulse 30–250 bpm,  
± 2 beats/minute

Dimensions: 58.5 mm x 31 mm x 32 mm  
(L x W x H)

Weight: approx. 27 g (without  
batteries)

Sensor system for  
measuring SpO<sub>2</sub>: red light (wavelength  
660 nm); infrared (wave-  
length 905 nm); silicon  
receiver diode

Admissible operating  
conditions: +10 °C to +40 °C,  
≤ 75 % relative humidity,  
700–1060 hPa ambient  
pressure

Admissible storage  
conditions: -40 °C to +60 °C,  
≤ 95 % relative humidity,  
500 – 1060 hPa ambient  
pressure

Power supply: 2 x 1.5 V  AAA batteries

GB/IE/NL | 19 

Battery life: 2 AAA batteries provide approx. 2 years of use at 3 measurements per day (60 seconds each).

Classification: IP22, applied part type BF

Expected service life: 3 years

Memory spaces: 100

Sensor information: Optical sensor for recording the user's oxygen saturation and heart rate.

Data transfer using *Bluetooth*<sup>®</sup> low energy technology:

The pulse oximeter uses *Bluetooth*<sup>®</sup> low energy technology, frequency band 2.402–2.480 GHz, maximum transmission power radiated in the frequency band < 2.87 dBm

For a list of compatible smartphones, information about the “HealthForYou” app and further details about the devices, please visit:  
[www.healthforyou.app](http://www.healthforyou.app)

The cloud is also an option for an additional backup. This means, for example, that data loss can be avoided if the smartphone is changed.

System requirements of the “HealthForYou” app:

iOS from version 14.0,  
Android™ from version 8.0,  
Bluetooth® from version 4.0.

The serial number is located inside the battery compartment.

Please always keep the operating system of the smartphone you use updated.

Technical specifications subject to change without notice due to updates.

## **Important safety instructions**

**⚠ WARNING!** This pulse oximeter is not intended to be used by persons (including children) with physical, sensory or mental impairments or lacking experience and/or knowledge unless supervised or instructed on the use of the pulse oximeter by a person responsible for their safety.

- Do NOT use the pulse oximeter
  - if you are allergic to rubber products.

- if the pulse oximeter or the respective finger is moist.
- on toddlers or infants.
- during an MRI or CT scan.
- on the arm with the cuff whilst measuring the blood pressure.
- on fingers with painted nails, dirt or adhesive bandages.
- on very large fingers which cannot easily be inserted in the pulse oximeter without using force (fingertip: width approx. > 20 mm, thickness approx. > 15 mm).

- on fingers with anatomic deformities, oedemas, scars or burns.
- on fingers which are too small in thickness and width, such as in toddlers (width approx. < 10 mm, thickness approx. < 5 mm).
- on patients who are restless in the area of use (e.g. trembling).
- near flammable or explosive gas mixtures.
- Prolonged use of the pulse oximeter may cause pain in persons with circulatory disorders. Therefore do not use the pulse oximeter on

the same finger for longer than approx. 2 hours.

- Do not look directly into the housing during measurement. The red light and the invisible infrared light of the pulse oximeter are harmful to the eye.
- Failure to observe the following instructions may result in faulty or failed measurements:
  - Do not use nail polish, artificial nails or other cosmetics on the monitored finger.

- Be sure the fingernail on the monitored finger is short enough so the finger pad covers the sensor elements inside the housing.
- Keep the hand, finger and body still during measurement.
- Oxygen saturation and heart rate measurements may be false or impossible in persons with arrhythmia.
- The pulse oximeter will display high measurements when carbon monoxide poisoning is present.

- To prevent false readings, keep bright light sources (e.g. fluorescent lamps or direct sunlight) out of the immediate surroundings of the pulse oximeter.
- Measurements may be faulty or false in persons with low blood pressure, jaundice or taking vasoconstriction medications.
- False measurements should be expected in patients who have received contrast agents in the past, and in patients with haemoglobin abnormality. This particularly applies to carbon monoxide poisoning and

methaemoglobin poisoning caused by e.g. local anaesthetics or due to methaemoglobin reductase deficiency.

- Protect the pulse oximeter from dust, shock, moisture, and extreme temperatures.
- Keep out of areas with explosive substances.
- Routinely inspect the pulse oximeter to ensure it is not visibly damaged before use and the battery level is sufficient. When in doubt, do not use and contact the service department or an authorised retailer.

- Do not use additional parts not recommended or sold as accessories by the manufacturer.
- Never open or repair the pulse oximeter or proper function cannot be guaranteed. Non-compliance will void the warranty.
- Please contact the service department or an authorised retailer for repairs.

**1. The following possible side effects of pulse oximeters have been reported:**

- finger injuries in the form of chemical or thermal burns
- sun tanning
- degradation through pressure
- loss of sensation
- gangrene

**2. Mechanisms of these complications may include:**

- pressure ischaemia
- prolonged use

- overheating of the sensor
- improper use of the sensor
- short circuiting

**3. Possible measurement error in a retaining member:** in this case the SpO<sub>2</sub> is determined to be too low.

**4. Low precision of the SpO<sub>2</sub> measurement in critically ill patients and newborns:**

- Inherent potential error of 3%–4% in measurements, which have been carried out on these patients.

## Battery safety information

- **DANGER TO LIFE!** Keep batteries out of the reach of children. Seek immediate medical attention if swallowed!
-  **EXPLOSION HAZARD!** Never recharge disposable batteries. Never short-circuit and/or open batteries. This could cause batteries to over-heat, catch fire or explode.

- Never throw batteries into fire or water.
- Never exert mechanical stress on batteries.

### Risk of battery leakage

- Avoid extreme conditions and temperatures which may affect batteries, e.g. radiators/ direct sunlight.
- If batteries have leaked, do not allow the chemicals to come into contact with skin, eyes and mucous membranes! Flush the affected areas immediately with clean water and seek medical attention!

■ **WEAR SAFETY GLOVES!** Batteries that have leaked or become damaged can cause burns upon contact with the skin. Therefore wear suitable protective gloves in this case.

■ In the event of a battery leak, immediately remove it from the pulse oximeter to prevent damage.

■ Only use the same type of batteries. Do not mix old and new batteries!

■ If the pulse oximeter is to be stored unused for a long period, the batteries should be removed.

#### **Risk of damage to the pulse oximeter**

■ Only use the specified battery type!

■ Insert the batteries according to the polarity marking (+) and (-) on the battery and the pulse oximeter.

■ Clean the contacts on the battery and inside the battery compartment before inserting!

- Remove spent batteries immediately from the pulse oximeter. When the battery symbol shows that the battery is spent, replace the battery.
- Do not use rechargeable batteries.

## ● **Prior to first use**

- Check that all parts listed in the scope of delivery are present.
- Remove all packaging material.

## ● **Inserting/replacing batteries**

- Slide the battery compartment cover 4 open (Fig. C).
- Insert the two included batteries (with the correct polarity) in the pulse oximeter (Fig. D). Please note the marking inside the battery compartment.
- Close the battery compartment cover 4 (Fig. E).

**Note:** All measurement data will be retained even after replacing the battery.

### ● **Securing the strap**

- To transport the pulse oximeter more easily, you can secure a strap to it.
  1. Push the narrow end of the strap through the bracket (Fig. I).
  2. Feed the other end of the strap through the loop of the narrow end, and pull tight (Fig. J).

### ● **“HealthForYou” app**

Download the free app “HealthForYou” from the Apple App Store and Google Play.

- Follow the installation instructions on the screen.

The “HealthForYou” privacy policy can be viewed via the following link:  
[www.healthforyou.app/privacy.html](http://www.healthforyou.app/privacy.html)

The “HealthForYou” app provides 2-factor authentication. On the one hand, this consists of a combination of an email address and password, selected by you. It also consists of confirming the smartphone that you use. The smartphone used

to log in is confirmed automatically. More information and an instruction manual can be found in the app within the account section of your profile as well as in the FAQs at [www.healthforyou.app](http://www.healthforyou.app).

## ● Use

The function key **2** on the pulse oximeter has 4 functions:

Switching on

- Briefly press the function key **2** to switch on the pulse oximeter.

Activating and deactivating  
*Bluetooth*<sup>®</sup>

- Briefly press the function key **2** to switch on the pulse oximeter.

Activating and deactivating  
*Bluetooth*<sup>®</sup>

- Hold the function key **2** on the switched on pulse oximeter down for 5 seconds to proceed to *Bluetooth*<sup>®</sup> settings. “ ON” or “ OFF” will appear on the display.
- Briefly press the function key **2** to activate ( ON) or deactivate ( OFF) *Bluetooth*<sup>®</sup>.
- To return to measuring, hold the function key **2** down for 5 seconds.
- To switch off the pulse oximeter, wait 10 seconds. The pulse oximeter will switch off automatically.

Transferring measurements to the "HealthForYou" app and synchronising the time and date

There are two options for transferring data to the "HealthForYou" app. *Bluetooth*<sup>®</sup> must be activated (⌘ ON) on your smartphone and on the pulse oximeter for both options.

**Note:** The "HealthForYou" app must be open for the transfer to occur.

**Option 1:** Hold the function key 2 on the switched off pulse oximeter down for 5 seconds to transfer the data to the app.

**Option 2:** The data will be transferred to the app automatically after the measurement. If *Bluetooth*<sup>®</sup> is deactivated (⌘ OFF), the notification ⌘ OFF will be displayed after your measurement has been carried out.

## Brightness

To set your desired display brightness, hold the function key **2** down for longer during operation.

### **Note**

- The orientation of the display (landscape, portrait) is automatic. This means that it is always easy to read the values on the display, regardless of how you hold the pulse oximeter.
- The memory of the pulse oximeter has over 100 storage slots. If these are all full, the oldest value will be overwritten.
- To reset the pulse oximeter to its factory settings, proceed as follows:
  1. Remove one of the two batteries inserted into the pulse oximeter.

2. Press and hold the function key **2** and reinsert the battery.
3. After inserting the battery, hold down the function key **2** for a further 5 seconds. If you have successfully followed the instructions, "RESET ✓" will appear in the display.

**Note:** If the device has been reset to factory settings, then it must be removed from *Bluetooth*® settings on your smartphone in order to reconnect it.

## ● Measurements

- Slide one finger into the finger opening **3** on the pulse oximeter. Hold the finger still (Fig. F).
- Press the function key **2**. The pulse oximeter will now begin the measurement. Do not move during measurement (Fig. G).
- After a few seconds the display **1** will show your measurements (Fig. H).
- When you have removed your finger from the pulse oximeter, the pulse oximeter will automatically switch off after approx. 10 seconds.

## ● Transferring measurements to the “HealthForYou” app via Bluetooth®

**Note:** Bluetooth® must be activated (🔌 ON) for the transfer to occur.

**Note:** the “HealthForYou” app must be open for the transfer to occur.

The time and date will be synchronised with the smartphone during every data transfer.

## ! Note

As soon as you have connected the pulse oximeter with your smartphone, the current date and time are updated on the pulse oximeter.

We therefore recommend connecting the pulse oximeter with your smartphone before taking your first measurement, so that all measurements are saved with the current date and correct time.

To transfer measurements to your smartphone via *Bluetooth*<sup>®</sup> proceed as follows:

- Activate *Bluetooth*<sup>®</sup> in your smartphone settings, open the “HealthForYou” app and follow the instructions.
- Select and connect to the SPO 55 in the settings menu of the app. A randomly generated six digit PIN code will be displayed on the pulse oximeter when connecting for the first time. At the same time, an entry

field will appear on your smartphone where you should enter this six digit code.

There are two options for synchronising data to your smartphone. *Bluetooth*<sup>®</sup> must be activated (ON) on your smartphone and on the pulse oximeter for both options. The “HealthForYou” app must also be open on the smartphone.

**Option 1:** Hold the function key 2 on the switched-off pulse oximeter down for 5 seconds.

"SYNC" will flash on the display **1**. The pulse oximeter will now attempt to connect to the app for 30 seconds. Once connected, "SYNC" will stop flashing. All stored measurements will be automatically transferred to the app. The pulse oximeter will then automatically switch off.

**Option 2:** The data will be transferred to the app automatically after the measurement. "SYNC" will flash on the display **1**.

The pulse oximeter will attempt to connect to the app for 30 seconds.

Once connected, "SYNC" will stop flashing. All stored measurements will be automatically transferred to the app. The pulse oximeter will then automatically switch off.

If *Bluetooth*<sup>®</sup> is deactivated (⌘ OFF), the notification ⌘ OFF will be displayed after your measurement has been carried out.

## ● Evaluating measurements

**⚠ WARNING!** The following table for assessing measurements does NOT apply to persons with certain pre-existing conditions (e.g. asthma, heart failure, respiratory disorders) or in altitudes over 1500 metres.

If you have a pre-existing condition, always consult your physician to analyse your measurements.

SpO <sub>2</sub> (oxygen saturation) measurement in %	Classification/required action
99-94	Normal range
93-90	Low range: medical consultation recommended
< 90	Critical range: medical consultation urgently required

<b>SpO<sub>2</sub> (oxygen saturation) measurement in %</b>	<b>Classification/ required action</b>
Source: According to "W" Windisch et al., S2k-Guideline: Non-invasive and Invasive Ventilation for Treatment of Chronic Respiratory Failure - Update 2017; Pneumologie 2017; 71: 722-795"	

## ● Altitude-Related drop in oxygen saturation

**Note:** The following table provides information on how different altitudes affect the oxygen saturation rate and how this affects the human organism. The following table does NOT apply to persons with certain pre-existing conditions (e.g. asthma, heart failure, respiratory disorders, etc.). Persons with pre-existing conditions may already experience symptoms (e.g. hypoxia) in low altitudes.

<b>Altitudes</b>	<b>Expected SpO<sub>2</sub> value (oxygen saturation rate) in %</b>	<b>Affects on people</b>
1500-2500 m	> 90	No anoxia (typically)
2500-3000 m	~ 90	Anoxia, acclimatisation recommended

Source: Hackett PH, Roach RC: High-Altitude Medicine. In: Auerbach PS (ed): Wilderness Medicine, 3rd edition; Mosby, St. Louis, MO 1995; 1-37.

## ● Cleaning and care

### **ATTENTION! POSSIBLE PROPERTY**

**DAMAGE!** Do not use high pressure sterilisation on the pulse oximeter! Never hold the pulse oximeter under water, as liquid may enter and damage the pulse oximeter.

- Clean the housing and the rubberised inner surface of the pulse oximeter with a soft cloth dampened with medical alcohol after

every use. The unit can be cleaned with 75% alcohol for 3 seconds.

- Replace the batteries if the display **1** on the pulse oximeter shows the battery is low **9**.
- If the pulse oximeter will not be used for more than one month, remove both batteries from the pulse oximeter to prevent possible battery leakage.

## ● Storage

### **ATTENTION! POSSIBLE PROPERTY**

**DAMAGE!** Store the pulse oximeter in a dry location (relative humidity  $\leq 95\%$ ). High humidity can shorten the life of or damage the pulse oximeter. Store the pulse oximeter in a location with ambient temperatures between  $-40\text{ }^{\circ}\text{C}$  and  $60\text{ }^{\circ}\text{C}$ .

## ● Troubleshooting

Problem	Possible cause	Solution
The pulse oximeter doesn't display measurements.	The batteries inside the pulse oximeter are drained.	Replace the batteries.
	The batteries were inserted incorrectly.	Reinsert the batteries. If measurements are not displayed after inserting the batteries correctly, please contact service.

<b>Problem</b>	<b>Possible cause</b>	<b>Solution</b>
Pulse oximeter shows interrupted measurements or severe jumps in the measurement.	Circulation problems in the finger being measured.	Refer to warnings and safety notes.
	The finger being measured is too large or too small.	The fingertip must be the following size: Between 10 and 20 mm in width Between 5 and 15 mm thick
	The finger, hand or body are moving.	Keep the finger, hand and body still during measurement.
	Cardiac arrhythmia	Seek medical attention.

Problem	Possible cause	Solution
Unable to transfer the measurements.	The "HealthForYou" app is not open or <i>Bluetooth</i> ® is off in the smartphone settings.	Activate <i>Bluetooth</i> ® on your smartphone and start the app.
	The batteries in the pulse oximeter are too low or drained.	Replace the batteries.
	The pulse oximeter is still on.	Wait for the pulse oximeter to switch off.
Unable to transfer the measurements.	<i>Bluetooth</i> ® is not switched on.	Activate <i>Bluetooth</i> ® (ⓑ ON).

## ● Disposal

The packaging is made from environmentally-friendly materials which can be disposed of through your local recycling facilities.

Please note the markings on the packaging when sorting waste, these are marked with abbreviations (a) and numbers (b) meaning: 1-7: plastics/20-22: paper and cardboard/80-98: composites.

Contact your local authorities for information on how to dispose of the pulse oximeter at the end of its product life.

When your pulse oximeter has come to the end of its life, please do not throw it out with your household refuse, but in the interest of the environment dispose of it properly. Please contact your municipality for information on collection facilities and their opening hours.

## **User notices for the disposal and scrapping of electronic equipment waste in Germany**

In this context, owners of electronic waste can return it via return or collection options for electronic waste provided by public disposal authorities to ensure the proper disposal of electronic waste. Returning these items is a legal requirement. Furthermore, it is also possible to return the devices to distributors if certain requirements are met.

According to the Electrical and Electronic Equipment Act (ElektroG), the following distributors are required to accept electronic waste free of charge:

- Electronics stores with a retail space for electrical and electronic devices of at least 400 square metres
- Grocery stores with a total retail space of at least 800 square metres, who offer and sell electrical and electronic devices multiple times per year or permanently.

- Mail-order businesses, however, the requirement for 1:1 returns for private households only applies to heat exchangers (cooling/refrigeration appliances, air conditioners, etc.), devices with screens and large appliances. For 1:1 returns of lamps, small devices, and small computer or telecommunication devices, as well as 0:1 returns, mail-order retailers must provide users with return options within a reasonable distance.

When furnishing a user with a new electrical or electronic device, these distributors are required

- to accept electronic waste of the same type, which fundamentally fulfils the same functions as the new device, at the location of the provision, or in the immediate vicinity, free of charge and
- upon the request of the end user, electronic waste, with external dimensions that are no larger than 25 centimetres at the retail location or its immediate vicinity; the return may not require the purchase of an electrical or electronic device and is limited to three electronic waste devices per device type.

If the products contain batteries or rechargeable batteries or light bulbs that can be removed from the electronic waste device without destroying it, then these must be removed before disposal and disposed of separately as a battery or light bulb. If the electrical or electronic waste device contains personal data, you are responsible for erasing this data before returning it.

Defective or used batteries must be recycled according to Directive 2006/66/EC and its

amendments. Return the batteries and/or the product to the recycling facilities provided.

### **Improper disposal of batteries can harm the environment!**

Never dispose of batteries in your household waste. They may contain toxic heavy metals and are subject to hazardous waste regulations. The chemical symbols of the heavy metals are: Cd = cadmium, Hg = mercury, Pb = lead. Therefore,

you should dispose of used batteries through your local collection facilities.

## ● **Warranty/service**

The pulse oximeter is covered by a 3 year warranty from the date of purchase for defects in material and manufacture.

The warranty does not cover:

- damage due to improper use.
- wear parts.

- damage caused by the customer.
- instances where the pulse oximeter has been opened by an unauthorised repair shop.

This warranty does not affect the customer's legal rights. The customer must provide proof of purchase in order for any warranty claims within the warranty period to be honoured. The warranty claim must be filed with Hans Dinslage GmbH, Uttenweiler, Germany, within 3 years from the date of purchase.

**In the event of a complaint, please contact our Service Department listed below:**

**Service Hotline (toll-free):**

**GB**

**0800 93 10 319**  
**(Mon–Fri: 8 am–6 pm)**

**service-uk@sanitas-online.de**

**IE NI**

**1800 93 92 35 (Mon – Fri: 8 am – 6 pm)**

**service-ie@sanitas-online.de**

If we ask you to return the defective pulse oximeter, please send the product to the following address:

NU Service GmbH  
Lessingstraße 10 b  
89231 Neu-Ulm  
GERMANY

## **IAN 437992\_2304**

This pulse oximeter complies with European Standards EN60601-1 and EN60601-1-2 group 1, class B (conformity with CISPR 11, IEC 61000-4-2, IEC 61000-4-3, IEC 61000-4-8) and is subject to special precautions with respect to electromagnetic compatibility. Please note that portable and mobile RF communication devices may interfere with this pulse oximeter. You can request further information from the customer

service address below. We wish to expressly indicate that the software at hand is not a medical product in accordance with EU Regulation 2017/745.

The CE declaration of conformity for this pulse oximeter is available at: [www.sanitas-online.de/web/de/landingpages/cedecclarationofconformity.php](http://www.sanitas-online.de/web/de/landingpages/cedecclarationofconformity.php)

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Riedlinger Straße 28

88524 Uttenweiler

GERMANY

## ● **Information on electromagnetic compatibility**

The pulse oximeter is suitable for use in all environments that are listed in these instructions for use, including in the home.

54 | GB/IE/NI

In the presence of electromagnetic disturbances, the pulse oximeter may under some circumstances only be useable to a limited extent. This can result in error messages or the breakdown of the display/pulse oximeter, for example.

The use of the pulse oximeter immediately next to other devices or stacked with other devices is to be avoided, as this can result in faulty operation. If it is necessary to use this pulse oximeter in such a way despite this, this pulse oximeter and

the other devices should be monitored to make sure that they are working correctly.

The use of equipment other than that which has been specified or made available by the manufacturer of this pulse oximeter can lead to electromagnetic interference or the reduced electromagnetic stability of the pulse oximeter and can lead to faulty functioning.

Keep portable RF communication devices (including peripheral equipment such as antenna cables

or external antennae) at least 30 cm away from all device parts, including all cables supplied.

Failure to observe these instructions may lead to a reduction in the performance features of the pulse oximeter.

*Bluetooth*<sup>®</sup> and the related logo are registered trademarks of *Bluetooth SIG, Inc.* Use of these brands by Hans Dinslage GmbH is under license.

All other brands and trade names are the property of the respective owners.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries.

Google Play and the Google Play logo are trademarks of Google LLC. Android is a trademark of Google LLC.

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*Bluetooth*<sup>®</sup> and the related logo are registered trademarks of *Bluetooth* SIG, Inc. Use of these

brands by Hans Dinslage GmbH is under license. All other brands and trade names are the property of the respective owners.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries. Google Play and the Google Play logo are trademarks of Google LLC. Android is a trademark of Google LLC.

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*Bluetooth*<sup>®</sup> and the related logo are registered trademarks of *Bluetooth SIG, Inc.* Use of these brands by Hans Dinslage GmbH is under license. All other brands and trade names are the property of the respective owners.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other

countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries. Google Play and the Google Play logo are trademarks of Google LLC. Android is a trademark of Google LLC.