GB IE NI

Before reading, fold out the illustration page and get to know all of the functions of your unit.

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



GB IE NI DK FR  
BE NL DE AT CH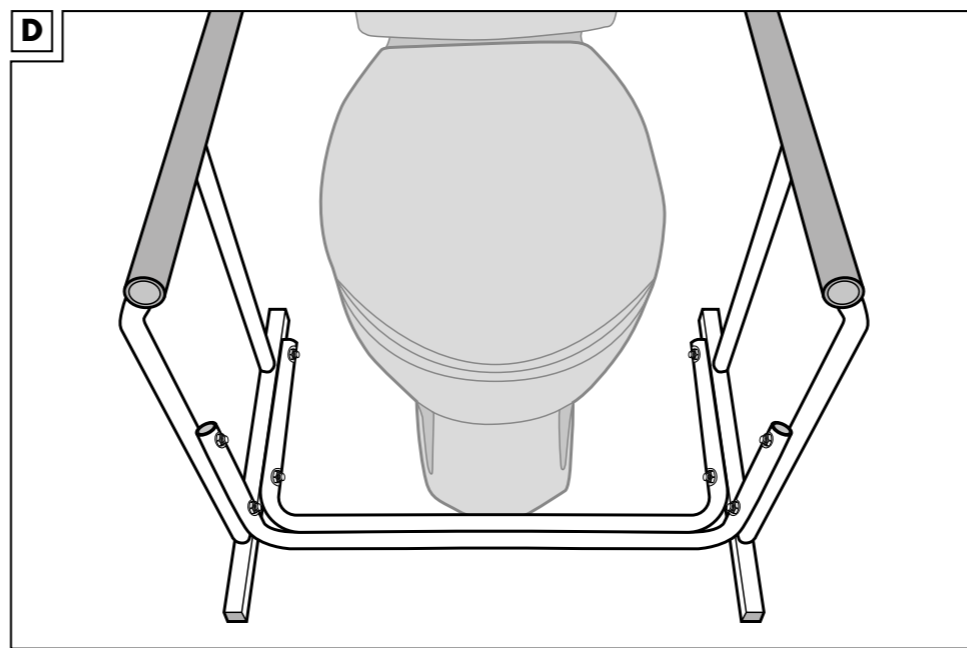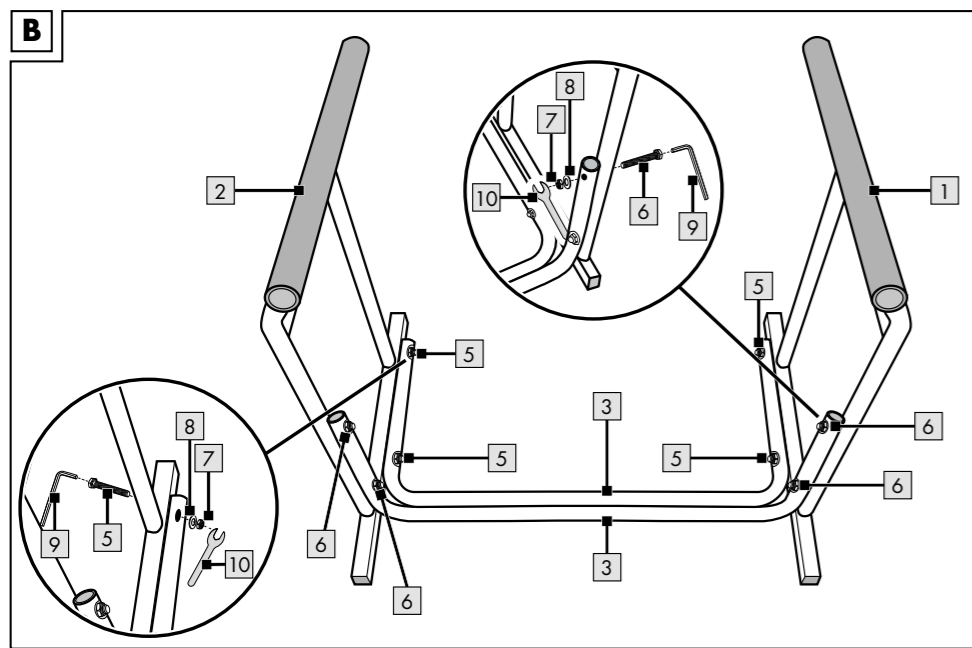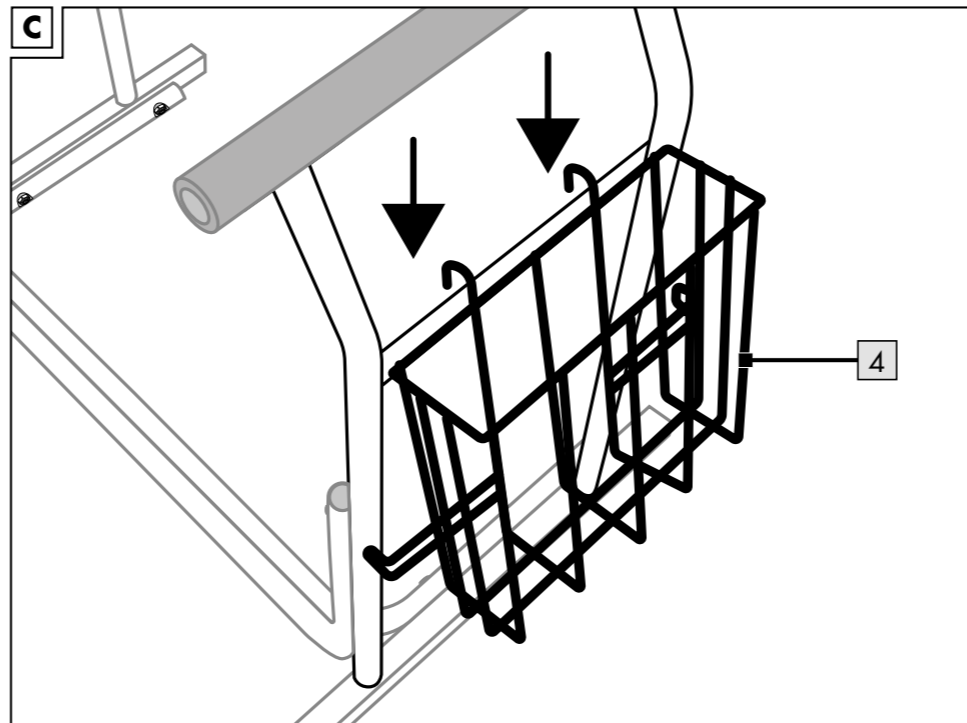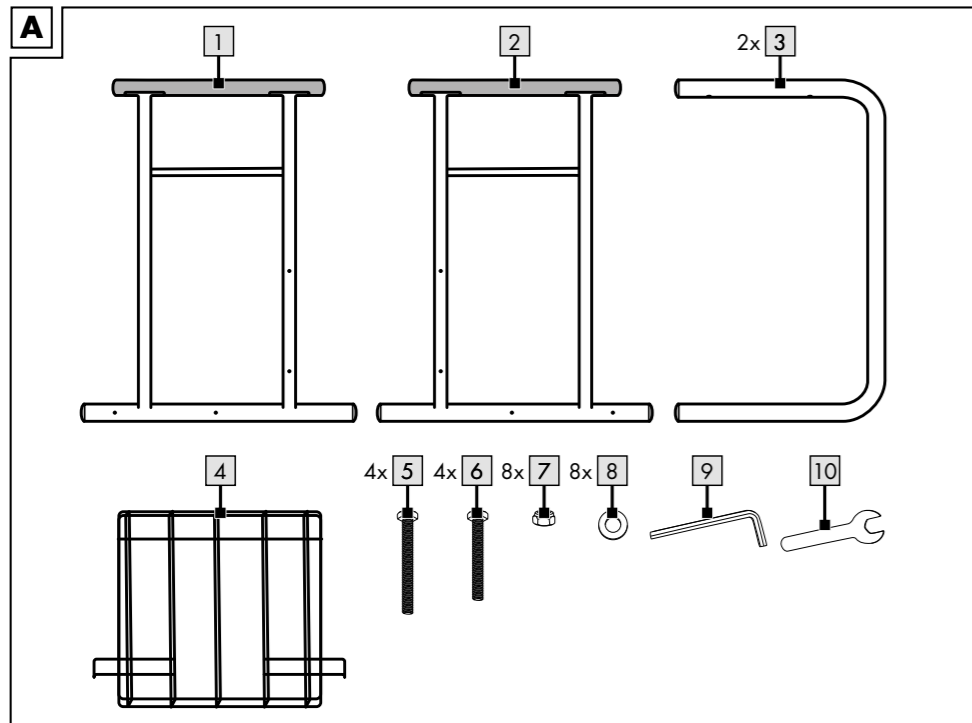

Congratulations!

You have chosen to purchase a high-quality item. Familiarise yourself with the item before using it for the first time.

 **Read the following instructions for use carefully.**

Only use this item in the manner described and for the purpose for which it is intended. Store these instructions for use carefully. When passing the item on to third parties, please also hand over all accompanying documents.

## Scope of delivery (fig. A)

- 1 x toilet aid, unassembled (1 - 3)
- 1 x hanging basket (4)
- 1 x assembly materials (5 - 8)
- 1 x Allen key (9)
- 1 x spanner key (10)
- 1 x instructions for use

## Technical data

Max. user weight: 110kg

Date of manufacture (month/year):  
05/2019

## Intended use

This product is for private use to be employed exclusively as an aid to facilitate standing up from the toilet. The product is not suitable for use by persons with mental disabilities.

This is not a medical device and has not been designed for the purpose of rehabilitation or medical care of individuals.

## Safety instructions

### **Danger to life!**

- Never leave children unsupervised with the packaging material. There is danger of suffocation.

### **Risk of injury!**

- Make sure that all parts are undamaged and have been assembled correctly. There is risk of injury if assembled incorrectly. Damaged parts can affect safety and function.

- Check the product before use each time for damage or wear. Check before first use that all screw connections are tight!  
The product may be used only when in good working order and condition! Do not use the product when there is visible or suspected damage.
- Use the product only for its intended purpose.
- The product may only be used under adult supervision and not as a toy.
- Do not grasp the product with wet hands. Otherwise the handles will lose their anti-slip properties.
- Never stand or sit on the product.
- Do not use the product as an aid to walking.

## Assembly

1. Assemble the product as shown in fig. B.
2. Hang the hanging basket (4) on one side of the product (fig. C).
3. Place the product in front of your toilet, as shown in fig. D.

**Note:** Do not secure the product in addition.

**Important:** Position the product very close to your toilet so that the frame cannot cause you to stumble. Be sure to approach the toilet only with dry footwear or feet since otherwise there is a risk of slipping or stumbling.

## Storage, cleaning

Always store the product clean and dry at room temperature.

Use only water to clean and then wipe dry with a cleaning cloth.

**IMPORTANT!** Never clean with harsh detergents.

## Disposal

Dispose of the article and the packaging materials in accordance with current local regulations.

Packaging materials such as foil bags are not suitable to be given to children. Keep the packaging materials out of the reach of children.

Dispose of the products and the packaging in an environmentally friendly manner.

The recycling code is used to identify various materials for recycling. The code consists of the recycling symbol – which is meant to reflect the recycling cycle – and a number which identifies the material.

## Notes on the guarantee and service handling

The product was produced with great care and under constant supervision. You receive a three-year warranty for this product from the date of purchase. Please retain your receipt.

The warranty applies only to material and workmanship and does not apply to misuse or improper handling. Your statutory rights, especially the warranty rights, are not affected by this warranty.

With regard to complaints, please contact the following service hotline or contact us by e-mail.

Our service employees will advise as to the subsequent procedure as quickly as possible. We will be personally available to discuss the situation with you.

Any repairs under the warranty, statutory guarantees or through goodwill do not extend the warranty period. This also applies to replaced and repaired parts. Repairs after the warranty are subject to a charge.

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