COLD COMPRESS SET

(GB)

COLD COMPRESS SET

Instructions for use (2)

(HU)

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(SI)

UDOBNJI HLADNI OBKLADKI

Navodila za uporabo (4)

(CZ)

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(SK)

STUDENÉ OBKLADY

Návod na použitie (6)

(DE AT CH)

KOMFORT-KÄLTEKOMPRESSEN

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IAN 379117_2110

(HU) (SI) (CZ) (SK)

(GB) (DE) (AT) (CH)

Congratulations on the purchase of your new product. You have chosen a high-quality article. The instructions for use are part of this product. They contain important information on safety and use. Please familiarise yourself with the instructions and safety information before using this product. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the compress and be sure to include them when handing on the product to another person.

Medical device

Only intended for private use!

Note:

The medical device is intended for adults who are able to read, understand and independently follow these instructions for use. A reason for use, as named under Indications, must be present at the user.

The item is not intended for groups of persons who are excluded from its application by the contraindications listed below.

Scope of delivery

- 2 x cold compress
- 1 x protective cover
- 1 x instructions for use

Intended purpose of the cold compress:

Cooling compress for external use

Indications

For supporting use in:

- To soothe headaches and migraines
- To treat hematoma and bruises
- To ease the pain of sprains and burns
- For fever reduction
- For the alleviation of tooth aches
- For the treatment of insect bites

Use of the cold compress

Prior to use, the gel cold compress will have to be placed into the refrigerator or freezer for about an hour **(1)**. When cold, place the cold compress into the designated protective cover **(2)** and then on the area that hurts **(3)**.

Contraindications

 **Please do not use the item or consult your doctor before using this product if you:**

- Have circulation problems
- Your temperature sensitivity is limited
- Are taking prescription pain medication
- Have open wounds or damaged skin

Warnings

These compresses are designated for external use only and thus have to be stored out of children's reach. They must not be used on children who are less than three years old. Prior to any use, please check the compress for damages and do not use it if you find any. In this case, please dispose of it along with your household waste. Keep away from naked flames. Do not use this compress outside of its protective cover under any circumstances, as this will expose the patient to a risk of freezing skin burns!

Care Instructions

Use a mild soap and rinse the compress under warm water (do not use any other cleaning products or disinfectants). Dry the compress with a soft towel.

Instructions for disposal

Please dispose of the product in an environmentally-friendly way through an approved disposal centre or your local waste facility. Observe currently applicable regulations.

Obligation to notify

All serious incidents relating to the product must be reported immediately to the manufacturer and the competent authority.

3-year warranty

The product has been created with great care and under constant monitoring. You will receive a 3-year warranty for this product from the date of purchase. Please keep the receipt. The warranty is valid only for material and manufacturing faults and lapses in the event of misuse or improper handling. Your statutory rights, particularly the warranty rights, are not restricted by this warranty. If you have any complaints, please contact the service hotline below or contact us by email. Our service staff will discuss how to proceed further with you as soon as possible. In any case, we will personally advise you. The warranty period is not extended by any repairs under the terms of the warranty, statutory warranty or goodwill. This also applies to replaced and repaired parts. After expiry of the warranty, incurred repairs are chargeable.

Service Germany

Tel.: +49 (0)40 82 22 44-80

E-Mail: info@horizonte.de

Consult instructions for use

Attention:

Always adhere to the stated warning information

Product number

Batch designation

Use by

Date of manufacture (year): 2022

Manufacturer

CE Marking of conformity

Medical device

Caution! Very cold surface!

Please do not place in the microwave

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