## WORKWEAR BACK SUPPORT / RÜCKEN-ARBEITSGURT / CEINTURE LOMBAIRE

**GB IE NI MT**

### WORKWEAR BACK SUPPORT

Instructions for use (2-3)

**DE AT BE CH**

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**FR BE CH**

### CEINTURE LOMBAIRE

Mode d'emploi (4-6)

**NL BE**

### RUG-WERKGORDEL

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**IT CH MT**

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**ES**

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**DK**

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Brugsanvisning (11-12)

**PL**

### PAS NA PLECY

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**HU**

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**SI**

### OPORNI PAS ZA HRBET

Navodila za uporabo (16-17)

**CZ**

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**SK**

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Návod na použitie (19-20)

**RO**

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**BG**

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GB IE NI MT

Congratulations on the purchase of your new product. You have chosen a high-quality article. Please familiarise yourself with the instructions and safety information before using this product. The instructions for use are part of this product. They contain important information on safety and use. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the product and be sure to include them when handing on the product to another person.

## Medical device

### Only intended for private use!

### Note

The medical device is intended for adults who are able to read, understand and independently follow these instructions for use. A reason for use, as named under Indications, must be present at the user. The item is not intended for groups of persons who are excluded from its application by the sizes and contraindications listed below.

## Scope of delivery

- 1 x back support belt
- 2 x shoulder strap
- 1 x cross-piece
- 2 x hook-and-loop fastener
- 1 x instructions for use

## Technical Data

Size	Waist circumference
S-M	69 – 92 cm
L-XL	88 – 107 cm

## Intended purpose

Stabilising brace which can help to relieve back strain. Increases physical resilience.

## Indications

To be used as support for:

- Muscle tension
- Muscular imbalance
- Physical stress

## Contraindications

 **Do not use – or use only after consultation with your doctor – for:**

- you are unsure about how to use the product
- you have acute injuries
- unforeseen problems arise
- Circulatory disorders.

## Safety instructions

Risk of injury

-  If problems occur while wearing the bandage, remove the it immediately and consult a doctor.
-  Check the bandage before each use for tears and damage.
-  Use the product only in perfect condition, as described and for the specified area of use.
-  Wear a T-shirt or a long-sleeved shirt under the brace.
-  Never put the bandage on so tightly that the blood circulation is restricted.
-  Do not wear the bandage longer than 2 to 3 hours at a stretch and a maximum of 6 to 8 hours a day.
-  Do not wear the bandage while sleeping or during longer rest phases such as long periods of sitting.
-  The bandage may only be used on uninjured skin.
-  To avoid allergic skin reaction, the skin must be free of grease, gels, creams, oils or similar materials.

## Putting the bandage on

The easiest way to put the bandage on is while standing.

- The compression belts should be undone or loosened before you put the brace on.
- Put the shoulder belts on as you would a jacket by inserting your arms through both loops (fig. 1).
- Lay the bandage around the desired area of the back and close the Velcro strip of the bandage (fig. 2).
- Fasten the side Velcro fasteners of the compression strap one at a time (fig. 3).
- The bandage should sit comfortably and be slightly stretched.
- The position of the shoulder belts can be adjusted as required by using the hook-and-loop fasteners and the cross-piece.
- The buckles on the front can be used to adjust to length of the belt.

## Cleaning and care

- Before using it for the first time, the bandage should be thoroughly washed.
- Close the Velcro pieces during the washing process.
- Wash separately or with similar colours.

## Instructions for disposal

Please dispose of the product in an environmentally-friendly way through an approved disposal centre or your local waste facility. Observe currently applicable regulations.

## Obligation to notify

All serious incidents relating to the product must be reported immediately to the manufacturer and the competent authority.

## 3-year warranty

The product has been created with great care and under constant monitoring. You will receive a 3-year warranty for this product from the date of purchase. Please keep the receipt. The warranty is valid only for material and manufacturing faults and lapses in the event of misuse or improper handling. Your statutory rights, particularly the warranty rights, are not restricted by this warranty. If you have any complaints, please contact the service hotline below or contact us by email. Our service staff will discuss how to proceed further with you as soon as possible. In any case, we will personally advise you. The warranty period is not extended by any repairs under the terms of the warranty, statutory warranty or goodwill. This also applies to replaced and repaired parts. After expiry of the warranty, incurred repairs are chargeable.

Service Germany

Tel.: +49 (0)40 82 22 44-80

E-Mail: support@horizonte.de

-  Consult instructions for use
-  Attention: Always adhere to the stated warning information
-  Catalogue number
-  Batch designation
-  Use by
-  Date of manufacture (year): 2025
-  Manufacturer
-  Indicates the authorised representative in Switzerland (for products purchased in Switzerland)
-  Importer
-  CE Marking of conformity
-  Medical device

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8048 Zürich, SWITZERLANDLidl Schweiz DL AG