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www.sanitas-online.de/web/de/landingpages/cedecclarationofconformity.php

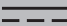
List of pictograms used	Page 47
Introduction	Page 49
Intended use	Page 50
Intended purpose.....	Page 50
Target group.....	Page 50
Clinical use.....	Page 50
Indication.....	Page 50
Contraindication.....	Page 51
Parts description.....	Page 51
Scope of delivery.....	Page 52
Technical data.....	Page 52
Warning and safety notes	Page 55
General warnings.....	Page 55
General precautions.....	Page 56
Battery safety information.....	Page 57
Information on electromagnetic compatibility.....	Page 58
Prior to first use	Page 59
Inserting batteries.....	Page 59
"HealthForYou" app.....	Page 60
Settings.....	Page 60
Bluetooth®.....	Page 61
Measuring blood pressure	Page 61
Attaching the cuff.....	Page 62
Adopting the correct posture.....	Page 63
User selection.....	Page 63
Measuring.....	Page 64
Transferring measurements via Bluetooth®	Page 64



Evaluating measurements	Page 66
General information about blood pressure	Page 66
Cuff position control	Page 66
Introduction to cardiac arrhythmia	Page 67
Risk indicator	Page 67
Resting indicator (through HSD diagnostic).....	Page 69
Reviewing and deleting measurements	Page 70
Cleaning and care	Page 72
Storage	Page 72
Accessories and space parts	Page 72
Troubleshooting	Page 73
Repairing and disposing of the device	Page 74
Disposal	Page 74
Warranty / service	Page 77

List of pictograms used	
	WARNING Warning of risks of injury or health hazards
	ATTENTION This symbol shows that caution should be exercised if devices or controllers are being operated near this symbol, or that the current situation requires attentiveness or intervention from the operator to avoid unintended consequences.
	Product Information Note on important information
	Instructions for use
	Observe the instructions Read the instructions before starting to work and/or operating devices or machines.
	Direct current This device is only suitable for direct current.
	Temperature range
	Humidity range
	Air pressure limit
	Manufacturer
	Authorised EU representative for manufacturers of medical devices
	EXPLOSION HAZARD! Never recharge disposable batteries. Never short-circuit and/or open batteries.

List of pictograms used	
	WEAR SAFETY GLOVES! Leaking or damaged batteries can cause burns upon contact with the skin. Therefore wear suitable protective gloves in this case.
	Importer
	Isolation of type BF applied parts Electrically isolated application part (F stands for floating); meets the requirements for Type B leakage currents.
	Serial number
	CE mark This product fulfils the requirements of the applicable European and national directives.
	Do not dispose of batteries containing hazardous substances with household waste.
IP21	Protected against solid foreign objects, 12.5 mm in diameter and larger and against water falling vertically.
	Disposal according to the EC Waste Electrical and Electronic Equipment Directive (WEEE).
	Labelling to identify the packaging material. A = material abbreviation B = material number 1-7 = plastics 20-22 = paper and cardboard
	Dispose of the packaging and blood pressure monitor in an environmentally friendly manner!

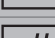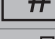List of pictograms used	
	Data transfer via <i>Bluetooth</i> ® low energy technology
	Medical product
	Unique Device Identifier (UDI) Unique identifier for product identification
	Batch reference
	Item number
	Model number
	Date of manufacture
	Remove the product from all packaging, then dispose of the packaging in accordance with local regulations.

Upper Arm Blood Pressure Monitor

● Introduction

 Read these instructions for use carefully. Observe the warnings and safety instructions. Keep these instructions for use for future reference. Make these instructions for use accessible to other users. Also include these instructions for use when passing the device on to others.

● **Intended use**

● **Intended purpose**

The blood pressure monitor is intended for the fully automatic, non-invasive measurement of arterial blood pressure and pulse frequency. It is designed for the purpose of self-monitoring by adults in domestic environments.

● **Target group**

Blood pressure monitoring is suitable for adult users whose upper arm circumference is within the area printed on the cuff.

● **Clinical use**

The user can quickly and easily record their blood pressure and pulse values using the device. The recorded blood pressure measurements are classified in accordance with internationally applicable guidelines and evaluated graphically. Furthermore, the device can identify irregular heartbeats that occur while measuring, and inform the user via a symbol on the display. The device saves the displayed measurements and can also provide average values of previous measurements. The data recorded can support healthcare providers when diagnosing and treating blood pressure problems and thus contribute to the long-term monitoring of the user's health.

● **Indication**

If the user has hypertension or hypotension, they can monitor their blood pressure, pulse values and potential irregular heartbeats independently at home. However, the user does not have to suffer from hypertension or irregular heartbeats to use the device.

Contraindication

- Do not use this product on newborns or pre-eclampsia patients. We recommend consulting your doctor before using the product if you are pregnant.
- The device is not suitable for use by patients with implanted electrical devices such as pacemakers or defibrillators. Do not use the device if you have metal implants.
- People with limited physical, sensory or mental abilities should be supervised by a person responsible for their safety and adhere to this person's instructions for using the device.
- Do not place the cuff over wounds as this could lead to further injuries.
- Ensure that the cuff is not placed on the arm of a person whose arteries or veins are undergoing medical treatment, i.e. intra-vascular access or intra-vascular therapy or an arteriovenous (A-V) shunt.
- Unforeseen side effects:
 - skin irritation
 - adverse impact on blood circulation

Parts description

- 1 Cuff
- 2 Cuff tubing
- 3 Cuff plug
- 4 Connection for cuff plug
- 5 Risk indicator
- 6 User button for user selection
- 7 Function buttons \vee/\wedge
- 8 START-/STOP button
- 9 Memory button
- 10 Settings button

Notifications in the display:

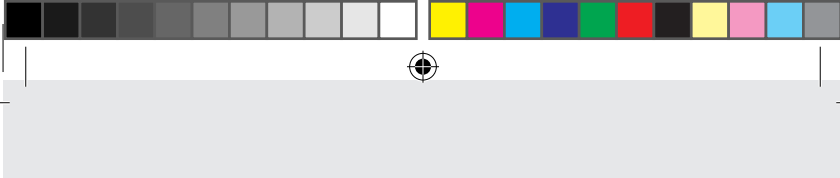
- 11 Time and date
- 12 Systolic pressure
- 13 Diastolic pressure
- 14 Risk indicator
- 15 Pulse value determined
- 16  Cardiac arrhythmia/ Pulse
- 17 User memory 
- 18 Resting indicator display 
- 19 Cuff position control 
- 20 Memory display: average value **AVG**, morning: , evening: 
- 21 Number of memory slots
- 22 Symbol for replace battery 

● Scope of delivery

- 1 upper arm blood pressure monitor
- 1 upper arm cuff with cuff tubing
- 4 batteries 1.5V, Type AAA LR03
- 1 set of instructions for use
- 1 quick start guide
- 1 blood pressure record card
- 1 storage bag

● Technical data

- Type: GCE614
- Model: SBM 70
- Measuring method: Oscillometric, non-invasive
blood pressure measurement on the upper arm
- Measurement range: Nominal cuff pressure: 0–300 mmHg,
Measuring pressure:
systolic: 50–280 mmHg

Display accuracy:	diastolic: 30–200 mmHg Pulse value: 40–199 Beats per minute systolic \pm 3 mmHg, diastolic \pm 3 mmHg, Pulse \pm 5 % of the display value
Measurement inaccuracy:	max. permissible standard deviation: according to clinical testing systolic 8 mmHg/ diastolic 8 mmHg
Memory:	2 x 120 memory spaces
Dimensions:	L 124 mm x B 95 mm x H 45 mm
Cuff:	suitable for an arm circumference between 22 and 42 cm
Weight:	approx. 358 g (with cuff, without batteries)
Permissible operating conditions:	+10 °C to +40 °C, < 90% relative humidity (non-condensing), 800–1050 hPa ambient pressure
Permissible storage conditions:	-20 °C to +55 °C, \leq 90% relative humidity (not condensing)
Power supply:	4 x 1.5V AAA batteries
Expected service life:	You can find information about your product's service life at www.sanitas-online.de
Battery life:	For approx. 225 measurements, depending on blood pressure level and inflation pressure/ <i>Bluetooth</i> [®]
Classification:	Internal supply, IP21, no AP or APG, continuous operation, applied part Type BF

Data transfer via *Bluetooth*[®]
low energy technology:

The product uses *Bluetooth*[®] low energy technology, frequency band 2.402–2.480 GHz, maximum transmission power radiated in the frequency band <20 dBm

A list of compatible smartphones, information about the “HealthForYou” app, and further details about the devices can be found on the following link:
www.healthforyou.app

System requirements for the
“HealthForYou” app:

iOS from version 14.0; Android™ from version 8.0; *Bluetooth*[®] from version 4.0

Sensor:

Pressure sensor in the motherboard for recording the user’s heart frequency and arterial blood pressure.

Please always keep the operating system of the smartphone you use updated.

The “HealthForYou” app provides 2-factor authentication. This consists firstly of a combination of an email address and password, selected by you. Secondly, you must confirm the smartphone that you use. The smartphone you use to register is confirmed automatically. More information and an instruction manual can be found in the app within the account section of your profile as well as in the FAQs at www.healthforyou.app.

The cloud is also an option for an additional backup. This means, for example, that data loss can be avoided if the smartphone is changed.

Technical specifications subject to change without notice to keep the product updated.

● **Warning and safety notes**

General warnings

- The measurements you take are only intended for your information - they do not replace a medical examination! Discuss your measured values with a doctor. Under no circumstances should you make medical decisions by yourself on the basis of these values (e.g. with regard to medication dosages)!
- The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage caused by improper or incorrect use.
- Using the blood pressure monitor outside of the home or while moving (e.g. whilst riding in cars, ambulances or helicopters as well as whilst practising physical activities such as sport) can impact the accuracy of the measurement and lead to measurement errors.
- Cardiovascular diseases may cause incorrect measurements and/or impair the device's measurement accuracy.
- Do not use the device at the same time as other medical electrical devices (ME devices). This could lead to a malfunction in the measurement device and/or an inaccurate measurement.
- Do not use the device outside of the stated storage and operating conditions. This could lead to incorrect measurement results.

- 
- 
- Only use the cuffs supplied with this device or described in these instructions for use. Using another cuff can lead to inaccurate measurements.
 - Ensure that the limb is not functionally impaired when inflating the cuff.
 - Do not take measurements more often than necessary. Due to the limitation to blood flow, it can lead to the development of bruising.
 - Blood circulation should not be restricted by the blood pressure monitor for an unnecessary amount of time. Remove the cuff from the arm if the device malfunctions.
 - Only place the cuff on the upper arm. Do not place the cuff on other body parts.
 - The air tube poses a risk of strangulation for infants. Furthermore, the small parts the product contains present a choking hazard for infants if swallowed. They should therefore always be supervised.
 - Do not allow the device to fall and also avoid stepping on or shaking the device.
 - Do not disassemble the device, as this can lead to damage, malfunctions and faults.
 - Before using the device under any of the following circumstances, it is absolutely necessary that you consult a doctor: cardiac arrhythmias, circulation problems, diabetes, hypertension, chills, tremors.
 - To exclude a difference between sides, measurements should be taken on both arms at first.
- 
- 

General precautions

-  **ATTENTION!** Potential damage to the device. The blood pressure monitor is made up of precision components and electronic components. The accuracy of the measurements and the service life of

the device depend on the device being used with a due amount of care.

- Protect the device from impacts, moisture, dirt, severe temperature fluctuations and direct sunlight.
- Bring the device to room temperature before taking measurements. If the measuring device has been stored near to the maximum or minimum storage and transport temperature and is brought into an environment with a temperature of 20 °C, it is recommended that you wait approx. 2 hours before using the measuring device.
- Avoid mechanically restricting, compressing or kinking the cuff tubing.
- Do not use the device in the vicinity of strong electromagnetic fields, keep it away from radio equipment and mobile telephones.
- Remove the batteries from the device if it will not be used over a longer period of time.

Battery safety information

- **DANGER TO LIFE!** Keep batteries out of the reach of children. Seek immediate medical attention if swallowed!
-  **EXPLOSION HAZARD!** Never recharge disposable batteries. Never short-circuit and/or open batteries. This could cause batteries to overheat, catch fire or explode.
- Never throw batteries into fire or water.
- Never exert mechanical stress on batteries.

Risk of battery leakage

- Avoid extreme conditions and temperatures which may affect batteries (e.g. radiators / direct sunlight).
- If batteries have leaked, do not allow the chemicals to come into contact with skin, eyes or mucous membranes! Flush the affected areas immediately with clean water and seek medical attention!
-  **WEAR SAFETY GLOVES!** Leaking or damaged batteries can cause burns upon contact with the skin. Therefore wear suitable protective gloves in this case.
- In the event of a battery leak, immediately remove the batteries from the product to prevent damage.
- Only use the same type of batteries. Do not mix old and new batteries!
- Remove the batteries if the product will not be used for an extended period.

Risk of product damage

- Only use the specified battery type!
- Insert the batteries according to the polarity marking (+) and (-) on the battery and the product.
- Clean the contacts on the battery and inside the battery compartment with a dry cloth before inserting the battery!
- Immediately remove drained batteries from the product.
- Do not use rechargeable batteries.

Information on electromagnetic compatibility

- The device is suitable for use in all environments that are listed in these instructions for use, including in the home.

- 
- 
- In the presence of electromagnetic disturbances, the device may under some circumstances only be useable to a limited extent. This can result in error messages or a malfunction of the display / device, for example.
 - The use of the device immediately next to other devices or stacked with other devices is to be avoided, as this can result in faulty operation. If it is necessary to use the device in such a way in spite of this, this device and the other device should be monitored to make sure that they are working correctly.
 - The use of equipment other than that which has been specified or made available by the manufacturer of this device can lead to increased electromagnetic interference or the reduced electromagnetic stability of the device and can lead to faulty functioning.
 - Keep portable RF communication devices (including peripheral equipment such as antenna cables or external antennae) at least 30 cm away from all device parts, including all cables supplied.
 - Failure to observe these instructions may lead to a reduction in the performance features of the device.
- 
- 

● **Prior to first use**

- Check that all parts listed in the scope of delivery are present.
- Remove all packaging material.

● **Inserting batteries**

- Remove the cover of the battery compartment on the back of the device (see Fig. C).
- Insert the batteries (see the section “Technical Information”). Insert batteries as indicated and observe correct polarity (see Fig. C).
- Close the battery cover.

If the  symbol appears permanently, it is no longer possible to take a measurement. Replace all batteries. As soon as the batteries have been removed from the device, you will need to reset the date and time. The saved measurements will not be lost.

● “HealthForYou” app

Download the free app “HealthForYou” from the Apple App Store and Google Play.

- Follow the installation instructions on the screen.

The “HealthForYou” privacy policy can be viewed via the following link:
www.healthforyou.app/privacy.html

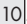
● Settings

- Correctly adjust the device settings before use to access all functions. This is the only way that your measurements can be saved with their time and date and recalled later.
- The settings menu can be accessed in two different ways:

1. Before first use and after each time the batteries are replaced:

When you insert batteries into the device, you will automatically arrive at the relevant menu.

2. If batteries have already been inserted:

- On the switched-on device, press and hold the settings button   for approx. three seconds.
- Adjust the following settings one after the other:**
Hour format -> Date -> Time
- Confirm each setting by pressing the settings button  .

Time

The time format flashes:

- Use the function buttons \vee/\wedge $\boxed{7}$ to set the time format.

Date

The year flashes:

- Use the function buttons \vee/\wedge $\boxed{7}$ to set the year.

The month flashes.

- Use the function buttons \vee/\wedge $\boxed{7}$ to set the month.

The day flashes.

- Use the function buttons \vee/\wedge $\boxed{7}$ to set the day.

Note: If the time format is set to 12-hour clock, the order of the day and month is reversed.

Clock

The hours flash:

- Use the function buttons \vee/\wedge $\boxed{7}$ to set the hours.

The minutes flash.

- Use the function buttons \vee/\wedge $\boxed{7}$ to set the minutes.

● Bluetooth®

The *Bluetooth*® symbol in the display flashes.

- Using the function buttons \vee/\wedge $\boxed{7}$, choose whether you want to activate *BLE ON* or deactivate *BLE OFF* data transfer and confirm your selection using the settings button $\boxed{10}$.

Note: The battery life is reduced by transfer via *Bluetooth*®.

● Measuring blood pressure

Note: To generate an informative profile of the development of your blood pressure and to thus guarantee that you will be able to compare the measured values adequately, measure your blood pressure regularly and always at the same time of day. It is


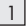
recommended that you measure blood pressure twice a day: once in the morning after getting up and once in the evening.

- You should not eat, drink, smoke or be physically active in the 30 minutes before each measurement.
 - Before taking the first blood pressure measurement, always rest for 5 minutes!
 - The measurement should always be taken in an adequate state of physical rest. You should therefore avoid taking measurements at stressful times.
 - Furthermore, if you wish to take multiple measurements one after the other, always wait at least 1 minute between each measurement.
 - Repeat the measurement process if the measured values seem incorrect.
 - Bring the device to room temperature before taking measurements.
 - Only place the cuff on the upper arm. Do not place the cuff on other body parts.
 - The blood pressure monitor can only be operated with batteries. Please be aware that data transfer and data storage are only possible if your blood pressure monitor is powered on. If the batteries are drained, the date and time will be deleted from the blood pressure monitor.
- 
- 

● **Attaching the cuff**

 **ATTENTION!** Only use the device with the original cuff. The cuff is suitable for an upper arm circumference between 22 and 42 cm. Ensure correct fit before the measurement is taken using the index markings described below.

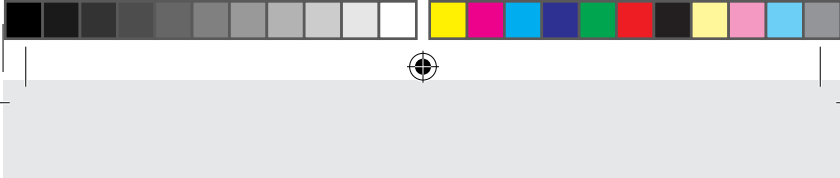- Place the cuff  on a bare upper arm (fig. D). The blood flow of the arm should not be constricted by clothing etc. which is too tight.

- Position the cuff **1** on the upper arm so the lower edge is 2–3 cm up from the elbow and over the artery. The cuff tubing **2** points towards the middle of the palm.
- Place the free end of the cuff **1** closely, but not too tightly, around the arm and close the hook and loop fastener (Fig. E). The cuff **1** should not be too tight. Check that two fingers can still fit beneath the cuff **1** (Fig. F).
- This cuff is suitable for you if, after putting it on, the index mark on the outside of the cuff (▼) is within the OK range on the cuff (Fig. G).
- Now insert the cuff tubing **2** with the cuff plug **3** into the connection **4** (Fig. H).
- If the measurement is being taken using the right upper arm, the air tube should be over the inner side of your elbow. Ensure that you do not press on the air tube with your arm.

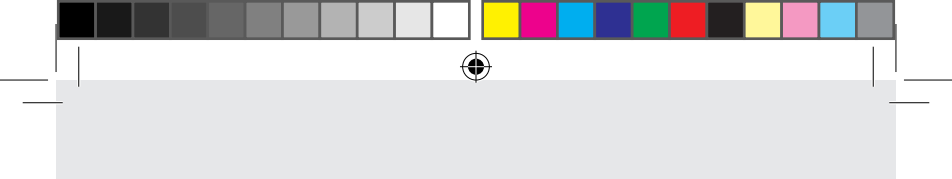
● Adopting the correct posture

Note: Stay still during the measurement process and do not talk to prevent the measurement result from being falsified.

- You can carry out the measurement while sitting or lying down. Always ensure that the cuff **1** is at the same level as the heart (Fig. I–K).
- Sit comfortably for the blood pressure measurement. Use your chair's back and arm rests. Do not cross your legs. Put your feet flat on the ground.
- To prevent falsifying the measurement result, it is important that you stay still and do not talk.

● User selection

This device has two user options with 120 memory spaces each to enable the separate storage of measurements taken by two different

persons. If the device is used by multiple users, ensure that the correct user is selected before each measurement:

- Use the user button **6** to select the desired user.

● **Measuring**

- Press the START/STOP button **8**. All display elements are shown briefly. The start display greets the selected user **1** or **2**. From this start display, you can access all menu points, e.g. user memory.
- Press the START/STOP button **8** again to start measuring. The cuff **1** inflates automatically. The measurement process starts. The pulse symbol **16** is displayed as soon as a pulse has been detected.

To cancel measuring, press the START/STOP button **8**.

The measurement results for systolic pressure, diastolic pressure and pulse will be displayed. The symbol indicating cuff fit control **19** is displayed for the entire duration of the measurement. If the cuff **1** is too loose, **Er** is displayed. In this case, measurement stops after approx. 15 seconds and the device switches off.

Er appears if the measurement has not been carried out correctly. In this case, please refer to the "Troubleshooting" section.

- If desired, re-attach the cuff **1** after 1 minute.

The device automatically switches off after 60 seconds. The value is stored for the last user that was selected or used.

● **Transferring measurements via Bluetooth®**

You have the option of transferring the measured values and values saved on your device to your smartphone using *Bluetooth®* low energy technology.

- Activate *Bluetooth*[®] in your smartphone settings and open the "HealthForYou" app.
Note: The "HealthForYou" app must be open for transfer to occur.
- Add "SBM 70" in the app.
- When connecting for the first time, a randomly generated six-digit PIN code will be displayed on the device in one line. At the same time, an entry field will appear on your smartphone where you should enter this six digit code. When you have entered it correctly, the device is connected with your smartphone and you can begin to transfer your measurement values.

⚠ WARNING! The displayed values are intended merely as a visualisation and should not be used as the basis for therapeutic measures. The software is not part of a diagnostic medical system. We wish to expressly indicate that the software at hand is not a medical product in accordance with the EU directive 93/42/EEC.
Note: During data transfer, button functions cannot be executed. To transfer measurements to your smartphone via *Bluetooth*[®] proceed as follows:

Manual transfer

Go to save mode (see "Saving, requesting and deleting measurements"). As soon as the average value of all measurements is displayed, a *Bluetooth*[®] connection will automatically be established. The data will be transferred and the *Bluetooth*[®] symbol will be shown on the display.

- Switch the blood pressure monitor off by pressing the START/STOP button 8.

Automatic transfer

If you forget to switch off the blood pressure monitor, it will automatically switch off after one minute. In this case, the value will be saved in the selected user memory and the data will be transferred if *Bluetooth*[®]

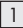
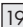
data transfer has been activated. The *Bluetooth*[®] symbol will be shown on the display during transfer.

● Evaluating measurements

● **General information about blood pressure**

- Blood pressure is the force with which the bloodstream presses against the arterial walls. Arterial blood pressure changes continuously over the course of a cardiac cycle.
- The blood pressure figure is always given in the form of two values.
 - The highest pressure in the cycle is called the systolic blood pressure. It occurs when the heart muscle draws together and presses the blood into the vessels.
 - The lowest number is the diastolic blood pressure, which then occurs when the heart muscle has fully stretched out again and the heart fills with blood.
- Blood pressure fluctuations are normal. Even on a repeat measurement, observable differences between the measured values can arise. One-time or irregular measurements therefore do not provide a reliable indication of the actual blood pressure. A reliable assessment is only possible if you regularly measure your blood pressure under comparable conditions.

● **Cuff position control**

The device has a cuff position control. Using this function, the device automatically checks the correct position of the cuff on your upper arm whilst it is taking a measurement. If the cuff  has been correctly positioned, the symbol   will appear in the display during inflation. It will stay there until the measurement has successfully been taken and will then be shown together with the recorded values.

● Introduction to cardiac arrhythmia

This device can identify irregular heartbeats and, if applicable, indicates this after the measurement process with the symbol . This can be an indicator for irregular heartbeat.

Arrhythmia is an illness in which the heart rhythm is abnormal due to errors in the bioelectric system, which controls the heartbeat. The symptoms (skipped or premature heartbeats, slow or rapid pulse) can stem from heart conditions, age, physical predispositions, an excess of luxury foods, stress or a lack of sleep. Arrhythmia can only be diagnosed by a doctor after a physical examination.

- Repeat the measurement process if the symbol  is shown on the display after the measurement. Please make sure that you rest for one minute before the measurement and do not talk or move during it.

 If the symbol  is displayed repeatedly, contact your doctor. Self-diagnosis and self-treatment on the basis of measurement results can be dangerous. Follow your doctor's instructions.

● Risk indicator

In accordance with the guidelines/definitions provided by the World Health Organisation (WHO) and the latest findings, the measurement results can be classified and evaluated according to the following table:

Range of blood pressure values	Colour of the Risk-Indicator	Systole (in mmHg)	Diastole (in mmHg)	Remedies
Stage 3: severe hypertension	red	≥ 180	≥ 110	seek medical attention

Range of blood pressure values	Colour of the Risk-Indicator	Systole (in mmHg)	Diastole (in mmHg)	Remedies
Stage 2: moderate hypertension	orange	160-179	100-109	seek medical attention
Stage 1: mild hypertension	yellow	140-159	90-99	regular monitoring by a doctor
High, normal	green	130-139	85-89	regular monitoring by a doctor
Normal	green	120-129	80-84	Self-monitor
Ideal	green	< 120	< 80	Self-monitor

Source: WHO, 1999

Note: These standard values merely serve as a general guideline as blood pressure deviates depending on the individual, different age groups, etc.

It is important that you consult your doctor at regular intervals. Your doctor will tell you your particular values for normal blood pressure as well as the value for when your blood pressure is classified as dangerous.

The bar chart on the display and the scale on the device state which range the blood pressure measured is in.

If the systole and diastole values are in two different ranges (i.e. systole is "high, normal" and diastole is "normal") the graphic categorisation

on the device will always show the higher range i.e. in the above example this would be “high, normal”.

● Resting indicator (through HSD diagnostic)

One of the most frequent errors made while measuring blood pressure is that the user’s circulatory system is not sufficiently at rest. In this case, the measured systolic and diastolic blood pressure value do not indicate the resting pressure, which could be used to assess the measured values.

This blood pressure monitor uses integrated Hemodynamic Stability Diagnostic (HSD) to measure the user’s hemodynamic stability while the device is in use and to then provide information whether the blood pressure was measured while the circulatory system was sufficiently at rest.

	The measured blood pressure was collected with sufficient circulatory rest and reflects the resting blood pressure of the user with sufficient certainty.
	There is a notification regarding a lack of circulatory rest. In this case, the measured blood pressure values do not reflect the resting blood pressure. For this reason, measurements should be repeated after a physical and mental rest time of at least 5 minutes.
No rest indicator symbol appears.	The measurement could not determine whether sufficient circulatory rest was achieved. In this case, a rest period of at least 5 minutes should be repeated.

Insufficient circulatory rest can have different causes, such as physical strain, mental tension/distraction, speaking, or arrhythmia occurring while measuring.

In the majority of usage cases, the HSD provides excellent information whether circulatory rest is achieved while measuring blood pressure. However, certain patients with arrhythmias or ongoing mental strain may remain consistently hemodynamically unstable - which also applies after repeated rest phases. For these users, the accuracy of determining the resting blood pressure is limited.

As any other medical measurement technique, HSD has limited accuracy and may result in incorrect results in individual cases. However, blood pressure measurements determined at sufficient circulatory rest present particularly reliable results.

● Reviewing and deleting measurements

Users

The results of each successful measurement are saved with the date and time. If there are over 120 measurement values, the oldest measurement value will be deleted.

- Press the memory button  **9** in the start display and select the desired user via the user button **6**.

Average value

AVG is displayed:

The average of **all saved measurements** for this user is displayed.

- Press the function button **^** **7**.

 **AVG** is displayed:

Average of the last 7 days of **morning measurements** is displayed (morning: 5:00 a.m. - 9:00 a.m.).

- Press the function button **^** **7**.

 **AVG** is displayed:

Average of the last 7 days of **evening measurements** is displayed (evening: 06:00 p.m. - 08:00 p.m.).

Individual measurements

- If you press the function button \wedge [7] again, the display will show the last individual measurement.
- If you push the function button \vee/\wedge [7] again, you can review all individual measurement.
- To switch the device back off, press the START/STOP button O [8] for approx. 2 seconds.
- To exit the menu, briefly press the START/STOP button O [8].

Deleting measurements

- To delete all measurements stored for a user, please go to the corresponding user memory.
The display shows **AVG** and the average of all stored measurements of a user.
- Press and hold the function buttons \vee and \wedge [7] for approx. 5 seconds.
The display will show E L 0 1 for M 1 / E L 0 2 for M 2 . All measurements for a selected user will be deleted. The device will automatically switch off.

Restoring the device to its factory settings

- To delete all measurements and settings, select user memory.
- Press the function button \wedge [7]. The display will show **AVG**.
- Press and hold the function buttons \vee and \wedge [7] for approx. 15 seconds.
The display will show E L . Now all of the data saved on the device are deleted and the device is restored to its factory settings.

The device will automatically switch off.

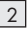
Note: If the device has been reset to factory settings, then it must be removed from *Bluetooth*[®] settings on your smartphone in order to re-connect it.

● Cleaning and care

-  **ATTENTION! POSSIBLE PROPERTY DAMAGE!** Never hold the product under water as liquid could enter the product and damage it.
- Clean the device and cuff  carefully using a slightly damp cloth.
 - Do not use cleaners or solvents.

● Storage

-  **ATTENTION! POSSIBLE PROPERTY DAMAGE!** Do not place any heavy objects on the device and cuff .
- If the product will not be used for more than one month, remove both batteries from the device to prevent possible battery leakage.
 - Do not kink the cuff tubing .
 - Store the product in a dry location (relative humidity $\leq 90\%$). Excessive humidity can shorten the service life or damage the product. Store the product in a location with ambient temperatures between $-20\text{ }^{\circ}\text{C}$ and $+55\text{ }^{\circ}\text{C}$.

● Accessories and spare parts

Accessories and spare parts are available via the corresponding service address. Be sure to specify the corresponding order code when ordering.

Designation	Article number or order number
Universal cuff (22-42cm)	164.492

● Troubleshooting

Error message	Problem	Solution
Er1	Pulse could not be measured correctly.	Please repeat the measurement after a one-minute break. Please refrain from talking or moving during the measurement process.
Er2	The measured blood pressure is outside of the measuring range.	
Er3 	A pneumatic system error has occurred.	Repeat the measurement. Ensure that the cuff tubing has been connected correctly; refrain from moving and talking.
Er4	An error occurred during the measurement process.	Please repeat the measurement after a one-minute break. Please refrain from talking or moving during the measurement process.
Er5	The inflation pressure is more than 300 mmHg.	Please complete a new measurement to check whether the cuff can be properly inflated. Ensure that neither your arm, nor heavy objects are placed on the cuff tubing and that the tube is not kinked.
Er6	A system error has occurred.	If this error message appears, please contact customer service.

Error message	Problem	Solution
Er7	Problems with the connection between the smartphone/tablet and the app.	Switch off the main unit, close the app, and briefly deactivate Bluetooth® on your smartphone/tablet before re-activating it. Try to re-establish the connection.
 LD	The batteries are almost dead.	Insert new batteries into the device.

● Repairing and disposing of the device

- Do not repair or adjust the device yourself. If you do so, proper function is no longer guaranteed.
- Do not open the device, except for the battery compartment. Failure to comply with this will void the warranty.
- Repairs may only be undertaken by customer service or authorised specialists. Before making a complaint, first check and replace the batteries.

● Disposal

The packaging is made from environmentally-friendly materials which can be disposed of through your local recycling facilities.

Observe the markings on the packaging materials when sorting the waste. These are marked with abbreviations (a) and numbers (b) with the following meanings: 1–7: plastics / 20–22: paper.

The device may not be disposed of with household waste. You may dispose of it using appropriate collection sites in your country. Dispose of the product according to the EC

Waste Electrical and Electronic Equipment Directive - WEEE.
Please contact your responsible local authority with any questions.

The product incl. accessories and packaging materials are recyclable and are subject to extended producer responsibility. Dispose them separately, following the illustrated Info-tri (sorting information), for better waste treatment. The Triman logo is valid in France only.

User notices for the disposal and scrapping of electronic equipment waste in Germany

In this context, owners of electronic waste can return it via return or collection options for electronic waste provided by public disposal authorities to ensure the proper disposal of electronic waste. Returning these items is a legal requirement.

Furthermore, it is also possible to return the devices to distributors if certain requirements are met.

According to the Electrical and Electronic Equipment Act (ElektroG), the following distributors are required to accept electronic waste free of charge:

- Electronics stores with a retail space for electrical and electronic devices of at least 400 square metres
- Grocery stores with a total retail space of at least 800 square metres, who offer and sell electrical and electronic devices multiple times per year or permanently.
- Mail-order businesses, however, the requirement for 1:1 returns for private households only applies to heat exchangers (cooling/

refrigeration appliances, air conditioners, etc.), devices with screens and large appliances. For 1:1 returns of lamps, small devices, and small computer or telecommunication devices, as well as 0:1 returns, mail-order retailers must provide users with return options within a reasonable distance.

When furnishing a user with a new electrical or electronic device, these distributors are required

- to accept electronic waste of the same type, which fundamentally fulfils the same functions as the new device, at the location of the provision, or in the immediate vicinity, free of charge and
- upon the request of the end user, electronic waste, with external dimensions that are no larger than 25 centimetres at the retail location or its immediate vicinity; the return may not require the purchase of an electrical or electronic device and is limited to three electronic waste devices per device type.

If the products contain batteries or rechargeable batteries or light bulbs that can be removed from the electronic waste device without destroying it, then these must be removed before disposal and disposed of separately as a battery or light bulb.

If the electrical or electronic waste device contains personal data, you are responsible for erasing this data before returning it.

Defective or used batteries must be recycled according to Directive 2006/66/EC and its amendments. Return the batteries and/or the product to the recycling facilities provided.

Incorrect disposal of batteries can harm the environment!

Never dispose of batteries in your household waste. They may contain toxic heavy metals and are subject to hazardous waste regulations. The chemical symbols of the heavy metals are: Cd = cadmium, Hg = mercury, Pb = lead. Therefore, you should dispose of used batteries through your local collection facilities.

● Warranty / service

Hans Dinslage GmbH, Riedlinger Straße 28, 89524 Uttenweiler (hereafter referred to as "HaDi") provides a guarantee for this product under the following prerequisites and the product warranty described hereafter.

In case of defects, warranty claims do not affect the legal warranty rights of the customer which emerge from the purchase contract. The customer may exercise their legal warranty rights free of charge. Furthermore, the warranty also applies in spite of obligatory legal liability provisions.

HaDi guarantees the fault-free function and completeness of this product.

The worldwide warranty is 3 years starting with the purchase of the new, unused product by the customer.

This warranty only applies to products that the customer has purchased and used for personal purposes in domestic contexts as a consumer. Subject to German law.

If the product proves to be incomplete or defective in terms of its function during the warranty period and according to the following

stipulations, HaDi will provide a free replacement or repair in accordance with these warranty conditions.

If the customer wishes to make a warranty claim, they should first contact HaDi customer service:

Service Hotline (toll-free):

 Tel.: 0800 931 0319

 Tel.: 1800 939 235

Email address:

 service-uk@sanitas-online.de

 service-ie@sanitas-online.de

Then, the customer will receive additional information about the processing of the warranty claim, e.g. where to send the product free of charge and which documents are required.

If we ask the customer to return the defective product, please send it to the following address:

NU Service GmbH
Lessingstraße 10b
89231 Neu-Ulm
GERMANY

Warranty claims are only considered if the customer can submit

- a copy of the invoice/receipt and
 - the original product
- to HaDi or an authorised HaDi partner.

The following is not covered by this warranty

- wear, which is the result of normal product use

- 
- 
- accessories included with this product, which become worn while used as intended (e.g. batteries, cuffs, seals, electrodes, illuminants, attachments, inhaler accessories);
 - products that have been used improperly and/or used, cleaned, stored, or serviced in violation of the guidelines of the instructions for use, as well as products that were opened, repaired, or retrofitted by the customer or a service centre not authorised by HaDi.
 - damage that occurs in transit from the manufacturer and the customer or between the service centre and the customer;
 - products purchased used or as factory seconds;
 - consequential damage based on a defect of this product (in this case, however, only product liability claims or other mandatory liability provisions may arise).

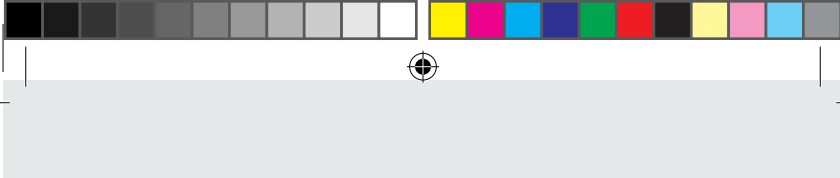
Repairs or replacements never extend the warranty period.

This device conforms to European standard EN 60601-1-2 (Group 1, Class B, in accordance with CISPR 11, IEC 61000-3-2, IEC 61000-3-3, IEC 61000-4-2, IEC 61000-4-3, IEC 61000-4-4, IEC 61000-4-5, IEC 61000-4-6, IEC 61000-4-7, IEC 61000-4-8, IEC 61000-4-11) and is subject to special precautions regarding electromagnetic compatibility. Please note that portable and mobile RF communication devices may interfere with this device.

The accuracy of this blood pressure monitor has been thoroughly tested and was developed with regard to a long utilizable service life. If the device is used in medicine, technical inspections using the suitable equipment are to be carried out. Precise information on how to test the accuracy of the device can be requested from our service address.

Note on reporting incidents

For users / patients in the European Union and identical regulation systems (Medical Device Regulation, MDR (EU) 2017/745)), the following applies: if, during or as a result of the application of the product, a serious incident occurs, report this to the manufacturer and/ or their authorised representative and to the relevant national authority of the member state, where the user/patient is.

We hereby confirm that this product complies with the European RED Directive 2014/53/EU.

The CE declaration of conformity for this product is available at:

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The *Bluetooth*[®] word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Hans Dinslage GmbH is under license. Other trademarks and trade names are those of their respective owners.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries.

Google Play and the Google Play logo are trademarks of Google LLC. Android is a trademark of Google LLC.

Globalcare Medical Technology Co., Ltd
7th Building, 39 Middle Industrial Main Road European
Industrial Zone, Xiaolan Town 528415 Zhongshan City,
Guangdong Province P.R.C

Donawa Lifescience
Piazza Albania 10, 00153 Rome, Italy

Imported by:
EU: Hans Dinslage GmbH,
Riedlinger Str. 28,
88524 Uttenweiler, Germany

CE 0123