List of pictograms used	Page 7
Introduction	Page 11
Intended use	Page 12
Intended purpose	Page 12
Target group	Page 12
Clinical use.....	Page 12
Indication	Page 13
Contraindication	Page 14
Description of parts	Page 15
Scope of delivery.....	Page 16
Technical data	Page 16
Warning and safety notes	Page 19
General warnings.....	Page 19
General precautions.....	Page 22
Battery safety information	Page 23
Information on electromagnetic compatibility.....	Page 25
Prior to first use	Page 27
Inserting batteries	Page 27
Settings.....	Page 28

Measuring blood pressure..... Page 30

Attaching the cuff..... Page 31

Adopting the correct posture Page 32

User selection..... Page 32

Measuring..... Page 33

Evaluating measurements..... Page 34

General information about blood pressure Page 34

Cuff position check..... Page 35

Introduction to cardiac arrhythmia Page 36

Risk indicator..... Page 37

Blood pressure too low Page 39

Reviewing and deleting measurements..... Page 40

Cleaning and care..... Page 42

Storage..... Page 42

Troubleshooting..... Page 43

Repairing the device..... Page 44

Disposal..... Page 45

Warranty/service..... Page 47

List of pictograms used

WARNING

Warning of risks of injury or health hazards

ATTENTION

This symbol indicates that caution should be exercised if devices or controllers are being operated near this symbol or that the current situation requires attentiveness or intervention from the operator to avoid unintended consequences.

Product information
Important safety notes

Instructions for use

Observe the instructions

Read the instructions before starting to work and/or operating devices or machines.

Direct current

This device is only suitable for direct current.

List of pictograms used

	Temperature range
	Humidity range
	Air pressure limit
	Manufacturer
	Authorised EU representative for manufacturers of medical devices
	EXPLOSION HAZARD! Never recharge disposable batteries. Never short-circuit and/or open batteries.
	WEAR SAFETY GLOVES! Leaking or damaged batteries can cause burns upon contact with the skin. Wear suitable protective gloves.

List of pictograms used

Importer

Isolation of type BF applied parts

Serial number

CE mark

This product meets the requirements of the applicable European and national directives.

Do not dispose of batteries containing hazardous substances with household waste.

IP22

Device protected against foreign objects ≥ 12.5 mm and against water falling diagonally.

Disposal according to the EC Waste Electrical and Electronic Equipment Directive (WEEE).

List of pictograms used

	Labelling to identify the packaging material.
	Dispose of the packaging and blood pressure monitor in an environmentally friendly manner!
	Medical product
	Unique Device Identifier (UDI) Unique identifier for product identification
	Batch reference
	Item number
	Model number
	Date of manufacture

List of pictograms used

Remove all packaging from the product, then dispose of the packaging in accordance with the local regulations.

Dispose of paper and cardboard packaging in the blue recycling container.

Wrist Blood Pressure Monitor

● Introduction

Read these instructions for use carefully. Observe the warnings and safety instructions.

Keep these instructions for use for future reference. Make these instructions for use accessible to other users. Also include these instructions for use when passing the device on to others.

● **Intended use**

● **Intended purpose**

The blood pressure monitor is intended for the fully automatic, non-invasive measurement of arterial blood pressure and pulse frequency at the wrist.

It is designed for the purpose of self-monitoring by adults in domestic environments.

● **Target group**

No specific knowledge and no specialist expertise is required to use the device. With the exception of those patients who require special support, patients can use the device by themselves.

Blood pressure monitoring is suitable for adult users whose wrist circumference is within the area printed on the cuff 2.

● **Clinical use**

The user can quickly and easily record their blood pressure and pulse values using the device. The recorded blood pressure measurements are classified in accordance

with internationally applicable guidelines and evaluated graphically. Furthermore, the device can identify irregular heartbeats that occur while measuring and inform the user via a symbol on the display. The device saves the displayed measurements and can also provide average values of previous measurements. The data recorded can support healthcare providers when diagnosing and treating blood pressure problems and thus contribute to the long-term monitoring of the user's health.

● Indication

If the user has hypertension or hypotension, they can monitor their blood pressure, pulse values and potential irregular heartbeats independently at home. However, the user does not have to suffer from hypertension or irregular heartbeats to use the device.

Contraindication

- Do not use this product on newborns or pre-eclampsia patients. We recommend consulting your doctor before using the product if you are pregnant.
- The device is not suitable for use by patients with implanted electrical devices such as pacemakers or defibrillators. Do not use the device if you have metal implants.
- People with limited physical, sensory or mental abilities should be supervised by a person responsible for their safety and adhere to this person's instructions for using the device.
- Do not place the cuff **2** over wounds, as this could lead to further injuries.
- Ensure that the cuff **2** is not placed on the arm of a person whose arteries or veins are undergoing medical treatment, i.e. intra-vascular access or intra-vascular therapy or an arteriovenous (A-V) shunt.

- Undesirable side-effects:
 - Skin irritation
 - Adverse impact on blood circulation

● Description of parts

- 1 Battery compartment cover
- 2 Cuff
- 3 Function button >
- 4 START-/STOP button ⓘ
- 5 Settings/memory button ⚙️

Information shown on the display (Fig. B):

- 6 Risk indicator
- 7 Systolic pressure
- 8 Diastolic pressure
- 9 Pulse value determined
- 10  Cardiac arrhythmia /  Pulse
- 11 User memory  1  2
- 12 Cuff position check 
- 13 Memory display: average value **AVG**, morning , evening 

14 Number of the memory slot 

15 Battery indicator 

16 Time

17 Date

● Scope of delivery

- 1 wrist blood pressure monitor
- 2 batteries 1.5 V, type AAA LR03
- 1 set of instructions for use
- 1 blood pressure record card
- 1 storage box

● Technical data

Type: SBC 35

Model: SBC 39

Measuring method: oscillometric, non-invasive
blood pressure measurement
at the wrist

Measurement range: nominal cuff pressure:
299 mmHg,
Measuring pressure:
systolic: 60–260 mmHg

diastolic: 40–200 mmHg
pulse value: 30–180 beats
per minute
Display accuracy:
systolic ± 3 mmHg,
diastolic ± 3 mmHg,
pulse $\pm 5\%$ of the displayed
value

Measurement
inaccuracy:

max. permissible standard deviation according to clinical testing:
systolic 8 mmHg/
diastolic 8 mmHg

Memory:
Dimensions:
Cuff:

2 x 120 memory slots
82 x 62 x 59 mm (without cuff)
suitable for a wrist circumference
of between 13.5 and 21.5 cm
approx. 92 g (with cuff, without
batteries)

Weight:

Permissible

operating conditions: +10 °C to +40 °C, 15-93 %
relative humidity (non-condens-
ing), 800–1060 hPa ambient
pressure

Permissible storage
conditions:

-25 °C to +55 °C, ≤ 93 % rel-
ative humidity (non-condensing)

Power supply:

2 x 1.5V AAA batteries

Expected

product service life: you can find information about
your product's service life at
www.sanitas-online.de

Battery life:

for approx. 300 measurements,
depending on blood pressure
level and inflation pressure

Classification:

internal supply, IP22, no AP or
APG, continuous operation,
applied part Type BF

Technical specifications subject to change without no-
tice due to updates.

● **Warning and safety notes**

General warnings

- The measurements you take are intended only for your information. They are no substitute for a medical examination! Discuss your measured values with a doctor. Under no circumstances should you make medical decisions by yourself on the basis of these values (e.g. with regard to medication dosages)!
- The device is intended only for the purpose described in these instructions for use. The manufacturer is not liable for damage caused by improper or incorrect use.
- Using the blood pressure monitor outside of the home or while moving (e.g. whilst riding in cars, ambulances or helicopters as well as during physical activities such as sport) can affect the accuracy of the measurement and lead to measurement errors.

- Cardiovascular diseases may cause incorrect measurements and/or impair the device's measurement accuracy.
- Do not use the device at the same time as other medical electrical devices (ME devices). This could lead to a malfunction in the measurement device and/or an inaccurate measurement.
- Do not use the device outside of the stated storage and operating conditions. This could lead to incorrect measurement results.
- Only use the cuffs supplied with this device or described in these instructions for use. Using another cuff can lead to inaccurate measurements.
- Note that the limb concerned may be functionally impaired when inflating the cuff 2.
- Do not take measurements more often than necessary. Due to the restriction of blood flow, this can lead to bruising.
- Blood circulation should not be restricted by the blood pressure monitor for an unnecessarily long period of time. Remove the cuff 2 from the arm if the device malfunctions.

- Only place the cuff 2 on the wrist. Do not place the cuff 2 on other body parts.
- Keep children away from the packaging material. There is a risk of suffocation.
- Store out of the reach of children, pets and pests.
- Do not use the blood pressure monitor in conjunction with a high-frequency surgical unit.
- Do not use the device with a defibrillator.
- Do not use the device during an MRI scan.
- Do not expose the device to static electricity. Always ensure that you cannot be the source of an electrostatic discharge before you operate the device.
- Do not place the device in pressure receptacles or gas sterilisation devices.
- The small parts that the product contains present a choking hazard for infants if swallowed. They should therefore always be supervised.
- Do not allow the device to fall and also avoid stepping on or shaking the device.
- Do not disassemble the device, as this can lead to damage, malfunctions and faults.
- Do not make any modifications to the device.

- Before using the device under any of the following circumstances, it is absolutely necessary that you consult a doctor: cardiac arrhythmias, circulation problems, diabetes, hypertension, chills, tremors.
- To exclude a difference between sides, measurements should be taken on both arms at first.
- Never operate the device during maintenance. Maintenance consists of service, inspection and repair.

General precautions

ATTENTION! Potential damage to the device.

The blood pressure monitor is made up of precision components and electronic components. The accuracy of the measurements and the service life of the device depend on the device being used with a due amount of care.

- Protect the device from impacts, moisture, dirt, severe temperature fluctuations and direct sunlight.

- Bring the device to room temperature before taking measurements. If the measuring device has been stored near to the maximum or minimum storage and transport temperature and is brought into an environment with a temperature of 20 °C, it is recommended that you wait approx. 2 hours before using the measuring device.
- Avoid mechanically restricting or compressing the cuff 2.
- Do not use the device in the vicinity of strong electromagnetic fields; keep it away from radio equipment and mobile telephones.
- Remove the batteries from the device if it is not going to be used for a longer period of time.

Battery safety information

- **DANGER TO LIFE!** Keep batteries out of the reach of children. Seek immediate medical attention if swallowed!

-  **EXPLOSION HAZARD!** Never re-charge disposable batteries. Never short-circuit and/or open batteries. This could cause batteries to overheat, catch fire or explode.
- Never throw batteries into fire or water.
- Never exert mechanical stress on batteries.

Risk of battery leakage

- Avoid extreme conditions and temperatures which may affect batteries (e.g. radiators/direct sunlight).
- If batteries have leaked, do not allow the chemicals to come into contact with skin, eyes or mucous membranes! Flush the affected areas immediately with clean water and seek medical attention!
-  **WEAR SAFETY GLOVES!** Leaking or damaged batteries can cause burns upon contact with the skin. Wear suitable protective gloves.
- In the event of a battery leak, immediately remove the batteries from the product to prevent damage.

- Only use the same type of batteries. Do not mix old and new batteries!
- Remove the batteries if the product will not be used for an extended period.

Risk of product damage

- Only use the specified battery type!
- Insert the batteries according to the polarity marking (+) and (-) on the battery and the product.
- Clean the contacts on the battery and inside the battery compartment with a dry cloth before inserting the battery!
- Immediately remove drained batteries from the product.
- Do not use rechargeable batteries.

Information on electromagnetic compatibility

- The device is suitable for use in all environments that are listed in these instructions for use, including in the home.

- In the presence of electromagnetic disturbances, the device may under some circumstances only be useable to a limited extent. This can result in error messages or a malfunction of the display/ device, for example.
- The use of the device immediately next to other devices or stacked with other devices is to be avoided, as this can result in faulty operation. If it is necessary to use the device in the described manner in spite of this, this device and the other device should be monitored to make sure that they are working correctly.
- The use of equipment other than that which has been specified or made available by the manufacturer of this device can lead to increased electromagnetic interference or the reduced electromagnetic stability of the device and can lead to faulty functioning.
- Keep portable RF communication devices (including peripheral equipment such as antenna cables or external antennae) at least 30 cm away from all device parts, including all cables supplied.

- Failure to observe these instructions may lead to a reduction in the performance features of the device.

● **Prior to first use**

- Check that all parts listed in the scope of delivery are present.
- Remove all packaging material.

● **Inserting batteries**

- Remove the battery compartment cover **1** on the back of the device (see Fig. C).
- Insert the batteries (see the section “Technical data”). Insert the batteries as indicated and observe the correct polarity.
- Fit the battery compartment cover **1** again.

If the  symbol appears permanently, it is no longer possible to take a measurement. Replace all batteries. Once the batteries have been removed from the device or replaced, you will need to reset the date and time. The saved measurements will not be lost.

● Settings

- Correctly adjust the device settings before use to access all functions. This is the only way that your measurements can be saved with their time and date and recalled later.
- The settings menu can be accessed in two different ways:

1. Before first use and after each time the batteries are replaced:

When you insert batteries into the device, you will automatically arrive at the relevant menu.

2. If batteries have already been inserted:

- Switch on the device by pressing the START/STOP button ① **4**. Press and hold the settings/memory button  **5** until the 24-hour display flashes.
- Select the 12- or 24-hour display by pressing the function button > **3** and confirm your entry by pressing the settings/memory button  **5**.

Adjust the following settings one after the other:

Setting the date

The year flashes

- Select the year by pressing the function button > **3** and confirm your entry by pressing the settings/memory button **EQ** **5**.

The month flashes

- Select the month (1-12) by pressing the function button > **3** and confirm your entry by pressing the settings/memory button **EQ** **5**.

The day flashes

- Select the day (1-31) by pressing the function button > **3** and confirm your entry by pressing the settings/memory button **EQ** **5**.

Setting the time

The hour flashes

- Select the hour by pressing the function button > **3** and confirm your entry by pressing the settings/memory button **EQ** **5**.

The minute flashes

- Select the minute by pressing the function button > **3** and confirm your entry by pressing the settings/memory button **Eq** **5**.

● **Measuring blood pressure**

Note: To generate an informative profile of the development of your blood pressure and to thus guarantee that you will be able to compare the measured values adequately, measure your blood pressure regularly and always at the same time of day. It is recommended that you measure blood pressure twice a day: once in the morning after getting up and once in the evening.

- You should not eat, drink, smoke or be physically active in the 30 minutes before each measurement.
- Before taking the first blood pressure measurement, always rest for 5 minutes!
- The measurement should always be taken in an adequate state of physical rest. You should therefore avoid taking measurements at stressful times.

- Furthermore, if you wish to take multiple measurements one after the other, always wait at least 5 minutes between each measurement.
- Repeat the measurement process if the measured values seem incorrect.
- Only place the cuff  on the wrist. Do not place the cuff  on other body parts.

● Attaching the cuff

ATTENTION! Only use the device with the original cuff . The cuff  is suitable for a wrist circumference of between 13.5 and 21.5 cm.

- Now place the cuff  on your wrist so that the palm of your hand and the display of the device are facing upwards (Fig. D). The blood flow of the arm should not be constricted by clothing, etc. that is too tight.
- Position the cuff  so that a distance of 1.0-1.5 cm remains between the cuff and the ball of your hand.
- Now fasten the cuff  firmly around your wrist with the help of the hook-and-eye closure. Ensure that is tight, but doesn't constrict your wrist.

● Adopting the correct posture

Note: Stay still during the measurement process and do not talk to prevent the measurement result from being falsified.

- Sit comfortably for the blood pressure measurement. Use your chair's back and arm rests. Do not cross your legs. Put your feet flat on the ground.
- Place your arm on a support. The cuff 2 must be at the same height as your heart.
- To prevent falsifying the measurement result, it is important that you stay still and do not talk.

● User selection

This device has two user options with 120 memory spaces each to enable the separate storage of measurements taken by two different persons. If the device is used by multiple users, ensure that the correct user is selected before each measurement:

- Press the START/STOP button 1 4; the device switches on.

- The user symbol    flashes.
- Use the function button   to select the user.

● Measuring

- Press the START/STOP button  . All display elements are shown briefly. The start display greets the selected user  or  . From this start display, you can access all menu points, e.g. user memory.
- Press the START/STOP button   again to start measuring. The cuff  inflates automatically. The measurement process starts. The pulse symbol   is displayed as soon as a pulse has been detected. To cancel measuring, press the START/STOP button  . The measurement results for systolic pressure , diastolic pressure  and pulse  will be displayed. The symbol indicating cuff position check   is displayed for the entire duration of the measurement. If the cuff  is too loose,   is displayed. In this case, measurement stops after approx. 15 seconds and the device switches off.  appears if the measurement has not been

carried out correctly. In this case, please refer to the “Troubleshooting” section.

- If desired, re-attach the cuff **2** after 2 minutes.

The device automatically switches off after approx. 60 seconds. The value is stored for the last user that was selected or used.

● **Evaluating measurements**

● **General information about blood pressure**

- Blood pressure is the force with which the blood-stream presses against the arterial walls. Arterial blood pressure changes continuously over the course of a cardiac cycle.
- The blood pressure figure is always given in the form of two values.
 - The highest pressure in the cycle is called the systolic blood pressure **7**. It occurs when the heart muscle contracts and presses the blood into the vessels.

- The lowest pressure is the diastolic blood pressure [8], which then occurs when the heart muscle has fully expanded again and the heart fills with blood.

- Blood pressure fluctuations are normal. Even on a repeat measurement, observable differences between the measured values can arise. One-time or irregular measurements therefore do not provide a reliable indication of the actual blood pressure. A reliable assessment is only possible if you regularly measure your blood pressure under comparable conditions.

● Cuff position check

The device has a cuff position check . Using this function, the device automatically checks the correct position of the cuff [2] on your wrist whilst it is taking a measurement. If the cuff [2] has been correctly positioned, the symbol  [12] will appear on the display during inflation. It will stay there until the measurement has successfully been taken and will then be shown together with the recorded values.

● Introduction to cardiac arrhythmia

During the measurement, this device can identify irregular heartbeats and, if applicable, it indicates this after the measurement with the symbol  10. This can be an indicator for irregular heartbeat.

Arrhythmia is an illness in which the heart rhythm is abnormal due to errors in the bioelectric system, which controls the heartbeat. The symptoms (skipped or premature heartbeats, slow or rapid pulse) can stem from heart conditions, age, physical predispositions, an excess of luxury foods, stress or a lack of sleep. Arrhythmia can only be diagnosed by a doctor after a physical examination.

- Repeat the measurement process if the symbol  is shown on the display after the measurement. Please make sure that you rest for one minute before the measurement and do not talk or move during it.
- ⚠ If the symbol  is displayed repeatedly, contact your doctor. Self-diagnosis and self-treatment on the basis of measurement results can be dangerous. Follow your doctor's instructions.

● Risk indicator

In accordance with the guidelines/definitions¹ provided by the World Health Organisation (WHO) and the latest findings, the measurement results can be classified and evaluated according to the following table:

Range of blood pressure values	Colour of the Risk-Indicator	Systolic (in mmHg)	Dias-tolic (in mmHg)	Remedies
Stage 3: severe hypertension	red	≥ 180	≥ 110	seek medical attention
Stage 2: moderate hypertension	orange	160–179	100–109	seek medical attention
Stage 1: mild hypertension	yellow	140–159	90–99	regular monitoring by a doctor

Range of blood pressure values	Colour of the Risk-Indicator	Systolic (in mmHg)	Dias-tolic (in mmHg)	Reme-dies
High normal	green	130–139	85–89	regular monitoring by a doctor
Normal	green	120–129	80–84	Self-monitor
Ideal	green	< 120	< 80	Self-monitor
Blood pressure too low ²	orange	< 90	< 60	seek medical attention

¹Source: WHO. 1999 (World Health Organization)

²Source: National Health Service, 2023

Note: These standard values merely serve as a general guideline as blood pressure deviates depending on the individual, different age groups, etc.

It is important that you consult your doctor at regular intervals. Your doctor will tell you your particular values for normal blood pressure as well as the value for when your blood pressure is classified as dangerous. The bar chart on the display and the scale on the device state which range the blood pressure measured is in. If the systolic and diastolic values are in two different ranges (i.e. systolic is “high, normal” and diastolic is “normal”) the graphic categorisation on the device will always show the higher range i.e. in the above example this would be “high, normal”.

● Blood pressure too low

WARNING

Blood pressure that is too low (hypotension) may be harmful to health, resulting in dizziness or blackouts. Blood pressure is considered to be too low if systolic and diastolic pressure are below 90/60 mmHg respectively (Source: National Health Service, 2023). Seek immediate medical attention if you suddenly suffer from low blood pressure.

● **Reviewing and deleting measurements**

Users

The results of each successful measurement are saved with the date and time. If there are over 120 measurement values, the oldest measurement value will be deleted.

- 1. Use the function button **> 3** to select the desired user (see section: "User Selection").
- 2. Press the settings/memory button **EQ 5**.

Average value

AVG is displayed:

The average of **all saved measurements** for this user is displayed.

- Press the function button **> 3**.

☀ AVG is displayed:

Average of the last 7 days of **morning measurements** is displayed (morning: 5 a.m. – 9 a.m.).

- Press the function button **> 3**.

🌙 AVG is displayed:

Average of the last 7 days of **evening measurements** is displayed (evening: 6 p.m. – 8 p.m.).

Individual measurements

- If you press the function button > **3** again, the display will show the last individual measurement.
- If you press the function button > **3** again, you can review all individual measurements.
- To switch the device back off, press the START/STOP button **1** **4** for approx. 3 seconds.
- To exit the menu, briefly press the START/STOP button **1** **4**.

Deleting measurements

- To delete all measurements stored for a user, please go to the corresponding user memory.
- Press the settings/memory button  **5**. The display will show the average **AVG** **13** of all stored measurements for this user.
- Press and hold the function buttons > **3** and  **5** for approx. 5 seconds.
The display will show **EL 01** for **1** / **EL 02** for **2**.

All measurements for the selected user will be deleted.

● Cleaning and care

-  **ATTENTION! POSSIBLE PROPERTY DAMAGE!** Never hold the product under water as liquid could enter the product and damage it.
- ☐ Clean the device and cuff 2 carefully using a slightly damp cloth.
 - ☐ Do not use cleaners or solvents.

● Storage

-  **ATTENTION! POSSIBLE PROPERTY DAMAGE!** Do not place any heavy objects on the device and cuff 2.
- ☐ If the product will not be used for more than one month, remove both batteries from the device to prevent possible battery leakage.
 - ☐ Store the product in a dry location (relative humidity $\leq 93\%$). Excessive humidity can shorten

the service life or damage the product. Store the product in a location with ambient temperatures between -25 °C and +55 °C.

● Troubleshooting

Error message	Problem	Solution
Er1	No pulse could be recorded.	Please repeat the measurement after a one-minute break. Please refrain from talking or moving during the measurement process.
Er2	The measured blood pressure is outside of the measuring range.	
Er3 	A pneumatic system error has occurred.	Repeat the measurement. Ensure that the cuff  has been connected correctly; refrain from moving and talking.

Error message	Problem	Solution
Er4	An error occurred during the measurement process.	Please repeat the measurement after a one-minute break. Please refrain from talking or moving during the measurement process.
Er5	The inflation pressure is more than 300 mmHg.	Please take a new measurement to check whether the cuff can be properly inflated.
Er6	A system error has occurred.	If this error message appears, please contact customer service.
 LO	The batteries are almost dead.	Insert new batteries into the device.

● Repairing the device

- Do not repair or adjust the device yourself. If you do so, proper function is no longer guaranteed.

- Do not open the device, except for the battery compartment. Failure to comply with this will void the warranty.
- Repairs may only be undertaken by customer service or authorised specialists. Before making a complaint, first check and replace the batteries.

● **Disposal**

The packaging is made entirely of recyclable materials, which you may dispose of at local recycling facilities.

Observe the marking of the packaging materials for waste separation, which are marked with abbreviations (a) and numbers (b) with following meaning: 1–7: plastics/20–22: paper and fibreboard/80–98: composite materials.

The device may not be disposed of with household waste. You may dispose of it using appropriate collection sites in your country. Dispose of the product according to the EC Waste Electrical and Electronic Equipment Directive - WEEE. Please contact your responsible local authority with any questions.

FR

FR

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The product incl. applied parts, manual and packaging materials are recyclable and are subject to extended producer responsibility. Dispose them separately, following the illustrated Info-tri (sorting information), for better waste treatment. The Triman logo is valid in France only.

Environmental damage through incorrect disposal of the batteries/rechargeable batteries!

Batteries/rechargeable batteries may not be disposed of with the usual domestic waste. They may contain toxic heavy metals and are subject to hazardous waste treatment rules and regulations. The chemical symbols for heavy metals are as follows: Cd = cadmium, Hg = mercury, Pb = lead. That is why you should dispose of used batteries/rechargeable batteries at a local collection point.

● Warranty/service

Hans Dinslage GmbH, Riedlinger Straße 28, 89524 Uttenweiler (hereafter referred to as "HaDi") provides a guarantee for this product under the following prerequisites and the product warranty described hereafter.

In case of defects, warranty claims do not affect the legal warranty rights of the customer which emerge from the purchase contract. The customer may exercise their legal warranty rights free of charge. Furthermore, the warranty also applies in spite of obligatory legal liability provisions.

HaDi guarantees the fault-free function and completeness of this product.

The worldwide warranty is 3 years starting with the purchase of the new, unused product by the customer.

This warranty only applies to products that the customer has purchased and used for personal purposes in domestic contexts as a consumer. Subject to German law.

If the product proves to be incomplete or defective in terms of its function during the warranty period and according to the following stipulations, HaDi will provide

a free replacement or repair in accordance with these warranty conditions.

If the customer wishes to make a warranty claim, they should first contact HaDi customer service:

Service Hotline (toll-free):

GB Tel.: 0800 931 0319

IE NI Tel.: 1800 939 235

MT Tel.: 1800 939 235

Email address:

GB service-uk@sanitas-online.de

IE NI service-ie@sanitas-online.de

MT service-mt@sanitas-online.de

Then, the customer will receive additional information about the processing of the warranty claim, e.g. where to send the product free of charge and which documents are required.

If we ask the customer to return the defective product,
please send it to the following address:

NU Service GmbH
Lessingstraße 10b
89231 Neu-Ulm
GERMANY

Warranty claims are only considered if the customer
can submit

- a copy of the invoice/receipt and
 - the original product
- to HaDi or an authorised HaDi partner.

The following is not covered by this warranty

- wear, which is the result of normal product use
- applied parts included with this product, which become worn while used as intended (e.g. batteries, cuffs, seals, electrodes, illuminants, attachments, inhaler accessories);
- products that have been used improperly and/or used, cleaned, stored, or serviced in violation of

the guidelines of the instructions for use, as well as products that were opened, repaired, or retrofitted by the customer or a service centre not authorised by HaDi;

- damage that occurs in transit from the manufacturer and the customer or between the service centre and the customer;
- products purchased used or as factory seconds;
- consequential damage based on a defect of this product (in this case, however, only product liability claims or other mandatory liability provisions may arise).

Repairs or replacements never extend the warranty period.

This device conforms to European standard EN 60601-1-2 (Group 1, Class B, in accordance with CISPR 11, IEC 61000-3-2, IEC 61000-3-3, IEC 61000-4-2, IEC 61000-4-3, IEC 61000-4-4, IEC 61000-4-5, IEC 61000-4-6, IEC 61000-4-7, IEC 61000-4-8, IEC 61000-4-11) and is subject to special precautions regarding electromagnetic compatibility.

Please note that portable and mobile RF communication devices may interfere with this device.

The accuracy of this blood pressure monitor has been thoroughly tested and was developed with regard to a long utilizable service life. If the device is used in medicine, technical inspections using the suitable equipment are to be carried out. Precise information on how to test the accuracy of the device can be requested from our service address.

Note on reporting incidents

For users/patients in the European Union and identical regulation systems (Medical Device Regulation, MDR (EU) 2017/745), the following applies: if, during or as a result of the application of the product, a serious incident occurs, report this to the manufacturer and/or their authorised representative and to the relevant national authority of the member state where the user/patient is located.

The CE declaration of conformity for this product is available at:

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Joytech Healthcare Co.,Ltd.
Shanghai International Holding Corp. GmbH
EU: Hans Dinslage GmbH,