PDF ONLINE  
[www.lidl-service.com](http://www.lidl-service.com)

## SPORTS BACK BRACE

**GB**

### SPORTS BACK BRACE

Instructions for use (2)

**CZ**

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**SK**

### ŠPORTOVÁ BANDÁŽ NA CHRÁTS

Návod na použitie (4)

**SI**

### ŠPORTNA BANDAŽ ZA HRBET

Navodila za uporabo (5)

**HU**

### HÁTBANDÁZS

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**DE AT CH**

### SPORTBANDAGE RÜCKEN

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IAN 424830\_2301

Congratulations on the purchase of your new product. You have chosen a high-quality article. The instructions for use are part of this product. They contain important information on safety and use. Please familiarise yourself with the instructions and safety information before using this product. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the brace and be sure to include them when handing on the product to another person.

## Medical device

**Only intended for private use!**

### Note

The medical device is intended for adults who are able to read, understand and independently follow these instructions for use. A reason for use, as named under Indications, must be present at the user. The item is not intended for groups of persons who are excluded from its application by the sizes and contraindications listed below.

### Scope of delivery

- 1 x back brace
- 1 x instructions for use

### Technical Data

Size	waist circumference
XS - M	66 - 82 cm
M - L	80 - 98 cm
L - XXL	96 - 116 cm

### Intended purpose

Stabilising, warming brace provides support for spine. May help to improve circulation.

### Indications

**To be used as support for:**

- Muscle tension
- Muscular imbalance
- Hypermobility of the lumbar spine
- Sports and physical strains

### Contraindications

**Do not use – or use only after consultation with your doctor – in case of:**

- External injuries
- Bone fractures
- Circulatory disorders
- Heightened instability of the lumbar spine (e.g. spondylolisthesis)
- Neurological deficits such as paralysis of the lower extremities, radiating nerve pain etc.

Quit the application in case

- unforeseen problems arise
- you are unsure about how to use the product

## Safety Advice

### Risk of Injury

- Use the product only in perfect condition, as described and for the specified area of use.
- Do not wear the bandage longer than 2 to 3 hours at a stretch and a maximum of 6 to 8 hours a day.
- Pay attention to a sufficient blood circulation.
- The bandage may only be used on uninjured skin.
- To avoid allergic skin reaction., the skin must be free of grease, gels, creams, oils or similar materials.

### Instructions for disposal

Please dispose of the product in an environmentally-friendly way through an approved disposal centre or your local waste facility. Observe currently applicable regulations.

### Obligation to notify

All serious incidents relating to the product must be reported immediately to the manufacturer and the competent authority.

### 3-year warranty

The product has been created with great care and under constant monitoring. You will receive a 3-year warranty for this product from the date of purchase. Please keep the receipt. The warranty is valid only for material and manufacturing faults and lapses in the event of misuse or improper handling. Your statutory rights, particularly the warranty rights, are not restricted by this warranty. If you have any complaints, please contact the service hotline below or contact us by email. Our service staff will discuss how to proceed further with you as soon as possible. In any case, we will personally advise you. The warranty period is not extended by any repairs under the terms of the warranty, statutory warranty or goodwill. This also applies to replaced and repaired parts. After expiry of the warranty, incurred repairs are chargeable.

Service Germany  
Tel.: +49 (0)40 82 22 44-80  
E-Mail: info@horizonte.de

- Consult instructions for use
- Attention: Always adhere to the stated warning information
- Catalogue number
- Batch designation
- Use by
- Date of manufacture (year): 2023
- Manufacturer
- CE Marking of conformity
- Medical device

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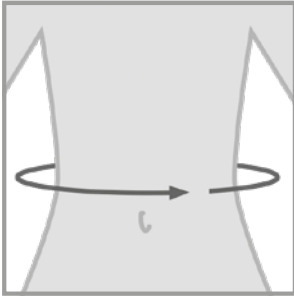
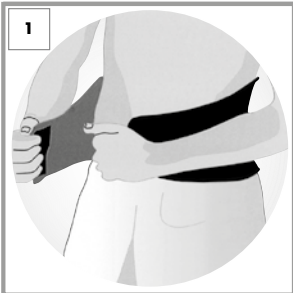
**(GB)** Putting the bandage on

The easiest way to put the bandage on is while standing.

- Lay the bandage around the desired area of the back **(Fig. 1)** and close the Velcro fastener of the bandage **(Fig. 2)**.
- The bandage should be slightly taut, but not uncomfortable.

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**GB** At [www.lidl-service.com](http://www.lidl-service.com) you can download this and many other manuals, product videos and installation software. This QR code will take you directly to the Lidl service page ([www.lidl-service.com](http://www.lidl-service.com)) and by entering the product number (IAN) 424830\_2301 you can open your instructions for use.

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