FUSSGELENKBANDAGE MALLEO-ACTIVE ANKLE SUPPORT MALLEO ACTIVE CHEVILLÈRE MALLEO-ACTIVE

DE AT CH

FUSSGELENKBANDAGE MALLEO-ACTIVE

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GB IE

ANKLE SUPPORT MALLEO ACTIVE

Instructions for use (3-4)

FR BE

CHEVILLÈRE MALLEO-ACTIVE

Mode d'emploi (4-5)

NL BE

ENKELBANDAGE MALLEO-ACTIVE

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PL

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Congratulations on the purchase of your new product. You have chosen a high-quality article. The instructions for use are part of this product. They contain important information on safety and use. Please familiarise yourself with the instructions and safety information before using this product. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the brace and be sure to include them when handing on the product to another person.

Medical Device

Only intended for private use!

Scope of delivery

- 1 x ankle brace
- 1 x instructions for use

Technical Data

| Size | Ankle circumference |
|------|---------------------|
| S | 19 – 21 cm |
| M | 21 – 23 cm |
| L | 23 – 25 cm |
| XL | 25 – 27 cm |

To be worn on the left and right ankles

Intended use

Stabilising brace supports and relieves the ankle

To be used as support for:

- Ankle injuries (e.g. strains, sprains and distortions)
- Joint pain due to degradation (osteoarthritis)
- Mild joint instability
- Tendency to swell
- Talipes valgus problems
- Rheumatoid conditions

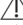
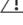
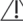
Usage restrictions

 **Do not use – or use only after consultation with your doctor – for:**

- External damage
- Complex chronic ligament instabilities
- Circulatory disorders.
- Lymph drainage disorders
- Eczema or other skin diseases

Safety instructions

Risk of injury

-  If problems occur while wearing the bandage, remove it immediately and consult a doctor.
-  Check the bandage before each use for tears and damage.
-  The product should only be used in perfect condition.
-  Make sure the bandage is comfortable but not too tight.
-  Never put the bandage on so tightly that the blood circulation is restricted.
-  Do not wear the bandage longer than 2 to 3 hours at a stretch and a maximum of 6 to 8 hours a day.
-  Do not wear the bandage while sleeping or during longer rest phases such as long periods of sitting.
-  Wash the areas of skin covered by the bandage thoroughly and regularly.
-  To avoid allergic skin reaction., the skin must be free of grease, gels, creams, oils or similar materials.

Possible side effects

-  If you experience symptoms (such as, e.g. tingling, pain, swelling) while wearing the bandage, remove the bandage immediately!
-  There is a risk of skin irritation and skin irritations! In individual cases, skin irritation and reactions and allergic reactions may occur. Make sure that the stated maximum wearing periods are not exceeded and the areas of skin covered by the bandage are thoroughly and regularly washed. Should any problems or complaints occur, consult a doctor immediately!

Putting the bandage on

- Pull the bandage over the foot from the front up to the ankle.
- To make sure that it is correctly positioned, fold over the top part of the bandage. The edge of the ankle must be positioned in the round part of the pad (**Fig. 1**).
- Make sure that the bandage is worn wrinkle-free (**Fig. 2**).

Cleaning and care

- Before using it for the first time, the bandage should be thoroughly washed.
- After washing, you should thoroughly rinse your bandage.
- After using it two or three times, it should be washed again.
- Wash separately.

Instructions for disposal

Please dispose of the product in an environmentally-friendly way through an approved disposal centre or your local waste facility. Observe currently applicable regulations.

3-year warranty

The product has been created with great care and under constant monitoring. You will receive a 3-year warranty for this product from the date of purchase. Please keep the receipt. The warranty is valid only for material and manufacturing faults and lapses in the event of misuse or improper handling. Your statutory rights, particularly the warranty rights, are not restricted by this warranty. If you have any complaints, please contact the service hotline below or contact us by email. Our service staff will discuss how to proceed further with you as soon as possible. In any case, we will personally advise you. The warranty period is not extended by any repairs under the terms of the warranty, statutory warranty or goodwill. This also applies to replaced and repaired parts. After expiry of the warranty, incurred repairs are chargeable.

Service Germany

Tel.: 0049 (0)40 8222448-0

E-Mail: info@horizonte.de

-  Consult instructions for use
-  Attention: Always adhere to the stated warning information
-  Order number (article number)
-  Batch designation
-  Use by
-  Date of manufacture (year): 2020
-  Manufacturer
-  CE Marking of conformity

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