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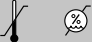
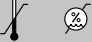
**Information on  
electromagnetic  
compatibility**.....Page 89

## List of pictograms used

	Warns of risks of injury or health hazards		Read instructions for use
	Safety note related to possible damage to the product / components	<b>%SpO<sub>2</sub></b>	Arterial oxygen saturation of haemoglobin (in percent)
	Safety notes	PR bpm	Pulse rate (beats per minute)
	Instructions		Permissible storage, transport and operating pressure

<p>Storage &amp; Transport</p> 	<p>Permissible storage and transport temperature and humidity</p>		<p>Applied part, type BF</p>
<p>Operating</p> 	<p>Permissible operating temperature and humidity</p>		<p>Serial number</p>
	<p>Manufacturer</p>		<p>Mute alarm</p>

	<p>The CE mark certifies compliance with the basic requirements of Medical Devices Directive 93/42/EEC.</p>	<p><b>IP22</b></p>	<p>Device protects against foreign matter <math>\geq 12.5</math> mm and against water falling diagonally</p>
	<p>Improper disposal of batteries can harm the environment!</p>		<p>Dispose of the packaging and pulse oximeter in an environmentally friendly manner!</p>

## Pulse oximeter

### ● Introduction

Familiarise yourself with the pulse oximeter before using it for the first time.

Please carefully read the following instructions for use and the important safety notes. Failure to observe the notes may result in personal injury or property damage. Only use the pulse oximeter as described and for the indicated

purposes. Keep these instructions for use in a safe place. Include all documentation when passing the pulse oximeter to third parties.

### ● **Intended use**

Only use the SPO 55 pulse oximeter on people to measure the arterial oxygen saturation ( $SpO_2$ ) of haemoglobin and the heart rate (pulse rate). The pulse oximeter is suitable for private environments (at home) and healthcare (hospitals, medical facilities).

Any other use not specified above or modification of the pulse oximeter is prohibited and may result in injuries and/or damage the pulse oximeter. The manufacturer assumes no liability for damages caused by improper use.

## ● **Function**

The SPO 55 pulse oximeter serves non-invasive measurement of the arterial oxygen saturation (SpO<sub>2</sub>) and the heart rate (pulse rate). The oxygen saturation indicates the percentage of haemoglobin

in the arterial blood is loaded with oxygen. The respiratory function is therefore an important parameter of assessment.

The pulse oximeter uses two rays with different wavelengths which strike the finger inside the housing to take measurements. A low oxygen saturation rate is primarily caused by illness (respiratory illnesses, asthma, heart failure, etc.). People with a low oxygen saturation rate often experience the following symptoms: shortness of breath, increased heart rate, impaired performance,

nervousness and perspiration. Chronic and known low oxygen saturation requires monitoring with your pulse oximeter under medical supervision. An acute low oxygen saturation, with or without symptoms, requires immediate medical attention. This could be a sign of a life-threatening condition. The pulse oximeter is therefore particularly suited for high-risk patients such as persons with heart disease, asthma, as well as for athletes and healthy persons in high altitudes (e.g. mounting climbers, skiers or amateur pilots).

## ● Improper use

**⚠ WARNING!** The pulse oximeter shows an instant measurement but cannot be used for continuous monitoring. The pulse oximeter does not have an alarm and is therefore not suitable for analysing medical results. Do not self-diagnose or self-treat based on the measurements without consulting your attending physician. Do not start new medication without consulting a physician and do not modify the type and / or dosage of current medications.

The pulse wave and the pulse bar do not allow an assessment of the intensity of the pulse or circulation at the measured site but solely visualise the current signal visualisation at the site measured, but does not allow certain pulse diagnosis.

## ● **Parts description**

- 1 Display
- 2 Function button 
- 3 Finger opening
- 4 Battery cover

## **Display**

- 5 Oxygen saturation (value in percent)
- 6 Pulse rate (value in beats per minute)
- 7 Pulse wave (plethysmographic waveform)
- 8 Pulse bar
- 9 Battery level

## ● **Scope of delivery**

- 1 SPO 55 pulse oximeter
- 2 1.5V AAA batteries
- 1 Set of instructions for use
- 1 Quick start guide

## ● Technical data

Model no.: SPO 55

Measuring method: non-invasive measurement of the arterial oxygen saturation of haemoglobin and pulse frequency of the finger

Measuring range: SpO<sub>2</sub> 0-100%,  
Pulse 30-250 beats/  
minute

Accuracy: SpO<sub>2</sub> 70-100%, ± 2%,

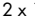
Dimensions: 58.5 mm x 31 mm x 32 mm  
(L x W x H)

Weight: approx. 27 g (without  
batteries)

Sensor system for  
measuring SpO<sub>2</sub>: red light (wavelength  
660 nm); infrared (wave-  
length 905 nm); silicon  
receiver diode

Admissible operating conditions: +10 °C to +40 °C, ≤ 75% relative humidity, 700–1060 hPa ambient pressure

Admissible storage conditions: -40 °C to +60 °C, ≤ 95% relative humidity, 500–1060 hPa ambient pressure

Power supply: 2 x 1.5 V  AAA batteries  
Battery life: 2 AAA batteries provide approx. 2 years of use at 3 measurements per day (60 seconds each).

Classification: IP22, applied part type BF  
Expected service life: 3 years  
Memory spaces: 100

Data transfer  
via *Bluetooth*<sup>®</sup>  
wireless

technology: The pulse oximeter uses *Bluetooth*<sup>®</sup> low energy technology, frequency band 2.402–2.480 GHz, the maximum transmitting power distributed in the frequency band < 2.87 dBm, compatible with *Bluetooth*<sup>®</sup> ≥ 4.0 smartphones/ tablets

For a list of compatible smart-phones, information about the “HealthForYou” app and software, and further details about the devices please visit:  
[www.healthforyou.lidl](http://www.healthforyou.lidl)

System requirements  
of the “HealthForYou”

web version: - MS Internet Explorer  
- MS Edge

- Mozilla Firefox
  - Google Chrome
  - Apple Safari
- (respective current versions)

#### System requirements of the "HealthForYou"

App: iOS version 10.0 and up,  
Android™ Version 5.0 and up,  
Bluetooth® 4.0.

The serial number is located inside the battery compartment.

Technical specifications subject to change without notice due to updates.

### **Important safety notes**

**⚠ WARNING!** This product is not intended to be used by persons (including children) with physical, sensory or mental impairments or lacking

experience and/or knowledge unless supervised or instructed on its use by a person responsible for their safety. Children should be supervised to ensure they do not play with the device.

- Do NOT use the pulse oximeter
  - if you are allergic to rubber products.
  - if the device or the respective finger is moist.
  - on toddlers or infants.
  - during an MRI or CT scan.
  - on the arm with the cuff whilst measuring the blood pressure.

- on fingers with painted nails, dirt or adhesive bandages.
- very large fingers which cannot easily be inserted in the device without using force (fingertip: width approx. > 20 mm, thickness approx. >15 mm).
- on fingers with anatomic deformities, oedemas, scars or burns.
- on too small of fingers such as in toddlers (width approx. < 10 mm, thickness approx. < 5 mm).

- in areas the patient is restless (e.g. trembling).
- near flammable or explosive gas mixtures.
- Prolonged use of the pulse oximeter may cause pain in persons with circulatory disorders. Therefore do not use the pulse oximeter on the same finger for longer than approx. 2 hours.
- Do not look directly into the housing during measurement. The red light and the invisible infrared light of the pulse oximeter is harmful to the eye.
- Failure to observe the following instructions may result in faulty or failed measurements:
  - Do not use nail polish, artificial nails or other cosmetic on the monitored finger.
  - Be sure the fingernail on the monitored finger is short enough so the finger pad covers the sensor elements inside the housing.
  - Keep the hand, finger and body still during measurement.
  - Oxygen saturation and heart rate measurements may be false or impossible in persons with arrhythmia.

- The pulse oximeter will display high measurements when carbon monoxide poisoning is present.
  - To prevent false readings, keep bright light sources (e. g. fluorescent lamps or direct sunlight) out of the immediate surroundings of the pulse oximeter.
  - Measurements may be faulty or false in persons with low blood pressure, jaundice or taking vasoconstriction medications.
  - False measurements should be expected in patients who have received contrast agents in the past, and in patients with haemoglobin abnormality. This particularly applies to carbon monoxide poisoning and methaemoglobin poisoning caused by e.g. local anaesthetics or due to methemoglobin reductase deficiency.
- Protect the pulse oximeter from dust, shock, moisture, and extreme temperatures.
  - Keep out of areas with explosive substances.

- Routinely inspect the pulse oximeter to ensure the device is not visibly damaged before use and the battery level is sufficient. When in doubt, do not use and contact service or an authorised retailer.
- Do not use additional parts not recommended or sold as accessories by the manufacturer.
- Never open or repair the device or the proper function cannot be guaranteed. Non-compliance will void the warranty.
- Please contact service or an authorised retailer for repairs.

## Battery safety notes

- **DANGER TO LIFE!** Keep batteries out of the reach of children. Seek immediate medical attention if swallowed!
-  **EXPLOSION HAZARD!** Never recharge disposable batteries. Never short-circuit and/or open batteries. The battery may overheat, ignite or explode.

- Never throw batteries into fire or water.
- Never exert mechanical stress on batteries.

### **Risk of battery leakage**

- Avoid extreme conditions and temperatures which may affect batteries, e.g. radiators / direct sunlight.
- If batteries / rechargeable batteries have leaked, avoid contact with skin, eyes and mucous membranes with the chemicals!

Flush immediately the affected areas with fresh water and seek medical attention!

### **WEAR SAFETY GLOVES!**

- Leaked or damaged batteries can cause burns on contact with the skin. Therefore wear suitable safety gloves in this case.
- In the event of a battery leak, immediately remove the batteries from the product to prevent damage.

- Only use the same type of batteries. Do not mix old and new batteries!
- Remove the batteries if the product will not be used for an extended period.

### **Risk of product damage**

- Only use the specified battery type!
- Insert the batteries according to the polarity marking (+) and (-) on the battery and the product.

- Clean the contacts on the battery and inside the battery compartment before inserting!
- Immediately remove drained batteries from the product.
- Do not use rechargeable batteries.

### **● Initial use**

- Verify the parts listed in the scope of delivery are complete.
- Remove all packaging material.

## ● **Inserting / Replacing batteries**

- Slide the battery cover **4** open (Fig. C).
- Insert the two included batteries (with the correct polarity) in the pulse oximeter (Fig. D). Please note the marking inside the battery compartment.
- Close the battery compartment **4** (Fig. E).

## ● **“HealthForYou” App / Web version**

Download the free app “HealthForYou” from the Apple App Store and Google Play.

- Follow the installation instructions on the screen.
- Alternatively, you can also visit the website at [www.healthforyou.lidl](http://www.healthforyou.lidl).

## ● Use

The function button **2** on the pulse oximeter has 4 functions:

On	Press and briefly hold the function key <b>2</b> to switch the pulse oximeter on.
<i>Activate and deactivate Bluetooth®</i>	<ul style="list-style-type: none"><li>• Briefly press the function key <b>2</b> to switch on the pulse oximeter.</li><li>• Hold the function key <b>2</b> on the switched on pulse oximeter pressed for 5 seconds to proceed to <i>Bluetooth®</i> settings. "📶 ON" or "📶 OFF" will appear on the display.</li></ul>

<p>Activate and deactivate Bluetooth®</p>	<ul style="list-style-type: none"> <li>• Briefly press the function key <b>2</b> to activate (⌘ ON) or deactivate (⌘ OFF) Bluetooth®.</li> <li>• To return to measuring, hold the function key <b>2</b> pressed for 5 seconds.</li> <li>• To switch off the pulse oximeter, wait 10 seconds. The pulse oximeter will then automatically switch off.</li> </ul>
<p>Transferring measurements to the “HealthForYou” app and synchronising the time and date</p>	<p>There are two options for transferring data to the “HealthForYou” app. Bluetooth® must be activated (⌘ ON) on your smartphone and on the pulse oximeter for both options.</p> <p><b>Note:</b> The “HealthForYou” app must be open for transfer to occur.</p>

<p>Transferring measurements to the "HealthForYou" app and synchronising the time and date</p>	<p><b>Option 1:</b> Hold the function key <b>2</b> on the switched off pulse oximeter pressed for 5 seconds to transfer the data to the app.</p> <p><b>Option 2:</b> After measurements are taken, the data will automatically be sent to the app. If <i>Bluetooth</i><sup>®</sup> is deactivated (✖ OFF), the notification ✖ OFF will be displayed after your measurement has been carried out.</p>
<p>Brightness function</p>	<p>To select your desired display brightness, hold down the function button <b>2</b> for slightly longer during operation.</p>

## ! Note

The orientation of the display (landscape, portrait) is automatic. This means that it is always easy to read the values on the display, regardless of how you hold the pulse oximeter.

## ● Measurements

- Slide one finger into the finger opening **3** on the pulse oximeter. Hold the finger still (Fig. F).
- Press the function button **2**. The pulse oximeter will now begin the measurement. Do not move during measurement (Fig. G).
- After a few seconds the display **1** will show your measurements (Fig. H).
- After removing your finger from the pulse oximeter, the device will automatically switch off after approx. 10 seconds.

## ● Transferring measurements to the “HealthForYou” app via Bluetooth®

**Note:** Bluetooth® must be activated (⌘ ON) for transfer to occur.

**Note:** The “HealthForYou” app must be open for transfer to occur.

The time and date will be synchronised with the smartphone during every data transfer.

## ⓘ **Note**

As soon as you have connected the pulse oximeter with your smartphone, the current date and time are updated on the pulse oximeter.

We therefore recommend connecting the pulse oximeter with your smartphone before taking your first measurement, so that all measurements are saved with the current date and correct time.

To synchronise measurements with your smartphone via *Bluetooth*<sup>®</sup>:

- Activate *Bluetooth*<sup>®</sup> in your smartphone settings, open the “HealthForYou” app and follow the instructions.
- Select and connect to the SPO 55 in the settings menu of the app. A randomly generated six digit PIN code will be displayed on the pulse oximeter. At the same time, an entry field will appear on your smartphone where you should enter this code.

There are two options for synchronising data to your smartphone. *Bluetooth*<sup>®</sup> must be activated (📶 ON) on your smartphone and on the pulse oximeter for both options. The “HealthForYou” app must also be open on the smartphone.

**Option 1:** Hold the function key 2 on the switched off pulse oximeter pressed for 5 seconds. “SYNC” will flash on the display 1. The device will now attempt to connect to the app for 30 seconds. Once connected, “SYNC” will stop flashing. All stored measurements will be

automatically transferred to the app. The pulse oximeter will then automatically switch off.

**Option 2:** After measurements are taken, the data will automatically be sent to the app. "SYNC" will flash on the display .

The device will attempt to connect to the app for 30 seconds.

Once connected, "SYNC" will stop flashing. All stored measurements will be automatically transferred to the app. The pulse oximeter will then automatically switch off.

If *Bluetooth*<sup>®</sup> is deactivated (🔌 OFF), the notification 🔌 OFF will be displayed after your measurement has been carried out.

## ● Analysing measurements

 **WARNING!** The following table for assessing measurements does NOT apply to persons with certain pre-existing conditions (e.g. asthma, heart failure, respiratory disorders) and in altitudes over 1500 metres.

If you have a pre-existing condition, always consult your physician to analyse your measurements.

SpO <sub>2</sub> (oxygen saturation) measurement in %	Classification / required action
99-94	Normal range
93-90	Low range: medical consultation recommended
< 90	Critical range: medical consultation urgently required

## ● Altitude-related drop in oxygen saturation

**Note:** The following table provides information on how different altitudes affect the oxygen saturation and how this affects the human organism. The following table does NOT apply to persons with certain pre-existing conditions (e.g. asthma, heart failure, respiratory disorders, etc.). Persons with pre-existing conditions may already experience symptoms (e.g. hypoxia) in low altitudes.

<b>Altitudes</b>	<b>Expected SpO<sub>2</sub> value (oxygen saturation) in %</b>	<b>Affects on people</b>
1500-2500 m	> 90	No anoxia (typically)
2500-3000 m	~ 90	Anoxia, acclimatisation recommended

Source: Hackett PH, Roach RC: High-Altitude Medicine. In: Auerbach PS (ed): Wilderness Medicine, 3rd edition; Mosby, St. Louis, MO 1995; 1-37.

## ● Cleaning and care

### **ATTENTION! POSSIBLE PROPERTY**

**DAMAGE!** Do not use high pressure sterilisation on the pulse oximeter!

Never hold the pulse oximeter under water, as liquid may enter and damage the pulse oximeter.

- Clean the housing and the rubberised inner surface of the pulse oximeter with a soft

cloth dampened with surgical spirits after every use.

- Replace the batteries if the display  on the pulse oximeter shows the battery  is low.
- If the pulse oximeter will not be used for more than one month, remove both batteries from the device to prevent possible battery leakage.

## ● Storage

### **ATTENTION! POSSIBLE PROPERTY**

**DAMAGE!** Store the pulse oximeter in a dry location (relative humidity  $\leq 95\%$ ). High humidity can shorten the life or damage the pulse oximeter. Store the pulse oximeter in a location with ambient temperatures between  $-40\text{ }^{\circ}\text{C}$  and  $60\text{ }^{\circ}\text{C}$ .

## ● Troubleshooting

<b>Problem</b>	<b>Possible cause</b>	<b>Solution</b>
The pulse oximeter doesn't display measurements.	The batteries inside the pulse oximeter are drained.	Replace the batteries.
	The batteries were inserted incorrectly.	Reinsert the batteries. If measurements are not displayed after inserting the batteries correctly, please contact service.

<b>Problem</b>	<b>Possible cause</b>	<b>Solution</b>
Pulse oximeter shows interrupted measurements or severe jumps in the measurement.	Circulation problems in the finger being measured.	Refer to warnings and safety notes.
	The finger being measured is too large or too small.	The fingertip must be the following size: between 10 and 20 mm wide. Between 5 and 15 mm thick.
	The finger, hand or body are moving.	Keep the finger, hand and body still during measurement.
	Arrhythmia.	Seek medical attention.

<b>Problem</b>	<b>Possible cause</b>	<b>Solution</b>
Unable to transfer the measurements.	The "HealthForYou" app is not open or <i>Bluetooth</i> <sup>®</sup> is off in the smartphone settings.	Activate <i>Bluetooth</i> <sup>®</sup> on your smartphone and start the app.
	The batteries in the pulse oximeter are low or drained.	Replace the batteries.
	The device is still on.	Wait for the device to switch off.
	<i>Bluetooth</i> <sup>®</sup> is not switched on.	Activate <i>Bluetooth</i> <sup>®</sup> (📶 ON).

## ● Disposal

The packaging is made entirely of recyclable materials, which you may dispose of at local recycling facilities.

Observe the marking of the packaging materials for waste separation, which are marked with abbreviations (a) and numbers (b) with following meaning:  
1-7: plastics / 20-22: paper and fibreboard /  
80-98: composite materials.

Contact your local refuse disposal authority for more details of how to dispose of your worn-out product.

To help protect the environment, please dispose of the product properly when it has reached the end of its useful life and not in the household waste. Information on collection points and their opening hours can be obtained from your local authority.

Per Directive 2006/66/EC and its amendments, defective or used batteries must be recycled. Return the batteries and/or the product to a collection site.

### **Improper disposal of batteries can harm the environment!**

Never dispose of batteries in your household bin. They may contain toxic heavy metals and are subject to hazardous waste regulations. The chemical symbols of the heavy metals are: Cd = cadmium,

Hg = mercury, Pb = lead. Therefore dispose of used batteries through your local collection facilities.

## **● Warranty / Service**

The pulse oximeter is covered by a 3 year warranty from the date of purchase for defects in material and manufacture.

The warranty does not cover:

- Damage due to improper use.
- Wear parts.

- Damage caused by the customer.
- If the device has been opened by an unauthorised repair shop.

This warranty does not affect the customer's legal rights. The customer must provide proof of purchase in order for any warranty claims within the warranty period to be honoured. The warranty claim must be filed with Hans Dinslage GmbH, Uttenweiler, Germany, within 3 years from the date of purchase.

**In the event of a complaint, please contact our Service Department listed below:**

**Service Hotline (toll-free):**

**0800 93 10 319**

**(Mon–Fri: 8 am–6 pm)**

**service-uk@sanitas-online.de**

IE

 **1800 93 92 35**

**(Mon – Fri: 8 am – 6 pm)**

 **service-ie@sanitas-online.de**

If we ask you to return the defective pulse oximeter, please send the product to the following address:

NU Service GmbH  
Lessingstraße 10b  
DE-89231 Neu-Ulm / GERMANY

**IAN 322246**

This device complies with European Standards EN60601-1 and EN60601-1-2 (conformity with CISPR 11, IEC 61000-4-2, IEC 61000-4-3, IEC 61000-4-8) and is subject to special precautions with respect to electromagnetic compatibility. Please note that portable and mobile HF communication devices may interfere with this device. You can request further information from the customer service address below.

This device complies with EU Medical Devices Directive 93/42/EEC, the Medical Devices Act and standard DIN EN ISO 80601-2-61 (Medical electrical equipment–Particular requirements for basic safety and essential performance of pulse oximeter equipment).

We hereby confirm that this product complies with the European RED Directive 2014/53/EU. The CE declaration of conformity for this product is available at: [www.beurer.com/web/we-landingpages/de/cedclarationofconformity.php](http://www.beurer.com/web/we-landingpages/de/cedclarationofconformity.php).

## ● **Information on electromagnetic compatibility**

The device is suitable for use in all environments that are listed in these instructions for use, including in the home.

In the presence of electromagnetic disturbances, the device may under some circumstances only be useable to a limited extent. This can result in

error messages or the breakdown of the display / device, for example.

The use of the device immediately next to other devices or stacked with other devices is to be avoided, as this can result in faulty operation.

If it is necessary to use the device in such a way in spite of this, this device and the other device should be monitored to make sure that they are working correctly.

The use of equipment other than that which has been specified or made available by the manufacturer of this device can lead to electromagnetic interference or the reduced electromagnetic stability of the device and can lead to faulty functioning.

Keep portable RF communication devices (including peripheral equipment such as antenna cables or external antennae) at least 30 cm away from all device parts, including all cables supplied. Failure to observe these instructions may lead to

a reduction in the performance features of the device.

Failure to observe these instructions may lead to a reduction in the performance features of the device.

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*Bluetooth*<sup>®</sup> and the related logo are registered trademarks of Bluetooth SIG, Inc. Use of these brands by Hans Dinslage GmbH is under license. All other brands and trade names are the property of the respective owners.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries. Google Play and the Google Play logo are trademarks of Google LLC. Android is a trademark of Google LLC.

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