## POSTURE TRAINER

(GB)

### POSTURE TRAINER

Instructions for use

(SE)

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Bruksanvisning

(PL)

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(EE)

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Kasutusjuhend

(FI)

### RYHTITUKI

Käyttöohje

(DK)

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(LT)

### LAIKYSENOS TRENERUOKLIS

Naudojimo instrukcija

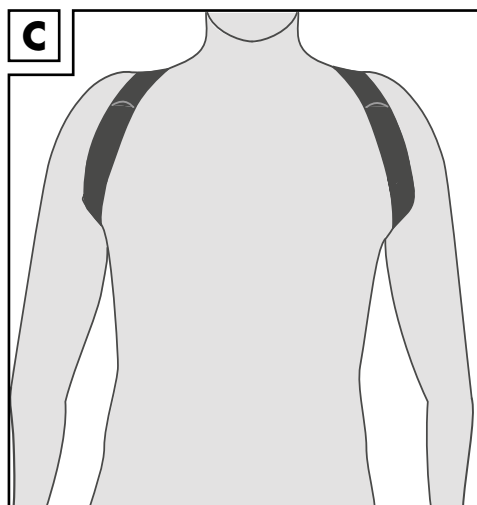
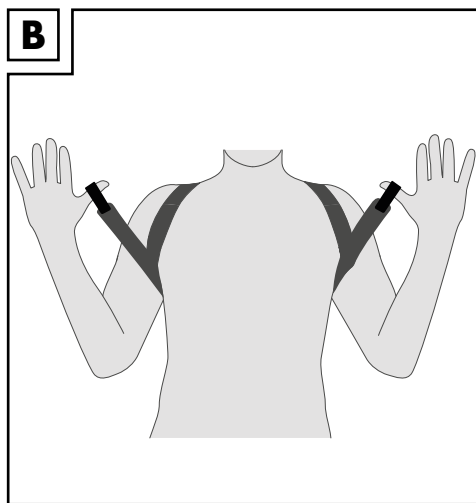
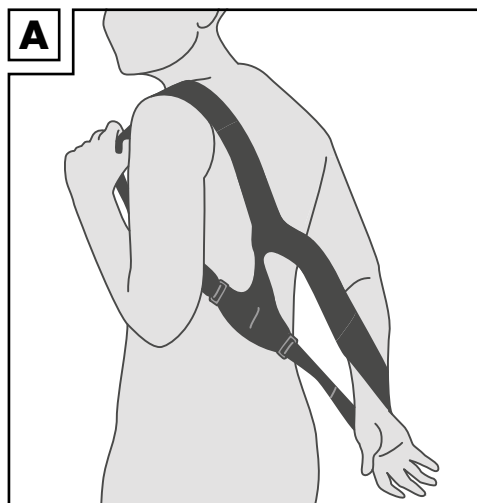
(LV)

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IAN 495197\_2504

Congratulations!

You have chosen to purchase a high-quality product. Familiarise yourself with the product before using it for the first time.

### **Read the following instructions for use carefully.**

Use the product only as described and only for the given areas of application. Store these instructions for use carefully. When passing the product on to third parties, please also hand over all accompanying documents.

Class I medical device – This product is compliant with the Regulation (EU) 2017/745 (MDR) concerning medical devices. Please use our service if you have questions about the product or have trouble reading these instructions.

## **Package contents**

1 x posture trainer  
1 x instructions for use

## **Technical data**

| Size | Chest measurement |
|------|-------------------|
| S–M  | 84–100cm          |
| L–XL | 100–116cm         |

## **Symbols used**

Follow the instructions for use

Date of manufacture (year-month):  
2025-09

Manufacturer

Importer. Indicates the company that imports or has imported the medical device at the location

Usable until

Batch number

Model number

Product number

Medical device

Authorised representative in  
Switzerland

Please ensure that you take note of the warning notices or safety information in the instructions for use

Instructions for use as PDF on the Lidl  
Service website

Clear product identification. Displays a carrier that contains information on a unique device identification (UDI)

This product is compliant with the Regulation (EU) 2017/745 on medical devices.

## **Intended use**

This product is manufactured for private use only and is reusable.

It is not a sports product and is not suitable for wearing during sport. Wearing the product regularly can support an upright posture or posture correction.

### **Indications:**

- for use with muscle tension and muscular imbalances

### **Do not use during or in the case of:**

- uncertainty about use
- acute injuries
- problems that arise
- circulatory disorders

## Safety information

**Important: read these instructions for use carefully and keep them in a safe place!**

### Life-threatening hazard!

- Never leave children unattended with the packaging materials. There is a risk of suffocation.

### Risk of injury for children!

- Children must not be allowed to play with the product. Specifically inform children that the product is not a toy.
- Bear in mind that children are playful and like to experiment. Avoid situations and behaviours that are not appropriate for the product.
- The product must be stored somewhere that is inaccessible to children when not in use.

### Risk of injury!

- No modifications may be made to the product!
- If problems occur during use, remove the support immediately and consult a doctor.
- Check the product for cracks and damage before each use.
- Only use the product if it is in perfect condition, as described and for the specified areas of use.
- Wear a T-shirt or long-sleeved shirt under the product.
- Never put the product on so tightly that the blood circulation is impaired.
- Do not wear the product for more than two to three hours at a time and a maximum of six to eight hours a day.
- Do not wear the product while sleeping.
- The product may only be used on skin that is not injured.
- To avoid allergic skin reactions, the skin must be free of gels, greases, ointments, oils or similar substances.

## Putting on the product

**Note:** the product is easiest to put on while standing.

**Note:** the hook and loop fasteners should be unfastened before putting on the harness, but the straps should be fed through the buckles.

1. Pull the product on like a jacket by passing both arms through the loops (Fig. A).
2. Pull the product tight using the loops at the ends of the straps (Fig. B).

**Note:** the product should sit comfortably and slightly taut on the body.

3. Secure the hook and loop fasteners on the respective shoulder strap (Fig. C).

## Storage, cleaning

When not in use, always store the product clean and dry at room temperature. **IMPORTANT!** Never clean the product with harsh cleaning agents.

- Before using the product for the first time, you should wash it thoroughly. Close the hook and loop fastener before washing.
- After every two or three uses, another wash cycle is recommended.
- After washing, you should rinse the product thoroughly.
- Wash separately or with similar colours.

## Disposal

Dispose of the product and packaging materials in accordance with current local regulations. Store the packaging materials (foil bags, for example) out of the reach of children. For further information about disposal of the product no longer needed, contact your local council. Dispose of the product and the packaging in an environmentally friendly manner.

The Recycling Code distinguishes different materials to be returned for recycling.

The Code consists of the recycling symbol for the recycling process and a number that identifies the material.

## Notes on the guarantee and service handling

The product was produced with great care and under continuous quality control. DELTA-SPORT HANDELSKONTOR GmbH gives private end customers a three-year guarantee on this product from the date of purchase (guarantee period) in accordance with the following provisions. The guarantee is only valid for material and manufacturing defects. The guarantee does not cover parts that are subject to normal wear and tear and that are thus considered wear parts (e.g. batteries) and it does not cover fragile parts such as switches or parts that are made of glass.

Claims under this guarantee are excluded if the product has been used incorrectly, improperly, or contrary to the intended purpose, or if the provisions in the instructions for use were not observed, unless the end customer proves that a material or manufacturing defect exists that was not caused by one of the aforementioned circumstances.

Claims under the guarantee can only be made within the guarantee period by presenting the original sales receipt. Please therefore keep the original sales receipt. The guarantee period is not extended by any repairs carried out under the guarantee, under statutory guarantees, or as a gesture of goodwill. This also applies to replaced and repaired parts.

If you wish to make a claim please first contact the service hotline mentioned below or contact us by e-mail. If there is a guarantee case, then the product will be repaired or replaced free of charge to you or the purchase price will be refunded, depending on our choice. There are no further rights from the guarantee.

Your legal rights, in particular guarantee claims against the respective seller, are not limited by this guarantee.

IAN: 495197\_2504

Service Great Britain

Tel.: 0800 404 7657

E-Mail: [deltasport@idl.co.uk](mailto:deltasport@idl.co.uk)

### Important note

All serious incidents that occur in connection with the product must be reported to the manufacturer and the pertinent authority of the member state in which the user and/or patient resides. Please consult the contact data for the company DELTA-SPORT in the instructions for use under "Notes on the guarantee and service handling".

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LIDL Schweiz DL AG.